

HLA-C14 2020 7K9

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-C allele ^{1,2,3}																																	
*14:02:01:01-14:02:01:17, 14:02:04-14:02:09, 14:02:11-14:02:20, 14:02:22-14:02:31, 14:02:33-14:02:34, 14:30, 14:36-14:37, 14:39-14:40, 14:42, 14:47:02N, 14:50-14:52, 14:56, 14:59, 14:62, 14:65, 14:67-14:69, 14:71-14:72, 14:75-14:76, 14:80, 14:83-14:84, 14:89, 14:91, 14:94-14:104, 14:106-14:107, 14:109, 14:111, 14:114-14:115	1				5				9																						31		
*14:02:02, 14:02:21, 14:82	1				5																										31		
*14:02:03		2		4	5				9																							31	
*14:02:10, 14:85, 14:88	1				5				9																								
*14:02:32, 14:45, 14:63, 14:105Q	1								9																							31	
*14:03:01:01-14:03:01:02, 14:03:03, 14:38, 14:61, 14:112-14:113		2		4	5			8	9																								
*14:03:02	1				5			8	9																								
*14:04	1	3			5		7		9																							31	
*14:05	1					6			9																							31	
*14:06	1								9	10																						31	
*14:07N	1				5				9		11																					31	
*14:08		2		4					9			12																				31	
*14:09	1					6																										31	
*14:10		2		4	5			8	9				13																				
*14:11	1				5				9					14																		31	
*14:12, 14:108	1		3		5				9						15																	31	
*14:13	1				5				9							16																31	
*14:14, 14:46:01-14:46:02 ⁴	1				5				9				13																			31	
*14:15, 14:87	1								9	10				14																		31	
*14:16	1								9								17															31	
*14:17	1				5													18														31	
*14:18	1				5				9										19														
*14:19	1				5				9												20											31	
*14:20	1				5				9													21										31	
*14:21N	1				5				9	10																						31	
*14:22	1			4	5			8	9								17																
*14:23	1				5				9			12																				31	
*14:24:01-14:24:02, 14:31 ⁴	1				5				9													22										31	
*14:25	1				5				9														23									31	
*14:26	1				5				9									18														31	
*14:27	1				5				9								17															31	
*14:28:01	1																				20											31	
*14:28:02	1																												29		31		

Negative Control

<p>*01:17, 01:23, 01:128, 01:152, 01:157, 06:11, 06:82, 06:210, 06:252, 07:01:01-07:01:02:08, 07:01:04-07:01:40, 07:01:42-07:01:73, 07:01:75-07:01:92, 07:02:01:01-07:02:07, 07:02:10-07:02:28, 07:02:30-07:02:31, 07:02:33-07:02:40, 07:02:42-07:02:86, 07:02:88-07:02:90, 07:02:92-07:06:03, 07:10-07:17:02, 07:17:04-07:19, 07:22-07:33N, 07:35-07:38:02, 07:41-07:63, 07:65-07:68, 07:70-07:71, 07:74-07:75, 07:77-07:91, 07:93:01-07:95, 07:97-07:107, 07:109:01-07:126, 07:128-07:151, 07:153-07:155, 07:157-07:171, 07:173-07:176, 07:178-07:209, 07:211-07:222, 07:224-07:237, 07:239-07:241, 07:243-07:262, 07:264N-07:266, 07:268-07:294, 07:296-07:314:03, 07:316-07:321, 07:324-07:327, 07:329N-07:334, 07:336-07:356, 07:358-07:360, 07:362-07:389, 07:391-07:401, 07:403-07:404, 07:407-07:414, 07:417, 07:419-07:492, 07:494Q-07:558:01:02, 07:560-07:561, 07:564-07:577, 07:579-07:582Q, 07:584-07:597, 07:599-07:655, 07:657-07:722, 07:724-07:834, 07:836-07:837, 12:16:02, B*07:13, B*07:15, B*07:160, B*08:123, B*08:145, B*42:18</p>			<p>7</p>							
<p>*01:48, 03:02:08, 03:03:32, 03:04:27, 03:04:30, 03:04:59, 03:04:64, 03:05, 03:14, 03:25, 03:40:02, 03:43:02, 03:58, 03:86-03:87:01, 03:89-03:90, 03:94, 03:102-03:103, 03:119:01, 03:122, 03:128, 03:135, 03:143, 03:153, 03:167, 03:178, 03:190, 03:202, 03:209, 03:220, 03:240, 03:262-03:263:03, 03:267, 03:276, 03:296:01-03:296:02, 03:300, 03:330-03:331, 03:335, 03:338, 03:346, 03:353, 03:355, 03:361, 03:386, 03:391N, 03:402, 03:407, 03:412, 03:439, 03:447N, 03:464, 03:474-03:475, 03:482, 03:493-03:494, 15:43, B*07:234, B*15:315, B*18:126, B*39:138, B*46:77, B*50:56, B*55:30</p>			<p>8</p>							<p>Negative Control</p>

<p>*02:87, 07:583, 07:723, 08:01:01:01-08:01:06, 08:01:08-08:02:06, 08:02:08-08:02:10, 08:02:12- 08:09, 08:11-08:13, 08:15:01-08:33:01, 08:33:03- 08:57, 08:59-08:63, 08:65-08:79, 08:81-08:94, 08:96- 08:102, 08:104, 08:106-08:118, 08:120-08:133, 08:135-08:154, 08:156-08:162, 08:164-08:182, 08:184-08:203, 08:205-08:206, 12:02:01-12:02:10, 12:02:12-12:02:18, 12:02:20-12:02:22, 12:02:24- 12:02:27, 12:02:30-12:03:03, 12:03:05-12:03:08, 12:03:10-12:03:19, 12:03:21-12:03:23, 12:03:25- 12:03:33, 12:03:35-12:03:49, 12:03:51-12:03:61, 12:06-12:08, 12:11-12:15, 12:17-12:20, 12:22-12:26, 12:28-12:32, 12:34-12:40, 12:42Q-12:53, 12:56- 12:59, 12:61-12:69, 12:73-12:103, 12:105N-12:114, 12:116-12:122, 12:124-12:134, 12:136-12:139, 12:141-12:145, 12:148N-12:153, 12:155Q, 12:157- 12:187, 12:189-12:194, 12:195:02, 12:196-12:208, 12:210-12:226, 12:228-12:238, 12:240-12:268, 12:270N-12:278, 12:280-12:304, 16:01:01:01- 16:01:01:15, 16:01:03-16:01:05, 16:01:07-16:01:15, 16:01:17-16:01:25, 16:01:27-16:01:36, 16:04:01:01- 16:04:01:03, 16:04:03-16:04:05, 16:06-16:08, 16:10- 16:11, 16:13-16:18, 16:20-16:24, 16:26-16:33, 16:35- 16:36, 16:38-16:45, 16:49-16:56, 16:58-16:59, 16:61- 16:62, 16:64-16:68, 16:72-16:73, 16:75-16:76, 16:78- 16:79, 16:81-16:83, 16:87, 16:92-16:93, 16:95- 16:98, 16:100, 16:105, 16:109-16:114, 16:116, 16:118-16:119, 16:122, 16:124-16:125, 16:127- 16:131, 16:135, 16:137-16:139, 16:141-16:142, 16:146, 16:148-16:149, 16:151-16:152, 16:154, 16:157-16:162, 16:164</p>	4							31	
<p>*02:106, 15:163, B*07:380 *02:182, B*15:06, B*15:27:01-15:27:04, B*15:84, B*15:109, B*15:195, B*15:327, B*15:344, B*15:398</p>			12				29		Negative Control

HLA-C14 2020 7K9

*03:27, 07:20:01-07:20:02, 07:96:01-07:96:02, 07:127:01-07:127:02, 07:263, 07:390, 07:562, 07:578			7	8															
*03:32, 03:60			6	8					16										
*03:38:01-03:38:02, 03:69, 03:246, 03:274, 03:431			6	7	8														
*03:45			6						15	16									
*03:88, 03:410			6	8									20						
*03:130			6	7															
*03:136			6	7	8					16									
*03:163, 03:461			6	7					15										
*03:231			6						13										31
*03:247			6	8													26		
*03:271				8	9	10							21						
*03:302				8	9														
*03:378, 03:441			6	8				12											
*04:01:01:01-04:01:01:29, 04:01:01:31-04:01:02, 04:01:04, 04:01:06, 04:01:08, 04:01:10-04:01:22, 04:01:24-04:01:28, 04:01:30-04:01:32, 04:01:34-04:01:61, 04:01:63-04:01:74, 04:01:76, 04:01:78-04:01:120, 04:04:01:01-04:04:01:02, 04:05, 04:07:01, 04:08-04:10, 04:12-04:14, 04:18-04:20, 04:23-04:25, 04:28, 04:30-04:32, 04:34-04:35:02, 04:38-04:41, 04:43-04:51, 04:53, 04:57-04:70, 04:72-04:79, 04:81-04:89, 04:91-04:94:01, 04:95N-04:97, 04:99, 04:101-04:102, 04:104-04:106, 04:108-04:132, 04:134-04:139, 04:141-04:144, 04:146, 04:148-04:159, 04:161-04:165, 04:167-04:168, 04:170N, 04:173N-04:177, 04:179, 04:181-04:189, 04:191N-04:197, 04:199-04:201:02, 04:203N-04:205N, 04:207-04:213, 04:215N-04:216, 04:218-04:219, 04:221, 04:223:02, 04:225N-04:229, 04:232-04:235, 04:237-04:238, 04:240-04:241, 04:244-04:247, 04:250-04:255N, 04:257-04:260, 04:262, 04:264-04:285, 04:287-04:293, 04:296-04:298, 04:300N-04:301, 04:303-04:312, 04:314-04:334, 04:336, 04:338-04:350N, 04:352-04:353, 04:355-04:356, 04:358-04:359, 04:361-04:362N, 04:364N-04:366, 04:368-04:369N, 04:371N-04:380, 04:382, 04:384-04:385N, 04:388-04:392, 04:394-04:399, 04:401, 04:403-04:404	2	3								16									31
*04:01:03, 04:133	2	3								16									

Negative Control

HLA-C14 2020 7K9

*05:30, 16:144													26			31
*06:02:01-01-06:02:01:43, 06:02:03-06:02:07, 06:02:09-06:02:71, 06:02:73-06:04:02, 06:06-06:10, 06:12-06:27, 06:29-06:34:01, 06:35-06:43:02, 06:45-06:60, 06:62-06:75, 06:77-06:81, 06:83-06:100, 06:102:01-06:123, 06:126-06:137, 06:139-06:141, 06:143-06:146, 06:148-06:182, 06:185-06:197, 06:199-06:209, 06:211:01:01N-06:216, 06:218-06:234, 06:236-06:246, 06:249-06:251, 06:253-06:292, 07:07, 07:09, 07:76:01-07:76:02, 07:315, 07:406, 07:559, 07:598, 07:656, 17:01:08, 17:03:03, 17:41																
*06:02:08, 06:34:02																
*06:28, 06:76:01, 12:04:01, 17:01:01:02-17:01:03, 17:01:05-17:01:07, 17:01:09-17:03:02, 17:04-17:21, 17:23-17:26, 17:28-17:39, 17:42-17:51:01:02																
*06:101																
*06:125																
*06:142																
*07:01:41, 07:02:41, 07:223, 07:357, 07:361, 07:415-07:416																
*07:02:32, 07:17:03, 07:242																
*07:08, 07:108:01-07:108:02																
*07:64	1															
*07:73:01-07:73:02, 07:92, 07:172:01-07:172:02, 12:02:11, 12:02:29, 12:03:04, 12:03:09, 12:03:34:01-12:03:34:02, 12:195:01, 12:195:03, 17:22																
*07:267, 07:322																
*07:402	1															
*08:01:07, 08:02:07, 08:33:02, 12:02:19																
*08:95, 12:02:23, 12:02:28, 12:03:24, 12:72, 12:135, 12:154, 16:37, 16:71																
*08:119, 12:115																
*12:03:20, 16:01:06																
*12:10:01-12:10:02, 16:117																
*12:70, 12:104N																
*12:156																
*12:209																
*15:36																
*16:85																
*16:86, 16:147																
*17:27N																

Negative Control

