


Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27
HLA-C allele ^{1,2,3}																											
*15:02:01-15:02:17, 15:02:19, 15:49-15:51, 15:53, 15:57, 15:60, 15:62, 15:67-15:68, 15:71, 15:73, 15:75, 15:78-15:80	1	2									11				15												
*15:02:18		2									11				15												
*15:03	1	2	3								11				15												
*15:04		2		4											15												
*15:05:01-15:05:09, 15:54, 15:59, 15:70	1	2			5										15												
*15:06:01-15:06:03, 15:55	1	2				6									15												
*15:07	1	2					7				11	12															
*15:08, 15:74	1	2						8			11				15												
*15:09, 15:61	1	2		4											15												
*15:10:01-15:10:03	1									10	11				15												
*15:11	1								9		11				14												
*15:12	1	2									11		13		15												
*15:13	1	2									11			14	15												
*15:15	1	2									11				15	16											
*15:16	1		3								11			14													
*15:17	1										11			14													
*15:18	1	2							9		11				15												
*15:19	1	2		4				8							15												
*15:21	1	2					7				11	12			15												
*15:22	1	2			5	6									15												
*15:23	1	2			5				9						15												
*15:24	1	2											13		15												
*15:25			w				7					12															
*15:26	1	2				6					11				15	16											
*15:27	1			4												16											
*15:28	1	2	3								11	12			15												
*15:29	1	2			5										15							23					
*15:30		2		4											15											27	
*15:31	1	2									11				15		18										
*15:32Q, 15:41 ⁴	1	2									11				15							24					
*15:33	1	2									11				15												
*15:34, 15:39 ⁴	1	2									11				15	17											
*15:35, 15:47 ⁴	1	2									11				15					21							
*15:36	1	2			5										15	17											
*15:37, 15:58, 15:72	1	2				6					11				15												
*15:38	1	2									11				15						22						
*15:40		2			6										15												
*15:42	1	2									11				15			19									
*15:43	1										11	12		14													
*15:44-15:45 ⁴	1	2									11				15				20								
*15:46	1	2			5										15			19									
*15:48	1	2									11											23					
*15:52	1	2									11				15									25			
*15:56	1	2									11				15										26		
*15:63	1								9		11				15												
*15:64, 15:66, 15:76		2													15												
*15:65		2		4		6									15												
*15:69	1	2			5	6									15	16											
*15:77		2		4											15	16											
*15:81	1	2									11				15												
*15:82	1	2									11				15												
*01:04, 01:54, 02:02:13, 02:02:26, 02:21, 02:26:01-02:26:03, 02:39, 02:45, 03:02:10, 06:02:01:01-06:02:01:03, 06:02:03-06:02:34, 06:03:02-06:04, 06:06-06:08, 06:10-06:27, 06:29-06:33, 06:35-06:37, 06:39-06:43:02, 06:45-06:73, 06:75, 06:77-06:88, 06:90-06:102, 06:104-06:106:02, 06:108-06:117, 07:01:01:01-07:02:08, 07:02:11-07:03, 07:05-07:06, 07:08, 07:10, 07:14-07:19, 07:21-07:27:02, 07:30-07:33N, 07:35-07:40, 07:42-07:44, 07:46-07:62, 07:65-07:72, 07:74-07:91, 07:93-07:95, 07:97-07:100, 07:102-07:122, 07:124-07:126, 07:128-07:138, 07:140-07:141:02, 07:143-07:171, 07:175-07:176, 07:178-07:180, 07:182-07:183, 07:185-07:198N, 07:200-07:218, 07:220-07:230, 07:232-07:245, 07:247, 07:249-07:262, 07:264N-07:271, 07:273-07:288, 07:291-07:293, 07:296-07:297, 07:299-07:322, 07:325-07:327, 07:330-07:335, 07:337, 14:02:10, 14:18				4																							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27

[illegible]


Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27
*04:55							7			10			13														
*04:108	1				5					10																	
*04:112										10					15		17										
*05:21										10											21						
*05:36										10					15												
*06:44, 07:07, 07:09				4											15												
*06:74Q				4																				24			
*06:89	1			4																							
*07:02:09, 12:16				4				7		9																	
*07:02:10, 07:127, 07:246			w	4																							
*07:20		2		4																							
*07:96:01-07:96:02		2	w	4								12															
*07:123, 07:173, 07:294	1			4														18									
*07:174, 07:298				4																							27
*07:263		2	w	4																							
*07:289												11		14													
*08:09, 08:11, 08:59, 12:02:01-12:03:03, 12:03:05-12:03:08, 12:03:10-12:03:22, 12:03:25-12:03:28, 12:06-12:08, 12:10:01-12:14:02, 12:17-12:20, 12:22-12:26, 12:28-12:30, 12:32, 12:34-12:40, 12:42Q-12:53, 12:55-12:57, 12:59, 12:61-12:70, 12:72-12:86, 12:88-12:111, 14:02:03, 14:08, 16:01:01, 16:01:03-16:01:15, 16:04:01, 16:06-16:08, 16:11, 16:13-16:18, 16:22-16:24, 16:26-16:33, 16:35-16:44, 16:49-16:56, 16:58-16:59				4				7		10																	
*08:14, 08:58, 08:80, 14:54, B*35:08:02, B*35:08:05, B*67:02								7																			
*08:22, 08:56								7		10													23				
*08:37								7		9	10																
*08:51								7		10								17									
*12:03:23			w	4				7		10		12															
*12:03:24				4				w		10																	
*12:15	1							7	8	10	11				14												
*12:58				4				7		10					13												
*12:87				4				7		10								17									
*14:10				4				7							13												
*16:20	1			4				7		10																	
*16:21				4				7		9	10																
*18:05															15												
B*07:78, B*13:18, B*13:31, B*13:41, B*15:73, B*54:10, B*54:20, B*55:09, B*55:21, B*55:37, B*55:52, B*56:43									w																		
B*15:200, B*39:82, B*51:115, B*58:05																			18								
B*18:91																										25	
B*35:03:11, B*35:205					5																						
B*35:132																											26
B*35:222																	16										
B*46:11, B*46:18, B*56:08, B*56:14														14													
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27
																</											

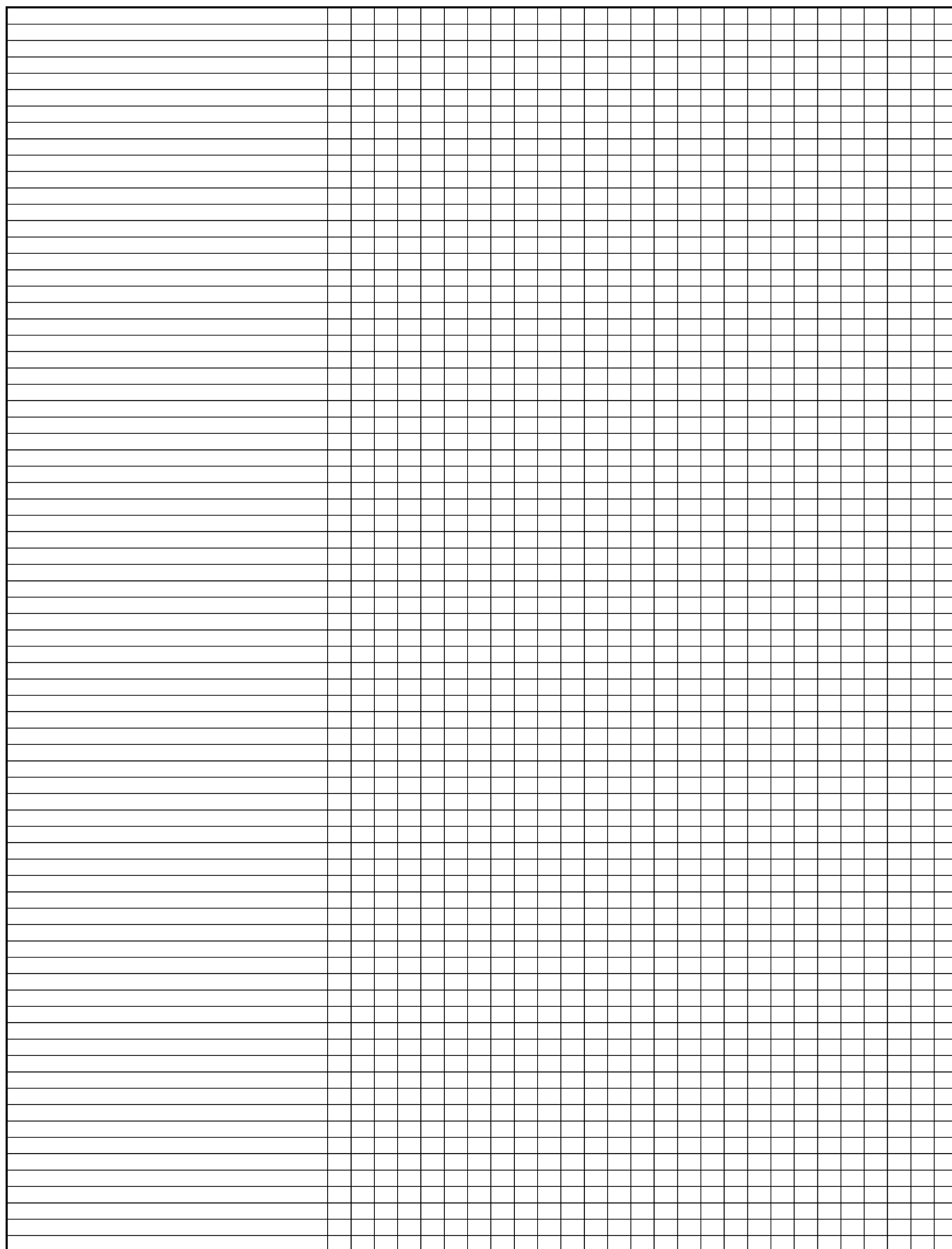

[illegible]

[illegible]

[illegible]

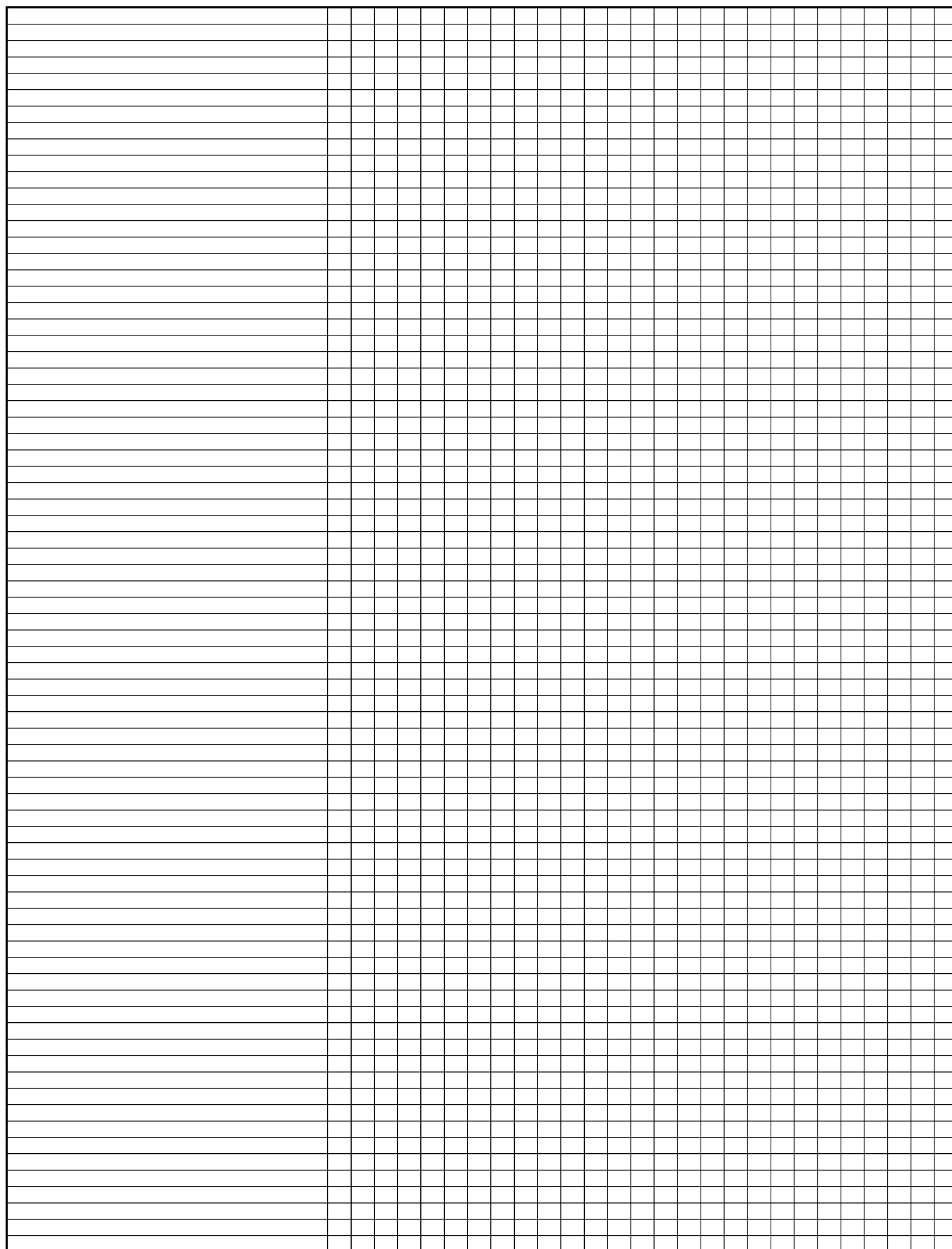

[illegible]

[illegible]

[illegible]

[illegible]

