

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44								
HLA-C allele ^{1,2,3}																																																				
*15:02:01:01-15:02:01:07, 15:02:01:09-15:02:02:57, 15:49-15:51, 15:53, 15:60, 15:62, 15:67-15:68, 15:73, 15:75, 15:78:01-15:80, 15:83, 15:86, 15:88-15:89, 15:91, 15:93-15:94, 15:98-15:100, 15:106, 15:109, 15:112, 15:119-15:121, 15:131, 15:133-15:137, 15:139, 15:141, 15:146, 15:149-15:151, 15:154-15:155, 15:157, 15:159-15:163, 15:166, 15:168, 15:170-15:171, 15:173-15:176, 15:178-15:179, 15:182-15:184, 15:186-15:189N, 15:191, 15:193, 15:196-15:200, 15:202-15:203, 15:205-15:207, 15:209-15:210, 15:212-15:213N, 15:215-15:217, 15:219-15:221, 15:223, 15:225-15:228, 15:230-15:232, 15:237-15:243, 15:245, 15:249-15:250	1	2									11				15																																					
*15:02:01:08N	1	2									11				15																																					
*15:03	1	2	3								11				15																																					
*15:04:01:01-15:04:06, 15:211, 15:236, 16:70 ⁴		2		4											15																																					
*15:05:01:01-15:05:07, 15:05:09-15:05:10, 15:05:13-15:05:18, 15:54, 15:59, 15:70, 15:90, 15:104, 15:140, 15:147-15:148, 15:152:01-15:152:03, 15:164N-15:165, 15:167, 15:169, 15:185N, 15:204, 15:218, 15:224, 15:229, 15:233-15:235Q	1	2			5										15																																					
*15:05:11-15:05:12, 15:244	1	2			5										15																																					
*15:06:01-15:06:03, 15:55, 15:177N, 15:180, 15:214	1	2				6									15																																					
*15:07:01:01-15:07:01:02	1	2					7				11	12																																								
*15:08:01-15:08:02	1	2						8			11				15																																					
*15:09:01:01-15:09:02, 15:61, 15:126, 15:143	1	2		4											15																																					
*15:10:01-15:10:04	1										10	11			15																																					
*15:11	1								9		11			14																																						
*15:12	1	2									11		13		15																																					
*15:13:01:01-15:13:02	1	2									11			14	15																																					
*15:15	1	2									11				15	16																																				
*15:16:01-15:16:02	1		3								11			14																																						
*15:17	1										11			14																																						
*15:18, 15:118, 15:138 ⁵	1	2							9		11				15																																					
*15:19	1	2		4				8							15																																					
*15:21	1	2					7				11	12			15																																					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44								

Negative Control

HLA-C15 2022 0R9.xlsx

*15:97	1	2									11			15			20	22											33															
*15:101	1	2									11			15															33			37												
*15:102		2												15														33		36														
*15:103	1	2									11		14	15											26			33																
*15:105Q		2												15						24						27		33																
*15:107	1		4								10			15													33																	
*15:108	1	2		5										15													33	34				39												
*15:110		2		5										15												33																		
*15:113, 15:192	1										11			15												33																		
*15:114	1	2									11			15												33	34																	
*15:115N	1	2		5										15												33		35																
*15:116	1	2		5		7						12		15												33																		
*15:117	1	2		5										15												33										42								
*15:122N	1	2									11			15											25		33				37													
*15:123	1	2									11			15												27		33																
*15:124	1										11			15												33						38												
*15:125	1	2		5	6									15												33																		
*15:128, 15:142, 15:247N	1	2												15												33																		
*15:129		2									11			15												33																		
*15:132	1	2									11			15												33						39												
*15:144	1	2				7					11	12	13													31																		
*15:145N	1	2									11			15												33										41								
*15:153	1	2	3		5							12		15												33																		
*15:156N	1	2									11			15																														
*15:158	1	2			6		9							15												33																		
*15:172	1	2									11		14	15			19									33																		
*15:181	1	2		5										15											31	33																		
*15:194	1	2	4		6									15	16											33																		
*15:195	1	2									11			15	16	17										33																		
*15:208	1			5							10			15												33																		
*15:246		2			6									15												33											43							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44

Negative Control

<p>*01:04, 01:97, 02:02:13, 02:02:26, 02:02:29, 02:21, 02:26:01-02:26:03, 02:39, 02:45, 02:152, 02:167, 02:191, 03:02:10, 03:264, 06:02:01-01-06:02:01:92, 06:02:03-06:02:47, 06:02:49-06:02:71, 06:02:73-06:02:100, 06:03:02-06:04:02:02, 06:06:01:01-06:08, 06:10-06:27, 06:29-06:33, 06:35-06:37, 06:39-06:43:02, 06:45-06:70:01, 06:71-06:73, 06:75, 06:77-06:88, 06:90-06:102:02, 06:104-06:106:02, 06:108-06:127:02, 06:129-06:131, 06:133-06:143, 06:145-06:167, 06:169-06:178, 06:180-06:203, 06:205-06:206, 06:208N-06:222, 06:225-06:265, 06:267N-06:289, 06:291-06:305, 06:307-06:319, 06:321-06:348, 06:351, 07:01:01-01-07:01:02:14, 07:01:04-07:01:67, 07:01:69-07:01:73, 07:01:75-07:02:08, 07:02:11-07:02:62, 07:02:64-07:02:136, 07:02:138-07:03, 07:05-07:06:06, 07:08, 07:14-07:19, 07:21-07:27:02, 07:30-07:33N, 07:35-07:40, 07:42, 07:44, 07:46, 07:47:02-07:62, 07:65-07:72, 07:74-07:91, 07:93:01-07:95, 07:97-07:100, 07:102-07:122, 07:124-07:125:01, 07:126, 07:127:02-07:138, 07:140:01-07:141:03, 07:143-07:171, 07:175-07:176, 07:178-07:180, 07:182-07:183, 07:185-07:195, 07:197-07:198N, 07:200-07:209, 07:211-07:218, 07:220-07:230, 07:232-07:245, 07:246:02, 07:249-07:262, 07:264N-07:271, 07:273-07:288, 07:291-07:293, 07:296-07:297, 07:299-07:314:01, 07:314:03-07:322, 07:325-07:327, 07:330:01-07:335, 07:337, 07:339-07:353, 07:356, 07:359-07:360, 07:362-07:363, 07:366, 07:368:01-07:377, 07:379-07:393N, 07:396-07:402, 07:404-07:405:02, 07:407-07:419, 07:421-07:425, 07:427, 07:429-07:446, 07:448-07:458, 07:460-07:465, 07:468-07:479, 07:481-07:483N, 07:485-07:486, 07:488-07:500, 07:502-07:521:02, 07:524-07:533, 07:537-07:551N, 07:553-07:558:01:02, 07:560-07:561, 07:564-07:567, 07:570-07:577, 07:579-07:582Q, 07:584, 07:587-07:599, 07:601-07:621, 07:623-07:625, 07:627-07:650:02, 07:652-07:654, 07:657-07:663Q, 07:665-07:671, 07:673, 07:675N-07:692, 07:694-07:697Q, 07:699-07:707, 07:709-07:722, 07:724, 07:726N-07:741, 07:743N-07:750N, 07:752N-07:755, 07:757-07:774, 07:776N-07:779:01:02, 07:781-07:796N, 07:798-07:830, 07:832-07:833N, 07:835-07:837, 07:840N-07:849N, 07:851, 07:853, 07:855-07:856N, 07:859-07:867, 07:869-07:875, 07:877-07:880, 07:882-07:888, 07:895-07:898, 07:899-07:901, 07:902-07:908</p>	<p>4</p>										<p>Negative Control</p>
--	----------	--	--	--	--	--	--	--	--	--	-------------------------

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44
----------	---	---	---	---	---	---	---	---	---	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----

*03:02:01-03:02:09, 03:02:11-03:02:14, 03:02:16-03:02:27, 03:16, 03:33, 03:36, 03:40:01-03:40:04, 03:42:01-03:43:02, 03:60, 03:71, 03:84, 03:89, 03:95, 03:108, 03:110, 03:119:01-03:119:02, 03:121N, 03:132, 03:139, 03:146, 03:169Q, 03:190, 03:194, 03:197-03:198, 03:200-03:201N, 03:216, 03:221-03:222, 03:224N-03:226, 03:240, 03:248, 03:258, 03:271, 03:279, 03:298-03:300, 03:315, 03:329-03:330, 03:332, 03:338, 03:349-03:350, 03:365, 03:371, 03:373, 03:391N, 03:402-03:403, 03:406, 03:411, 03:425, 03:429, 03:449N, 03:468, 03:473, 03:475, 03:489, 03:536, 03:544, 03:547, 03:549, 03:561, 03:575, 03:584, 03:589, 03:596, 07:756	4				12																Negative Control																							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44
*03:02:15, 03:03:01:01-03:03:14, 03:03:16-03:03:20, 03:03:22-03:03:50, 03:03:52-03:03:54, 03:03:56-03:03:63, 03:03:65-03:04:16, 03:04:18-03:04:24, 03:04:27-03:04:32, 03:04:34-03:04:38, 03:04:40-03:04:68, 03:04:70-03:04:76, 03:04:78-03:04:84, 03:04:86-03:06:02, 03:09:03:11:02, 03:13:01:01-03:13:02, 03:17:01-03:17:02, 03:20N-03:26:02, 03:28, 03:30, 03:32:01:01-03:32:01:02, 03:34-03:35:02:02, 03:37:01-03:37:02, 03:41:01-03:41:02, 03:44, 03:46-03:57:02, 03:59, 03:61-03:63, 03:65-03:66, 03:68, 03:70, 03:72-03:80:02, 03:82-03:83, 03:85, 03:87:01-03:88, 03:90-03:93, 03:96-03:98, 03:100-03:101, 03:103-03:107, 03:109, 03:111-03:114, 03:116:01-03:118, 03:120, 03:122-03:129, 03:131, 03:133, 03:135, 03:138, 03:141-03:143, 03:145, 03:147, 03:149-																																												

*05:36									10					15																			33															
*05:132	1							8	10	11				14																																		
*05:215	1		4						10																																							
*05:216:01:01-05:216:01:02									10								17																															
*06:44, 07:07, 07:09			4													15																33																
*06:74Q			4																				24																									
*06:128N			4																													35																
*06:168								8		11				14																	32																	
*06:207, 06:320			4														17																															
*06:223			4																			21																										
*06:224	1							8		11				14																	32																	
*06:266			4																																													
*06:349, 07:708			4																			20																										
*07:01:74, 07:02:09, 07:125:02, 12:16:01, 12:147, 16:80			4					7	9																						31																	
*07:02:10, 07:127:01, 07:246:01, 07:989			w	4																																												
*07:10, 07:43:01-07:43:02, 07:196, 07:367, 07:568			4															18																														
*07:20:01-07:20:02		2	4																																													
*07:96:01-07:96:02		2	w	4								12																																				
*07:174, 07:298			4																																													
*07:184, B*15:88, B*44:247, B*44:295:01-44:295:02, B*58:05																		18																														
*07:210, 07:247			4																												33																	
*07:263		2	w	4																																												
*07:289											11			14																																		
*07:559, 07:914				4										15																																		
*07:626	1				5																																											
*07:725	1										11																																					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44				
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44				
*08:14, 08:58, 08:80, 08:103:01:01-08:103:01:02, 08:105, 08:134, 08:155, 08:239, 14:54, 14:79, B*07:02:77, B*08:01:36, B*15:01:50, B*15:436, B*18:03:02, B*35:08:02, B*35:08:05, B*55:01:25, B*67:02:01:01-67:02:01:02								7																								31																
*08:22:01:01-08:22:01:02, 08:56, 08:154, 08:178, 08:202								7		10														23								31																
*08:37								7		9	10																						31															
*08:44, 08:61, 08:82, 08:209								7		10									18														31															
*08:51, 08:114, 08:243								7		10								17															31															
*08:95							w			10																							w															
*08:112							7			10																							31															
*08:113, 12:03:57, 12:113, 16:20, 16:109	1			4				7		10																							31															

Negative Control

