

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-C allele^{1,2,3}																								
*16:01:01, 16:01:03-16:01:15, 16:01:17-16:01:18, 16:34, 16:36, 16:41, 16:44, 16:51, 16:54, 16:59, 16:62, 16:65	1			4	5																			
*16:01:02, 16:01:16, 16:70	1				5																			
*16:02:01-16:02:12, 16:47, 16:57, 16:60, 16:69	1	2			5																			
*16:04:01, 16:33, 16:66	1		3	4																				
*16:06				4	5	6										16								
*16:07:01				4	5		7									16								
*16:07:02				4	5											16								
*16:08	1			4	5			8																
*16:09	1	2			5		7																	
*16:10	1			4		6																		
*16:11	1			4	5	6																		
*16:12	1	2			5				9															
*16:13	1			4	5					10														
*16:14	1			4	5								13											
*16:15:01-16:15:02, 16:20 ⁴	1			4	5						11													
*16:16Q-16:17 ⁴	1			4	5							12												
*16:18	1			4	5									14										
*16:19	1	2			5					10														
*16:21	1			4	5										15									
*16:22	1			4	5							12												
*16:23	1			4	5									14						20				
*16:24, 16:58 ⁴	1			4	5																	22		
*16:25	1	2			5						11													
*16:26	1			4	5												17	18						
*16:27, 16:32 ⁴	1			4	5															20				
*16:28, 16:31, 16:50 ⁴	1			4	5																19			
*16:29	1		3	4																	19			
*16:30N, 16:56 ⁴	1			4	5																		23	
*16:35				4					9															
*16:37	1			w	5																			
*16:38	1			4	5			8											18					
*16:39:01-16:39:02	1			4	5	6									15									
*16:40				4																	21			
*16:42	1		3	4																		23		
*16:43				4	5																			
*16:45	1			4			7																	
*16:46	1	2			5												17	18						
*16:48		2							9															
*16:49	1			4	5																21			
*16:52	1			4	5				9															
*16:53				4				8														21		
*16:55	1		3	4													17	18						
*16:61	1		3	4						10														
*16:63	1	2			5																21			
*16:64	1			4	5						11						17	18						
*16:67	1			4								12							19					
*16:68	1			4				8																
*01:04, 01:09, 12:03:04, 12:03:09, 14:16			3																					
*01:05, 01:36, 01:55, 01:79:01-01:79:02, 02:02:12, 02:66, 03:05, 03:13:01-03:13:02, 03:25, 03:27, 03:35, 03:135, 03:167, 03:178, 03:198, 04:169, 05:36, 07:01:23, 07:02:11, 07:03, 07:07, 07:16, 07:20, 07:36, 07:51, 07:56:01-07:56:02, 07:59-07:60, 07:64, 07:79, 07:88, 07:96:01, 07:109, 07:127, 07:172:02, 07:227N, 07:263, 07:327, 08:02:11, 12:02:11, 12:27, 12:123, 14:09, 15:22, 15:65, 15:72																16								
*01:10, 14:25, B*40:243																				19				
*01:14, 01:59, 02:02:09, 02:37, 02:57, 02:62, 02:74, 02:81, 03:07, 03:15, 03:45, 03:130, 03:140, 03:163, 03:243, 05:01:21, 05:07N, 05:10, 05:16, 05:31, 05:85, 05:96, 07:328, 12:09, 14:04, 14:12, 14:49, 14:64, 15:11, 15:16-15:17, 17:01:01-17:10, 17:12-17:15, 17:17-17:20, 17:23-17:24		2																						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control

*06:08, 06:22, 06:90		2	3													18	19							
*06:31	1	2	3													18								
*06:32		2	3									13				18								
*06:76:02		2	3													17	18							
*06:87		2	3						10							18								
*06:101, 06:127, 07:49, 07:76:01-07:76:02, 07:238, 07:315, 18:01-18:04, 18:06-18:07N		2													16	18								
*06:118		2						9								18								
*07:01:01:01-07:01:10, 07:01:12-07:01:22, 07:01:24-07:01:27, 07:01:29-07:02:08, 07:02:10, 07:02:12-07:02:59, 07:05-07:06, 07:08-07:09, 07:13-07:15, 07:17:01-07:19, 07:21-07:23, 07:26-07:30, 07:32N-07:33N, 07:35, 07:37-07:42, 07:44, 07:46-07:48, 07:50, 07:52, 07:54-07:55N, 07:57-07:58, 07:61N-07:62, 07:65-07:75, 07:77-07:78, 07:80-07:87, 07:89-07:95, 07:96:02-07:100, 07:102-07:108:02, 07:110-07:113, 07:115-07:126, 07:128-07:138, 07:140-07:141:02, 07:143-07:172:01, 07:173-07:176, 07:178-07:180, 07:182-07:183, 07:185-07:194, 07:197-07:215, 07:217, 07:219-07:226, 07:228-07:237, 07:239-07:262, 07:264N-07:271, 07:273-07:294, 07:296-07:301, 07:303-07:314, 07:316-07:322, 07:325-07:326, 07:330-07:331, 07:333-07:335, 07:337, 07:339-07:345, 07:347N-07:353, 07:356, 07:359-07:360, 07:362-07:363, 07:368-07:374, 18:05															16	18								
*07:02:09				4											15	16	17	18						
*07:24, 07:114, 07:218									10						16	18								
*07:25															16	18					22			
*07:31:01-07:31:02									9							18								
*07:53				5					11						16	18								
*07:177, 14:15									9															
*07:216				5	6				11						16	18								
*08:28			4												16	17	18							
*08:37, 12:16			4												15	16								
*08:69			4												16							23		
*08:96			4				8								16	18								
*12:03:01:01-12:03:03, 12:03:05-12:03:08, 12:03:10-12:03:23, 12:03:25-12:03:30, 12:06-12:07, 12:11-12:13, 12:15, 12:23, 12:25-12:26, 12:29, 12:31-12:32, 12:34-12:35, 12:37-12:39N, 12:42Q-12:43, 12:46N-12:48, 12:50-12:53, 12:55, 12:57-12:59, 12:61-12:63, 12:65-12:66, 12:70-12:71, 12:75-12:79, 12:81-12:82, 12:87-12:95, 12:97-12:102, 12:107-12:111, 12:113, 12:115-12:116, 12:120-12:122, 12:125			3	4																				
*12:03:24			3	w																				
*12:28			3	4												17	18							
*12:40			4										13		16									
*12:45			3	4					10															
*12:119			3	4														19						
*14:65									10															
*15:21				w																				
*15:23, 15:63														15										
*15:25			4													17	18							
*15:62																17	18							
*15:75, A*24:73, A*24:157, B*07:66, B*51:55										11														
*17:21		2																19						
A*24:96, A*24:146							7																	
A*24:106							6																	
A*24:174						5																		
B*46:25						5	7			11														
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control