

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-C allele^{1,2,3}																								
*16:01:01, 16:01:03-16:01:15, 16:01:17-16:01:19, 16:34, 16:36, 16:41, 16:44, 16:51, 16:54, 16:59, 16:62, 16:65, 16:72-16:73, 16:75-16:76, 16:79, 16:81, 16:83	1			4	5																			
*16:01:02, 16:01:16, 16:70	1				5																			
*16:02:01-16:02:13, 16:47, 16:57, 16:60, 16:69, 16:74	1	2			5																			
*16:04:01, 16:04:03, 16:33, 16:66, 16:78, 16:82	1		3	4																				
*16:06				4	5	6										16								
*16:07:01				4	5		7									16								
*16:07:02				4	5											16								
*16:08	1			4	5			8																
*16:09	1	2			5		7																	
*16:10	1			4		6																		
*16:11	1			4	5	6																		
*16:12	1	2			5				9															
*16:13	1			4	5					10														
*16:14	1			4	5								13											
*16:15:01-16:15:02, 16:20 ⁴	1			4	5						11													
*16:16Q-16:17 ⁴	1			4	5							12												
*16:18	1			4	5									14										
*16:19	1	2			5					10														
*16:21, 16:80	1			4	5										15									
*16:22				4	5						12													
*16:23	1			4	5									14						20				
*16:24, 16:58 ⁴	1			4	5																	22		
*16:25	1	2			5						11													
*16:26	1			4	5												17	18						
*16:27, 16:32 ⁴	1			4	5																20			
*16:28, 16:31, 16:50 ⁴	1			4	5																19			
*16:29	1		3	4																	19			
*16:30N, 16:56 ⁴	1			4	5																		23	
*16:35				4					9															
*16:37, 16:71	1			w	5																			
*16:38	1			4	5			8										18						
*16:39:01-16:39:02	1			4	5	6								15										
*16:40				4																	21			
*16:42	1		3	4																			23	
*16:43				4	5																			
*16:45	1			4			7																	
*16:46	1	2			5												17	18						
*16:48		2							9															
*16:49	1			4	5																21			
*16:52	1			4	5				9															
*16:53				4				8														21		
*16:55	1		3	4													17	18						
*16:61	1		3	4						10														
*16:63	1	2			5																	21		
*16:64	1			4	5					11							17	18						
*16:67	1			4							12								19					
*16:68	1			4				8																
*16:77N	1	2			5								13											
*01:02:34, 03:04:25, 08:01:11, 08:12, 08:34, 08:44, 08:61, 08:82, 12:02:12, 12:19-12:20, 12:24, 12:44, 12:134, 14:02:03, 14:03, 14:08, 14:10, 14:22, 14:35N, 14:38, 14:41, 14:53-14:54, 14:61, 15:07, B*35:08:02, B*35:08:05, B*67:02				4																				
*01:04, 01:09, 12:03:04, 12:03:09, 12:03:34, 14:16			3																					
*01:05, 01:36, 01:55, 01:79:01-01:79:02, 02:02:12, 02:66, 03:05, 03:13:01-03:13:02, 03:25, 03:27, 03:35, 03:135, 03:167, 03:178, 03:198, 03:267, 04:169, 05:36, 07:01:23, 07:02:11, 07:03, 07:07, 07:16, 07:20, 07:36, 07:51, 07:56:01-07:56:02, 07:59-07:60, 07:64, 07:79, 07:88, 07:96:01, 07:109, 07:127, 07:172:02, 07:227N, 07:263, 07:327, 07:390, 07:402, 08:02:11, 12:02:11, 12:27, 12:123, 14:09, 14:28:02, 15:22, 15:65, 15:72																16								
*01:10, 14:25, B*07:239, B*14:46, B*40:243																							19	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control

