

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
<b>HLA-C allele<sup>1,2,3</sup></b>																								
*16:01:01:01-16:01:01:02, 16:01:03-16:01:15, 16:01:17-16:01:22, 16:34, 16:36, 16:41, 16:44, 16:51, 16:54, 16:59, 16:62, 16:65, 16:72-16:73, 16:75-16:76, 16:79, 16:81, 16:83, 16:86-16:87, 16:92-16:93, 16:95-16:96, 16:98	1			4	5																			
*16:01:02, 16:01:16, 16:70, 16:85, 16:94	1				5																			
*16:02:01-16:02:13, 16:47, 16:57, 16:60, 16:63, 16:69, 16:74, 16:84 <sup>6</sup>	1	2			5																			
*16:04:01, 16:33, 16:66, 16:78, 16:82, 12:176 <sup>4</sup>	1		3	4																				
*16:04:03	1		3	4											15									
*16:06				4	5	6										16								
*16:07:01				4	5		7									16								
*16:07:02				4	5											16								
*16:08	1			4	5			8																
*16:09	1	2			5		7																	
*16:10	1			4		6																		
*16:11	1			4	5	6																		
*16:12	1	2			5				9															
*16:13	1			4	5					10														
*16:14	1			4	5								13											
*16:15:01-16:15:02, 16:20 <sup>5</sup>	1			4	5						11													
*16:16Q-16:17 <sup>5</sup>	1			4	5							12												
*16:18	1			4	5									14										
*16:19	1	2			5				10															
*16:21, 16:80	1			4	5										15									
*16:22				4	5							12												
*16:23	1			4	5								14							20				
*16:24, 16:58 <sup>5</sup>	1			4	5																	22		
*16:25	1	2			5					11														
*16:26	1			4	5												17	18						
*16:27, 16:32 <sup>5</sup>	1			4	5															20				
*16:28, 16:31, 16:50 <sup>5</sup>	1			4	5														19					
*16:29	1		3	4															19					
*16:30N, 16:56 <sup>5</sup>	1			4	5																		23	
*16:35				4					9															
*16:37, 16:71	1			w	5																			
*16:38	1			4	5			8										18						
*16:39:01-16:39:02	1			4	5	6								15										
*16:40				4																	21			
*16:42	1		3	4																			23	
*16:43				4	5																			
*16:45	1			4			7																	
*16:46	1	2			5												17	18						
*16:48		2							9															
*16:49	1			4	5																	21		
*16:52	1			4	5				9															
*16:53				4				8														21		
*16:55	1		3	4													17	18						
*16:61	1		3	4						10														
*16:64	1			4	5						11						17	18						
*16:67	1			4								12							19					
*16:68	1			4				8																
*16:77N	1	2			5								13											
*16:88	1	2						8																
*16:89N	1	2			5			8																
*16:90	1	2			5										16									
*16:91	1	2	3																					
*16:97	1			4	5												17							
*01:02:34, 03:04:25, 08:01:11, 08:12, 08:34, 08:44, 08:61, 08:82, 08:126, 12:02:12, 12:19-12:20, 12:24, 12:44, 12:134, 12:153, 12:175, 12:183, 14:02:03, 14:03, 14:08, 14:10, 14:22, 14:35N, 14:38, 14:41, 14:53-14:54, 14:61, 14:70, 15:07, B*35:08:02, B*35:08:05, B*67:02				4																				
<b>Well No.</b>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
<b>Well No.</b>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*01:04, 01:09, 12:03:04, 12:03:09, 12:03:34:01-12:03:34:02, 14:16			3																					

Negative Control



*04:01:24, 04:37, 04:40, 04:100, 04:110, 04:117, 04:178, 04:224, 04:230, 04:232, 04:242	2																17									
*04:01:29, 04:92, 04:112, 04:173N, 04:225N																	16	17								
*04:01:74	2																16	w								
*04:14, 04:68	2					7				11							16	17								
*04:29, 04:36			4														16	17								
*04:42:01	2																15	16	17							
*04:42:02, 04:220	2																15	16	17	18						
*04:69	2							9									16	17								
*04:80	2																17	18								
*04:101	2									10							16	17								
*04:214			w														16	17								
*05:56	2																16									23
*05:81, 05:117	2									10							16									
*05:112	2					7											16							20		
*06:02:01:01-06:02:01:03, 06:02:03-06:02:09, 06:02:11, 06:02:14-06:02:15, 06:02:17-06:02:42, 06:02:44-06:03:02, 06:07, 06:09-06:10, 06:12-06:13, 06:15-06:21, 06:23-06:25, 06:27-06:29, 06:33-06:34:02, 06:36-06:39, 06:41-06:43:02, 06:45-06:51, 06:54-06:71, 06:73-06:76:01, 06:77-06:78, 06:80, 06:82-06:86, 06:88-06:89, 06:91-06:100, 06:102-06:117, 06:119-06:121, 06:126, 06:128N-06:131, 06:135, 06:137-06:142, 06:145-06:146, 06:148-06:152N, 06:154-06:163, 06:165-06:168, 06:170, 06:172-06:187	2	3																		18						
*06:02:10, 06:26, 06:30, 06:53:01-06:53:02, 06:125, 06:132:01-06:132:02, 06:169, 12:04:01-12:05, 12:33, 12:54, 12:60	2	3																								
*06:02:12-06:02:13, 06:11, 06:44, 06:52, 06:122, 06:124, 06:134N, 06:147, 06:164			3																		18					
*06:02:16, 06:02:43, 06:04:01-06:04:02, 06:06, 06:14, 06:35, 06:40, 06:72, 06:79N, 06:81, 06:123, 06:143, 06:153, 07:403, 07:406, 17:11	2																				18					
*06:05	2																15	17	18							
*06:08, 06:22, 06:90	2	3																			18	19				
*06:31	1	2	3																		18					
*06:32, 06:171N	2	3												13							18					
*06:76:02	2	3																			17	18				
*06:87	2	3								10											18					
*06:101, 06:127:01:01-06:127:02, 06:136, 06:144, 07:49, 07:76:01-07:76:02, 07:238, 07:315, 18:01-18:04, 18:06-18:10	2																				16	18				
*06:118	2									9											18					
*06:133	2	3								9											18					
<b>Well No.</b>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24		
<b>Well No.</b>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24		
*07:01:01:01-07:01:10, 07:01:12-07:01:22, 07:01:24-07:01:27, 07:01:29-07:02:08, 07:02:10, 07:02:12-07:02:76, 07:05-07:06, 07:08-07:09, 07:13-07:15, 07:17:01-07:19, 07:21-07:23, 07:26:01-07:30, 07:32N-07:33N, 07:35, 07:37-07:42, 07:44, 07:46-07:48, 07:50, 07:52, 07:54-07:55N, 07:57-07:58, 07:61N-07:62, 07:65-07:75, 07:77-07:78:02, 07:80-07:87, 07:89-07:95, 07:96:02-07:100, 07:102-07:108:02, 07:110-07:113, 07:115-07:126, 07:128-07:138, 07:140-07:141:02, 07:143-07:163, 07:165-07:172:01, 07:173-07:176, 07:178-07:180, 07:182-07:183, 07:185-07:194, 07:197-07:215, 07:217, 07:219-07:226, 07:228-07:237, 07:239-07:262, 07:264N-07:271, 07:273-07:294, 07:296-07:301, 07:303-07:314:02, 07:316-07:322, 07:325-07:326, 07:330:01-07:331, 07:333-07:335, 07:337, 07:339-07:345, 07:347N-07:353, 07:356, 07:359-07:360, 07:362-07:363, 07:368:01-07:377, 07:379-07:389, 07:391-07:393N, 07:396-07:401, 07:405, 07:407-07:419, 07:421-07:425, 07:427, 07:429-07:437N, 07:439-07:443, 07:445-07:446, 07:448-07:450, 07:452N-07:458, 07:460-07:462, 07:464-07:465, 07:468-07:479, 07:481-07:484N, 07:486, 07:488-07:500, 07:502-07:510, 07:512-07:513Q, 07:515-07:522, 07:524-07:530, 18:05																16					18					
*07:02:09				4													15	16	17	18						
*07:24, 07:114, 07:218										10							16	18								
*07:25, 07:404																	16	18					22			
*07:31:01-07:31:02, 07:514									9												18					
*07:53					5						11						16	18								
*07:164N, 07:451N													13			16	18									

Negative Control

