

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
<b>HLA-C allele<sup>1,2,3</sup></b>																
*17:01:01:01-17:01:11, 17:30	1			4												
*17:02		2														
*17:03	1		3													
*17:04	? ? ? ?				5											
*17:05	1			4		6										
*17:06	? ? ? ?						7									
*17:07	? ? ? ?							8								
*17:08	? ? ? ?								9							
*17:09	? ? ? ?									10						
*17:10	? ? ? ?										11					
*17:11	? ? ? ?											12				
*17:12	? ? ? ?												13			
*17:13	? ? ? ?													14		
*17:14, 17:16, 17:18-17:21, 17:23-17:26, 17:28-17:29, 17:31	? ? ? ?															
*17:15	? ? ? ?				5		7									
*17:17	1		3												14	
*17:22	? ? ? ?															15
*17:27N	? ? ? ?								9	10						
*01:17, 01:21, 01:23, 03:27, 03:38:01-03:38:02, 03:69, 03:136, 03:246, 03:274, 04:29, 04:172, 06:11, 07:01:01:01-07:01:40, 07:01:42-07:02:40, 07:02:42-07:06, 07:08, 07:10-07:33N, 07:35-07:38:02, 07:41-07:48, 07:50-07:60, 07:62-07:75, 07:77-07:78:01, 07:79-07:100, 07:102-07:114, 07:116-07:147, 07:149-07:160, 07:162, 07:164N-07:176, 07:178-07:209, 07:211-07:222, 07:224, 07:226-07:237, 07:239-07:246, 07:248-07:290, 07:292-07:294, 07:296-07:314:02, 07:316-07:327, 07:329N-07:334, 07:336-07:356, 07:358-07:360, 07:362-07:376, 07:378-07:402, 07:404-07:405, 07:407-07:414, 07:416-07:477, 07:479-07:499, 08:05, 08:21, 08:25, 12:02:01-12:03:37, 12:06-12:08, 12:10:01-12:20, 12:22-12:26, 12:28-12:32, 12:34-12:40, 12:42Q-12:48, 12:50-12:53, 12:55-12:59, 12:61-12:62, 12:64-12:71, 12:73-12:97, 12:99-12:107, 12:109-12:134, 12:136-12:145, 12:147-12:153, 12:155Q-12:174, 14:73, 15:25, 16:15:01-16:15:02, 16:64, B*07:13, B*67:02																15
*01:43, 03:280, 05:01:12, 06:67, 08:02:11						6										
*02:12, 06:122, 07:478, 12:72, 12:135, 12:154																w
*03:129, 03:234, B*15:116, B*15:352, B*40:63, B*40:92, B*40:324, B*44:169, B*44:182, B*46:43, B*57:59								8								
*07:101, 07:148, 07:161						6										15
*15:98, B*42:22													13			
B*40:260										10						
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>

Negative Control