

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-C allele^{1,2,3}																
*17:01:01:02-17:01:18, 17:21, 17:30, 17:37-17:38:01:02, 17:42-17:44, 17:46-17:48, 17:50-17:55, 17:57, 17:61, 17:64Q, 04:423 ⁴	1			4												
*17:02		2														
*17:03:01:01-17:03:03, 17:16:02, 17:35:02, 17:39-17:41, 17:49, 17:56, 17:58-17:60, 17:63	1		3													
*17:04	1			4	5											
*17:05	1			4		6										
*17:06	?	?	?	?			7									
*17:07	?	?	?	?				8								
*17:08	1			4					9							
*17:09	?	?	?	?						10						
*17:10	?	?	?	?							11					
*17:11	?	?	?	?								12				
*17:12	?	?	?	?									13			
*17:13	?	?	?	?										14		
*17:14, 17:16:01, 17:18-17:20, 17:23-17:26, 17:28-17:29, 17:31-17:35:01, 17:36, 17:45	?	?	?	?												
*17:15	?	?	?	?	5		7									
*17:17	1		3											14		
*17:22	?	?	?	?											15	
*17:27N	?	?	?	?					9	10						
*17:62	1			4												w
*01:17, 01:21, 01:23, 01:128, 01:152, 01:157, 02:115, 03:27, 03:38:01-03:38:02, 03:69, 03:136, 03:246, 03:274, 03:431, 04:29, 04:172, 04:473, 06:11, 06:252, 07:01:01:01-07:01:02:14, 07:01:04-07:01:40, 07:01:42-07:01:69, 07:01:71-07:02:40, 07:02:42-07:02:99, 07:02:101-07:02:130, 07:02:132-07:04:13, 07:04:15-07:06:06, 07:08, 07:10-07:33N, 07:35-07:38:02, 07:41-07:48, 07:50-07:60, 07:62-07:75, 07:77-07:78:01, 07:79-07:100, 07:102-07:114, 07:116-07:147, 07:149-07:160, 07:162, 07:164N-07:176, 07:178-07:209, 07:211-07:222, 07:224, 07:226Q-07:237, 07:239-07:246:02, 07:248-07:290, 07:292-07:294, 07:296-07:314:03, 07:316-07:327, 07:329N-07:334, 07:336-07:356, 07:358-07:360, 07:362-07:376, 07:378-07:402, 07:404-07:405:02, 07:407-07:414, 07:416-07:477, 07:479-07:517, 07:519-07:527, 07:529-07:554, 07:556-07:558:01:02, 07:560-07:562, 07:564-07:597, 07:599-07:653, 07:655, 07:657-07:664, 07:666-07:726N, 07:728-07:792, 07:794-07:913, 07:915-07:974Q, 07:976-07:992, 07:994, 07:996-07:1023, 08:05, 08:21, 08:25, 08:137, 12:02:01-12:02:22, 12:02:24-12:03:69, 12:03:71-12:03:81, 12:06-12:08, 12:10:01-12:20, 12:22-12:26, 12:28-12:32, 12:34-12:40, 12:42Q-12:48, 12:50-12:53, 12:55-12:59, 12:61-12:62, 12:64-12:71, 12:73-12:97, 12:99:01-12:107, 12:109-12:134, 12:136-12:145, 12:147-12:153, 12:155Q-12:187, 12:189-12:298, 12:300, 12:302-12:333, 12:335-12:352, 12:354-12:357, 12:359-12:361, 12:363-12:367, 14:73, 14:128, 15:25, 16:15:02, 16:64, 16:201, B*07:13, B*67:02:01:01-67:02:01:02																15
*01:43, 03:280, 03:530, 05:01:12, 06:67, 08:01:30, 08:02:11:01-08:02:11:02						6										
*02:12, 04:449, 06:122, 07:02:131, 07:478, 07:559, 07:656, 07:914, 12:02:23, 12:03:70, 12:72, 12:135, 12:154																w
*03:129, 03:234, B*15:116, B*15:352, B*15:412, B*15:507, B*35:331, B*40:63, B*40:92, B*40:324, B*44:169, B*44:182, B*46:43, B*51:324, B*51:338, B*57:59								8								
*03:319											11					
*04:366, 04:368												12				
*04:447, 04:480, 05:206, 08:185														14		
*06:239, 16:154	1															
*07:101, 07:148, 07:161						6										15
*07:665	1															15
*08:184, 15:135					5											
*15:98, A*31:01:38, B*42:22													13			
B*40:260										10						
B*51:250				4												
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16

Negative Control