

*0129, 0133⁵	1											
*0130	1											
*0131	1											
*0132	1											
*0134										11		
*0358, 0437	2											
*0359												
*0509, 0517, 0815, B*1533											12	
*0623							8					
*0737, B*0777				4								
A*0124												
B*5418												
HLA-Cw allele⁴												
Well No.	1	2	3	4	5	6	7	8	9	10	11	12

INTERPRETATION TABLE												
HLA-Cw*01 SSP subtyping												
Amplification patterns of the Cw*0101 to Cw*0134 alleles												
Well ⁶												
13	14	15	16	17	18	19	20	21	22	23	24	
155	240	230	255	75	195	100	110	125	255	235	250	Length of spec.
			295		260			245				PCR product(s)
800	1070	800	1070	800	800	1070	800	800	1070	1070	1070	Length of int.
												pos. control ¹
89	89	89	89	806	89	368	530	89	368	368	463	5'-primer(s) ²
5'-gAA ³	5'-gAA ³	5'-gAA ³	5'-gAA ³	5'-ggA ³	5'-gAA ³	5'-gTg ³	5'-ggT ³	5'-gAA ³	5'-gTg ³	5'-gTg ³	5'-TgA ³	
				818	453			368				
				5'-ggC ³	5'-AAT ³			5'-gTg ³				
201	289	278	301	846	244	427	601	172	583	560	3 rd I	3'-primer(s) ³
5'-CTC ³	5'-AgC ³	5'-ggT ³	5'-gCA ³	5'-CAC ³	5'-CTg ³	5'-gTA ³	5'-CTC ³	5'-CAT ³	5'-gTg ³	5'-ACA ³	5'-ATg ³	
			341		3 rd I			573				
			5'-CgT ³		5'-ATg ³			5'-AgA ³				
13	14	15	16	17	18	19	20	21	22	23	24	Well No.
												HLA-Cw allele ⁴
13												*010201-010209
13												*0103
13												*0104
13												*0105
13												*0106
13												*0107
13												*0108
13												*0109
13												*0110
13												*0111
13												*0112
13												*0113
13												*0114
13												*0115
13												*0116
13	14											*0117
13		15										*0118
13			16									*0119
13												*0120
	14											*0121
13												*0122
13	14		16									*0123
13				17								*0124
13	14	15	16	17	18	19	20	21	22	23	24	Well No.
155	240	230	255	75	195	100	110	125	255	235	250	Length of spec.
			295		260			245				PCR product(s)
13	14	15	16	17	18	19	20	21	22	23	24	Well No.
13				17								*0125
13					18							*0126
13						19						*0127
13							20					*0128

13								21				*0129, 0133 ⁵
13									22			*0130
13										23		*0131
13											24	*0132
13					18							*0134
												*0358, 0437
							20					*0359
												*0509, 0517, 0815, B*1533
												*0623
												*0737, B*0777
			16									A*0124
13												B*5418
												HLA-Cw allele ⁴
13	14	15	16	17	18	19	20	21	22	23	24	Well No.