

INTERPRETATION TABLE								
HLA-Cw*05 subtyping								
Amplification patterns of the Cw*0501 to 0517 alleles								
	Well							
	1	2	3	4	5	6	7	8
Length of spec.	155	165	155	120	225	270	265	85
PCR product								
Length of int.	800	1070	1070	800	1070	1070	1070	800
pos. control¹								
5'-primer(s)²	485	176	485	3rd I	419	379	379	176
	5'-CAA ^{3'}	5'-gCA ^{3'}	5'-CAA ^{3'}	5'-TgT ^{3'}	5'-gTC ^{3'}	5'-ACg ^{3'}	5'-ACC ^{3'}	5'-gCA ^{3'}
						361		
						5'-AgT ^{3'}		
3'-primer(s)³	601	302	601	668	601	601	601	221
	5'-CTT ^{3'}	5'-ggT ^{3'}	5'-CTC ^{3'}	5'-TgA ^{3'}	5'-CTT ^{3'}	5'-CTC ^{3'}	5'-CTC ^{3'}	5'-ACC ^{3'}
Well No.	1	2	3	4	5	6	7	8
HLA-Cw allele⁴								
*050101-050104	1	2					7	
*0503	1	2		4			7	
*0504	1	2			5		7	
*0505	1	2				6		
*0506	1	2					7	8
*0507N	1	2					7	
*0508	1	2					7	
*0509		2	3					
*0510	1	2					7	
*0511	1	2					7	
*0512		2					7	
*0513	1	2					7	
*0514	1	2					7	
*0515	1	2					7	
*0516	1	2				6	7	
*0517		2	3					
*0113, 0815			3					
*0605, 1221		2						
*070401-070402, 0711, 0712, 080101-080102, 0803, 0806, 0808, 0814							7	
*0741, 0802, 0805, 0807	1						7	
Well No.	1	2	3	4	5	6	7	8

Length of spec.	155	165	155	120	225	270	265	85
PCR product								
Well No.	1	2	3	4	5	6	7	8
*0752				4				
*0804, 0813	1						7	
*0809, 0811					5		7	
*0810		2					7	
*0812	1					6	7	
HLA-Cw allele⁴								
Well No.	1	2	3	4	5	6	7	8
A*2910						6		
B*1533			3					
Well No.	1	2	3	4	5	6	7	8

INTERPRETATION TABLE							
HLA-Cw*05 subtyping							
Amplification patterns of the Cw*0501 to 0517 alleles							
Well							
9	10	11	12	13	14	15	
285	95	285	95	120	240	195	Length of spec.
							PCR product
1070	1070	1070	1070	1070	1070	1070	Length of int.
							pos. control¹
355	485	355	485	176	115	446	5'-primer(s)²
5'-*CC ^{3'}	5'-CAA ^{3'}	5'-TCA ^{3'}	5'-CAA ^{3'}	5'-gCA ^{3'}	5'-ggA ^{3'}	5'-CgT ^{3'}	
				453			
				5'-AAT ^{3'}			
601	538	601	539	256	312	601	3'-primer(s)³
5'-CTT ^{3'}	5'-CCA ^{3'}	5'-CTT ^{3'}	5'-TCA ^{3'}	5'-CCA ^{3'}	5'-AgT ^{3'}	5'-CTT ^{3'}	
				527			
				5'-CCA ^{3'}			
9	10	11	12	13	14	15	Well No.
							HLA-Cw allele⁴
							*050101-050104
							*0503
							*0504
							*0505
							*0506
9							*0507N
	10						*0508
							*0509
		11					*0510
			12				*0511
				13			*0512
					14		*0513
						15	*0514
				13			*0515
							*0516
			12				*0517
							*0113, 0815
							*0605, 1221
							*070401-070402, 0711,
							0712, 080101-080102,
							0803, 0806, 0808, 0814
							*0741, 0802, 0805, 0807
9	10	11	12	13	14	15	Well No.

285	95	285	95	120	240	195	Length of spec.
							PCR product
9	10	11	12	13	14	15	Well No.
							*0752
			12				*0804, 0813
							*0809, 0811
							*0810
							*0812
							HLA-Cw allele⁴
9	10	11	12	13	14	15	Well No.
							A*2910
	10						B*1533
9	10	11	12	13	14	15	Well No.