

Lot No.: **34K**

Lot-specific Information

[www.olerup-ssp.com](http://www.olerup-ssp.com)

## **Olerup SSP<sup>®</sup> HLA-B\*27 unit dose single well<sup>1</sup>**

Product number:	101.911-96u – without Taq polymerase
Lot number:	34K
Expiry date:	2013-July-01
Number of tests:	96
Number of wells per test:	1
Storage - pre-aliquoted primers:	dark at -20°C
- PCR Master Mix:	-20°C
- Control DNAs:	-20°C

**This Product Description is only valid for Lot No. 34K.**

### **CHANGES COMPARED TO THE PREVIOUS *OLERUP SSP*<sup>®</sup> HLA-B\*27 UNIT DOSE SINGLE WELL LOT**

The HLA-B\*27 unit dose single well specificity and interpretation tables have been updated for the HLA-B alleles described since the previous *Olerup SSP*<sup>®</sup> HLA-B\*27 lot (**Lot No. Y54**) was made.

The HLA-B\*27 unit dose primer set is unchanged compared to the previous *Olerup SSP*<sup>®</sup> HLA-B\*27 unit dose single well lot.

1. The primers in this kit do **not** amplify the B\*27:23 and B\*27:29 alleles.

## PRODUCT DESCRIPTION

### HLA-B\*27 SSP typing

#### CONTENT

The primer set contains 5'- and 3'-primers for identifying the HLA-B27 specificity, B\*27:01 to B\*27:73. The primers in this kit do not amplify the B\*27:23 and B\*27:29 alleles.

Positive and negative control DNAs are included in the kit.

DNA 1; a B\*27-positive DNA as a positive control, **IHW 9315, CML, B\*08:01,27:05:02.**

DNA 2; a B\*73-positive DNA as a negative control, **IHW 9280, LK707, B\*52:01:01,73:01.** (A B\*73:01-positive DNA was chosen as negative control, as this is most similar to the B\*27 group of alleles in the primer matching regions.)

#### PLATE LAYOUT

Each 8 well strip is intended for testing 8 sample DNAs.

1	1	1	1	1	1	1	1
---	---	---	---	---	---	---	---

Well No. 1 is marked with the Lot No. '34K' in silver/gray ink.

The cut PCR plate is marked with 'B27'.

A faint row of numbers is seen between wells 1 and 2 or wells 7 and 8 of the PCR trays. These stem from the manufacture of the trays, and should be disregarded.

The PCR plates are covered with a PCR-compatible foil.

**Please note:** When removing each 8 well PCR plate, make sure that the remaining plates stay covered. Use a scalpel or a similar instrument to carefully cut the foil between the plates.

#### INTERPRETATION

Due to the sharing of sequence motifs between HLA-B alleles the B\*44:97 is amplified by the B\*27 unit dose single well primer mix. Thus, the interpretation of B\*27 unit dose single well SSP subtypings will only be influenced by this allele, and not by other HLA Class I alleles.

The primer mix in this kit does not amplify the B\*27:23 and B\*27:29 alleles.

#### UNIQUELY IDENTIFIED ALLELES

All the B\*27 alleles except the B\*27:23 and B\*27:29 alleles, i.e. **B\*27:01 to B\*27:73**, recognized by the HLA Nomenclature Committee in October 2010<sup>1</sup> are identified by the primers in the HLA-B\*27 unit dose single well SSP kit.

<sup>1</sup>HLA-B alleles listed on the IMGT/HLA web page 2010-October-15, release 3.2.0, [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla).

## PROTOCOL

### DNA EXTRACTION

Extracted, highly pure DNA is needed for SSP typings. DNA samples to be used for PCR-SSP HLA typing should be re-suspended in dH<sub>2</sub>O. The A260/A280 ratio should be 1.6 – 2.0 by UV spectrophotometry for optimal band visualization during electrophoresis.

We recommend automated DNA extraction with the QIAGEN EZ1 DSP DNA Blood System. ACD blood should be used as starting material.

Alternatively, the DNA can be extracted by any preferred method yielding pure DNA. When using alternative methods, the DNA concentration should be adjusted to 30 ng/μl. **Do not use heparinised blood with these methods.**

Recommended DNA concentration using:

EZ1-extracted DNA, 15 ng/μl.

DNA extracted by other methods, 30 ng/μl.

Concentrations exceeding 50 ng/μl will increase the risk for nonspecific amplifications and weak extra bands, especially for HLA Class I high resolution SSP typings. If necessary, dilute the extracted DNA in dH<sub>2</sub>O.

### PCR AMPLIFICATION

#### **101.911-96u – without Taq polymerase**

For one HLA-B\*27 typing add at room temperature to a PCR well:

2 μl sample or control DNA (30 ng/μl)

3 μl PCR Master Mix complete with Taq – mix well before taking your aliquot

0.1 μl Taq polymerase (5 units/μl)

4.9 μl dH<sub>2</sub>O

**The 8 well PCR plate is marked with the lot number.** Cover the primer tray(s) with the provided adhesive PCR seals. Check that all reaction wells are completely covered to prevent evaporative loss during PCR amplification. The Olerup SSP<sup>®</sup> Compression Pad (Product No. 103.505-06) can be applied on top of the adhesive PCR seals to prevent evaporation during thermal cycling.

Lot No.: **34K**

Lot-specific Information

[www.olerup-ssp.com](http://www.olerup-ssp.com)

**PCR cycling parameters:**

1.	1 cycle	94°C	2 min	denaturation
2.	10 cycles	94°C	10 sec.	denaturation
		65°C	60 sec.	annealing and extension
3.	20 cycles	94°C	10 sec.	denaturation
		61°C	50 sec.	annealing
		72°C	30 sec.	extension
4.	End - hold	RT		if less than 8 hours
		4°C		if longer than 8 hours

Total reaction volume in each well, 10 µl.

**The same PCR cycling parameters are used for all the *Olerup* SSP kits.**

**AGAROSE GEL ELECTROPHORESIS**

Prepare a 2% (w/v) agarose gel in 0.5 x TBE buffer. Dissolve the agarose by boiling in a microwave oven. Let the gel solution cool to 60°C. Stain the gel prior to casting with ethidium bromide (10 mg/ml), 5 µl per 100 ml gel solution. For maximal ease of handling use our ethidium bromide dropper bottles (Product No. 103.301-10). **Note: Ethidium bromide is a carcinogen. Handle with appropriate personal protective equipment.**

Load the PCR products, preferably using an 8-channel pipette. Load a DNA size marker (100 base pair ladder, DNA Size Marker Product No. 103.202-100 or DNA Size Marker for short gel runs 103.203-100) in one well per row.

Run the gel in 0.5 x TBE buffer, without re-circulation of the buffer, for 15-20 minutes at 8-10 V/cm.

**DOCUMENTATION AND INTERPRETATION**

Put the gel on a UV transilluminator and document by photography.

Record the presence and absence of specific PCR products. The length of the specific PCR product is helpful in the interpretation of the results.

Record the presence of the internal positive control bands.

Lanes without either control band or specific PCR products should be repeated.

Interpret the typings with the ***lot-specific Interpretation and Specificity Tables***.

Lot No.: **34K**

Lot-specific Information

[www.olerup-ssp.com](http://www.olerup-ssp.com)

### PCR MASTER MIX

The PCR Master Mix without *Taq* contains:

nucleotides	final concentration of each dNTP is 200 µM
PCR buffer	final concentrations: 50 mM KCl, 1.5 mM MgCl <sub>2</sub> , 10 mM Tris-HCl pH 8.3, 0.001% w/v gelatin
glycerol	final concentration of glycerol is 5%
cresol red	final concentration of cresol red is 100 µg/ml

**The same PCR Master Mix is used for all *Olerup* SSP kits without Taq polymerase.**

When stored at –20°C, the PCR Master Mix without Taq polymerase is stable for 33 months from the date of manufacture.

## SPECIFICITY TABLE

### HLA-B\*27 SSP typing

Specificity and size of the PCR product of the HLA-B\*27 unit dose single well SSP typing.

Primer Mix	Size of spec. PCR product <sup>1</sup>	Size of control band <sup>2</sup>	Amplified HLA-B*27 alleles <sup>3</sup>	Other amplified HLA-B alleles <sup>4</sup>
1	145 bp, 205 bp	430 bp	*27:01-27:21, 27:24-27:28, 27:30-27:73	*44:97

<sup>1</sup>Alleles are assigned by the presence of specific PCR product(s). However, the sizes of the specific PCR products may be helpful in the interpretation of HLA-B\*27 SSP typings.

When the primers in a primer mix can give rise to specific PCR products of more than one length this is indicated if the size difference is 20 base pairs or more. Size differences shorter than 20 base pairs are not given. For high resolution SSP kits the respective lengths of the specific PCR product(s) of the alleles amplified by these primer mixes are given.

Nonspecific amplifications, i.e. a ladder or a smear of bands, may sometimes be seen. GC-rich primers have a higher tendency of giving rise to nonspecific amplifications than other primers.

PCR fragments longer than the control bands may sometimes be observed. Such bands should be disregarded and do not influence the interpretation of the SSP typings.

PCR fragments migrating faster than the control bands, but slower than a 400 bp fragment may be seen in some gel read-outs. Such bands can be disregarded and do not influence the interpretation of the SSP typings.

Some primers may give rise to primer oligomer artifacts. Sometimes this phenomenon is an inherent feature of the primer pair(s) of a primer mix. More often it is due to other factors such as too low amount of DNA in the PCR reactions, taking too long time in setting up the PCR reactions, working at elevated room temperature or using thermal cyclers that are not pre-heated

<sup>2</sup>The internal positive control primer pairs amplify segments of the human growth hormone gene. The control primer pair gives rise to a band of 430 base pairs.

In the presence of a specific amplification the intensity of the control band often decreases.

<sup>3</sup>The primers in this kit do not amplify the B\*27:23 and B\*27:29 alleles.

<sup>4</sup>Due to the sharing of sequence motifs between HLA-B alleles the B\*44:97 is amplified by the B\*27 unit dose single well primer mix.

<b>INTERPRETATION TABLE</b>		
<b>HLA-B*27 SSP typing</b>		
<b>Amplification pattern of the B*27:01 to 27: 73 alleles<sup>1</sup></b>		
	<b>Well</b>	
	<b>1</b>	
<b>Length of spec. PCR products</b>	<b>145</b>	<b>Length of spec. PCR products</b>
	<b>205</b>	
<b>Length of int. pos. control<sup>2</sup></b>	<b>430</b>	<b>Length of int. pos. control</b>
<b>5'-primer<sup>3</sup></b>	<b>167</b>	<b>5'-primer<sup>3</sup></b>
	5' -gCT 3'	
	<b>363</b>	
	5' -AAT 3'	
<b>3'-primer<sup>4</sup></b>	<b>272</b>	<b>3'-primer<sup>4</sup></b>
	5' -TgC 3'	
	<b>272</b>	
	5' -TgC 3'	
	<b>527</b>	
	5' -CCT 3'	
<b>Well No.</b>	<b>1</b>	<b>Well No.</b>
<b>HLA-B allele<sup>5</sup></b>		<b>HLA-B allele<sup>5</sup></b>
<b>*27:01-27:21, 27:24-27:28, 27:30-27:73</b>	<b>1</b>	<b>*27:01-27:21, 27:24-27:28, 27:30-27:73</b>
<b>*44:97</b>	<b>1</b>	<b>*44:97</b>
<b>HLA-B allele<sup>5</sup></b>		<b>HLA-B allele<sup>5</sup></b>
<b>Well No.</b>	<b>1</b>	<b>Well No.</b>

<sup>1</sup>The primers in this kit do not amplify the B\*27:23 and B\*27:29 alleles.

<sup>2</sup>The internal positive control primer pairs amplify segments of the human growth hormone gene. The control primer pair gives rise to a band of 430 base pairs.

In the presence of a specific amplification the intensity of the control band often decreases.

<sup>3</sup>The nucleotide position, in the 2<sup>nd</sup> and 3<sup>rd</sup> exons, matching the specificity-determining 3'-end of the primer is given. Nucleotide numbering as on the [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla) web site. The sequence of the 3 terminal nucleotides of the primer is given.

<sup>4</sup>The nucleotide position, in the 2<sup>nd</sup> and 3<sup>rd</sup> exons, matching the specificity-determining 3'-end of the primer is given in the anti-sense direction. Nucleotide numbering as on the [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla) web site. The sequence of the 3 terminal nucleotides of the primer is given.

<sup>5</sup>Due to the sharing of sequence motifs between HLA-B alleles the B\*44:97 is amplified by primer mix 1.

The sequence of the B\*270501 allele has been shown to be identical to B\*27:05:02.

The B\*2722 sequence has been shown to be identical to the corrected B\*27:06 sequence.

Lot No.: **34K**

Lot-specific Information

[www.olerup-ssp.com](http://www.olerup-ssp.com)

CELL LINE VALIDATION SHEET				
HLA-B*27 unit dose SSP kit				
				Well
				1
				Production No.
				201074901
IHWC cell line			HLA-B	
1	9001	SA	*07:02	-
2	9280	LK707	*52:01	*73:01 -
3	9011	E4181324	*52:01	-
4	9275	GU373	*15:10	*53:01 -
5	9009	KAS011	*37:01	-
6	9353	SM	*39:01	*51:01 -
7	9020	QBL	*18:01	-
8	9025	DEU	*35:01	-
9	9026	YAR	*38:01	-
10	9107	LKT3	*54:01	-
11	9051	PITOUT	*44:03	-
12	9052	DBB	*57:01	-
13	9004	JESTHOM	*27:05	+
14	9071	OLGA	*15:01	*15:20 -
15	9075	DKB	*40:01	-
16	9037	SWEIG007	*40:02	-
17	9282	CTM3953540	*08:01	*55:01 -
18	9257	32367	*14:01	*56:01 -
19	9038	BM16	*18:01	-
20	9059	SLE005	*40:01	-
21	9064	AMALA	*15:01	-
22	9056	KOSE	*35:03	-
23	9124	IHL	*40:02	*56:02 -
24	9035	JBUSH	*38:01	-
25	9049	IBW9	*14:02	-
26	9285	WT49	*58:01	-
27	9191	CH1007	*07:05	*51:01 -
28	9320	BEL5GB	*44:02	*44:03 -
29	9050	MOU	*44:03	-
30	9021	RSH	*42:01	-
31	9019	DUCAF	*18:01	-
32	9297	HAG	*41:02	-
33	9098	MT14B	*40:01	-
34	9104	DHIF	*38:01	-
35	9302	SSTO	*44:02	-
36	9024	KT17	*15:01	*35:01 -
37	9065	HHKB	*07:02	-
38	9099	LZL	*15:01	-
39	9315	CML	*08:01	*27:05 +
40	9134	WHONP199	*13:02	*46:01 -
41	9055	H0301	*14:02	-
42	9066	TAB089	*46:01	-
43	9076	T7526	*46:01	-
44	9057	TEM	*38:01	-
45	9239	SHJO	*42:01	*50:01 -
46	9013	SCHU	*07:02	-
47	9045	TUBO	*51:01	-
48	9303	TER-ND	*35:01	*44:03 -



## CERTIFICATE OF ANALYSIS

### **Olerup SSP® HLA-B\*27 unit dose single well SSP**

Product number: 101.911-96u – without Taq polymerase  
Lot number: 34K  
Expiry date: 2013-July-01  
Number of tests: 96  
Number of wells per test: 1

#### **Well specifications:**

Well No.	Production No.
1	2010-749-01

The specificity of the primer solutions of the kit has been tested against 48 well characterized cell line DNAs.

**Results:** No false positive or false negative amplifications were obtained.

**Date of approval:** 2011-January-21

**Approved by:**

**Quality Control, Supervisor**

Lot No.: **34K**

Lot-specific Information

[www.olerup-ssp.com](http://www.olerup-ssp.com)

## Declaration of Conformity

**Product name:** Olerup SSP® HLA-B\*27 unit dose single well  
**Product number:** 101.911-96u  
**Lot number:** 34K

**Intended use:** HLA-B\*27 low resolution histocompatibility testing

**Manufacturer:** Olerup SSP AB  
Hasselstigen 1  
SE-133 33 Saltsjöbaden, Sweden  
**Phone:** +46-8-717 88 27  
**Fax:** +46-8-717 88 18

We, Olerup SSP AB, hereby declare that this product, to which this Declaration of Conformity relates is in conformity with the following Standard(s) and other normative document(s) ISO 9001:2008 and ISO 13485:2003, following the provisions of the 98/79/EC Directive on *in vitro* diagnostic medical devices, Annex II List B, conformity assessed using Annex IV, as transposed into the national laws of the Member States of the European Union.

The Technical Documentation File is maintained at Olerup SSP AB, Hasselstigen 1, SE-133 33 Saltsjöbaden, Sweden.

The Authorized Representative located within the Community is: Olerup SSP AB.

Notified Body: Lloyd's Register Quality Assurance Limited, Hiramford, Middlemarch Office Village, Siskin Drive, Coventry CV3 4FJ, United Kingdom. (Notified Body number: 0088.)

Saltsjöbaden, Sweden  
2011-January-21

Olle Olerup  
Managing Director

Lot No.: **34K**

Lot-specific Information

[www.olerup-ssp.com](http://www.olerup-ssp.com)

Lot No.: **34K**

Lot-specific Information

[www.olerup-ssp.com](http://www.olerup-ssp.com)

**ADDRESSES:**

**Manufacturer:**

**Olerup SSP AB**, Hasselstigen 1, SE-133 33 Saltsjöbaden, Sweden.

**Tel:** +46-8-717 88 27

**Fax:** +46-8-717 88 18

**E-mail:** [info-ssp@olerup.com](mailto:info-ssp@olerup.com)

**Web page:** <http://www.olerup-ssp.com>

**Distributed by:**

**Olerup GmbH**, Löwengasse 47 / 6, AT-1030 Vienna, Austria.

**Tel:** +43-1-710 15 00

**Fax:** +43-1-710 15 00 10

**E-mail:** [support-at@olerup.com](mailto:support-at@olerup.com)

**Web page:** <http://www.olerup.com>

**Olerup Inc.**, 901 S. Bolmar St., Suite R, West Chester, PA 19382

**Tel:** 1-877-OLERUP1

**Fax:** 610-344-7989

**E-mail:** [info.us@olerup.com](mailto:info.us@olerup.com)

**Web page:** <http://www.olerup.com>

For information on *Olerup* SSP distributors worldwide, contact **Olerup GmbH**.