

101.125-24/06 – including *Taq* pol., IFU-01.
101.125-24u/06u – without *Taq* pol., IFU-02.

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **83V**

Lot-specific information
Olerup SSP® DRB1*15

Product number:	101.125-24/06 – including <i>Taq</i> pol. 101.125-24u/06u – without <i>Taq</i> pol.
Lot number:	83V
Expiry date:	2016-December-01
Number of tests:	24 test – Product No. 101.125-24/24u 6 tests – Product No. 101.125-06/06u
Number of wells per test:	31+1
Storage - pre-aliquoted primers:	dark at -20°C
- PCR Master Mix:	-20°C
- Adhesive PCR seals	RT
- Product Insert	RT

This Product Description is only valid for Lot No. 83V.

Complete product documentation consists of generic Instructions for Use (IFU), lot specific Product Insert, Worksheet and Certificate.

**CHANGES COMPARED TO THE PREVIOUS OLERUP SSP®
DRB1*15 LOT (23S)**

The DRB1*15 kit is updated for new alleles to enable separation of:

- Confirmed¹ alleles as listed in the IMGT/HLA database
- Polymorphisms in exons outside of the region encoding the peptide binding domain
- Null and Alternatively expressed alleles

A well containing Negative Control primer pairs has been added.

The format of the Product Insert and Worksheet have been changed.

¹As described in section Uniquely Identified Alleles.

The DRB1*15 specificity and interpretation tables have been updated for the DRB1 alleles described since the previous *Olerup SSP®* DRB1*15 lot was made (**Lot No. 23S**).

As of lot series V, the Specificity Table is included in the lot-specific Product Insert, and the Interpretation Table is included in the Worksheet.

101.125-24/06 – including *Taq* pol., IFU-01.
101.125-24u/06u – without *Taq* pol., IFU-02.

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **83V**

Lot-specific information

The primers of the wells detailed below have been exchanged, added or modified compared to the previous lot.

Well	5'-primer	3'-primer	rationale
1	Added	-	5'-primer added for the DRB1*15:01:22 allele.
2	Added	-	5'-primer added for the DRB1*15:01:22 allele.
4	Added	-	5'-primer added for the DRB1*15:01:22 allele.
7	-	Added	3'-primer added for the DRB1*15:56 allele.
9	Added	-	5'-primer added for the DRB1*15:94 allele.
10	-	Added	3'-primer added from well 11.
11	Removed	Moved	3'-primer moved to well 11.
24	Added	-	5'-primer added for the DRB1*15:61 allele.
25	Added	-	5'-primers added for the DRB1*15:67 and DRB1*15:61 alleles.
26	Added	-	5'-primer added for the DRB1*15:75 allele.
27	Added	-	5'-primers added for the DRB1*15:64 and DRB1*15:67 alleles.
32	Moved	Moved	Primer pairs moved to well 11, Negative Control.

Change in revision R01 compared to R00:

1. The DRB1*15:28 allele is amplified by primer mixes 1, 2 and 4. The DRB1*16:12 allele is amplified by primer mixes 5 and 7. Thus, this lot of the DRB1*15 subtyping kit cannot distinguish the DRB1*15:24 and 15:28 alleles. This has been corrected in the Specificity and Interpretation Tables.

101.125-24/06 – including *Taq* pol., IFU-01.
101.125-24u/06u – without *Taq* pol., IFU-02.

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **83V**

Lot-specific information

Well **32** contains Negative Control primer pairs, that will amplify more than 95% of the *Olerup SSP®* HLA Class I, DRB, DQB1, DPB1 and DQA1 amplicons as well as all the amplicons generated by the control primer pairs matching the human growth hormone gene.

HLA-specific PCR product sizes range from 75 to 200 base pairs.
The PCR product generated by the positive control primer pair is 430 base pairs.

Length of PCR product	105	200	105	80	75	80	85
5'-primer¹	164	340	440	45	45	43	36
	5'-CAC ^{3'}	5'-Agg ^{3'}	5'-TTA ^{3'}	5'-Tgg ^{3'}	5'-Tgg ^{3'}	5'-Tgg ^{3'}	5'-TAC ^{3'}
							36
							5'-TAT ^{3'}
3'-primer²	231	2nd I	507	59	58	57	47
	5'-TgC ^{3'}	5'-AAA ^{3'}	5'-TTg ^{3'}	5'-CTC ^{3'}	5'-ggC ^{3'}	5'-CTC ^{3'}	5'-ACA ^{3'}
							48
							5'-gCA ^{3'}
							48
							5'-gCC ^{3'}
							52
							5'-TgT ^{3'}
A*	+	+	+				
B*	+	+	+				
C*	+	+	+				
DRB1				+	+		
DRB3				+	+		
DRB5				+			
DQB1					+		
DPB1						+	
DQA1							+

¹The nucleotide position for HLA class I genes and the codon for HLA class II genes, in the 2nd or 3rd exon, matching the specificity-determining 3'-end of the primer is given. Nucleotide and codon numbering as on the www.ebi.ac.uk/imgt/hla web site. The sequence of the 3 terminal nucleotides of the primer is given.

²The nucleotide position for HLA class I genes and the codon for HLA class II genes, in the 2nd or 3rd exon or the 2nd intron, matching the specificity-determining 3'-end of the primer is given in the anti-sense direction. Nucleotide and codon numbering as on the www.ebi.ac.uk/imgt/hla web site. The sequence of the 3 terminal nucleotides of the primer is given.

101.125-24/06 – including *Taq* pol., IFU-01.
101.125-24u/06u – without *Taq* pol., IFU-02.

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **83V**

Lot-specific information

PRODUCT DESCRIPTION

DRB1*15 SSP subtyping

CONTENT

The primer set contains 5'- and 3'-primers for identifying the DRB1*15:01 to DRB1*15:107 alleles.

PLATE LAYOUT

Each test consists of 32 PCR reactions in a 32 well cut PCR plate.

1	2	3	4	5	6	7	8
9	10	11	12	13	14	15	16
17	18	19	20	21	22	23	24
25	26	27	28	29	30	31	NC

The 32 well cut PCR plate is marked with 'DRB1*15' in silver/gray ink.

Well No. 1 is marked with the Lot No. '83V'.

Wells 1 to 31 – DRB1*15 high resolution primers.

Well 32 – Negative Control (NC).

A faint row of numbers is seen between wells 1 and 2 or wells 7 and 8 of the PCR trays. These stem from the manufacture of the trays, and should be disregarded.

The PCR plates are covered with a PCR-compatible foil.

Please note: When removing each 32 well PCR plate, make sure that the remaining plates stay covered. Use a scalpel or a similar instrument to carefully cut the foil between the plates.

INTERPRETATION

Due to the sharing of sequence motifs between DRB1 alleles, non-DRB1*15 alleles will be amplified by primer mixes 1, 3, 4, 5, 7, 12, 15 and 19. Thus, the interpretation of DRB1*15 subtypings is not influenced by other DRB genes.

For further details see Specificity Table.

UNIQUELY IDENTIFIED ALLELES

All the DRB1*15 alleles, i.e. **DRB1*15:01 to DRB1*15:107**, recognized by the HLA Nomenclature Committee in April 2014^{1,2} will be amplified by the primers in the DRB1*15 subtyping kit³.

The DRB1*15 kit enables separation of the confirmed DRB1*15 alleles as listed in the IMGT/HLA database. An HLA allele is listed as confirmed by IMGT/HLA if it has been sequenced by more than a single laboratory or from multiple sources. Current allele confirmation status for DRB1*15 alleles is listed below.

The DRB1*15 kit also enables identification of polymorphisms in exons outside of the region encoding the peptide binding domain and of null and alternatively expressed alleles.

101.125-24/06 – including *Taq* pol., IFU-01.
101.125-24u/06u – without *Taq* pol., IFU-02.

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **83V**

Lot-specific information

The following DRB1*15 alleles can be distinguished by the different sizes of the HLA-specific PCR product:

Alleles	Primer mix
DRB1*15:16, 15:35	28
DRB1*15:42, 15:64	27

The DRB1*15 subtyping kit cannot distinguish the following silent mutations: the DRB1*15:01:01:01-15:01:24 alleles, the DRB1*15:02:01-15:02:06 and DRB1*15:02:08-15:02:09 alleles, the DRB1*15:03:01:01-15:03:01:02 alleles, the DRB1*15:06:01-15:06:02 alleles, the DRB1*15:07:01-15:07:02, the DRB1*15:37:01-15:37:02 alleles or the DRB1*15:66:01-15:66:02 alleles.

¹DRB1 alleles listed on the IMGT/HLA web page 2014-April-14, release 3.16.0, www.ebi.ac.uk/imgt/hla.

²Alleles that have been deleted from or renamed in the official WHO HLA Nomenclature up to and including the last IMGT/HLA database release can be retrieved from web page <http://hla.alleles.org/alleles/deleted.html>.

³This lot of the DRB1*15 subtyping kit cannot distinguish the DRB1*15:24 and 15:28 alleles.

101.125-24/06 – including *Taq* pol., IFU-01.
101.125-24u/06u – without *Taq* pol., IFU-02.

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **83V**

Lot-specific information

ALLELE CONFIRMATION STATUS

Allele	Status ¹	Allele	Status ¹	Allele	Status ¹	Allele	Status ¹
DRB1*15:01:01:01	Confirmed	DRB1*15:04	Confirmed	DRB1*15:41	Unconfirmed	DRB1*15:80N	Unconfirmed
DRB1*15:01:01:02	Unconfirmed	DRB1*15:05	Unconfirmed	DRB1*15:42	Unconfirmed	DRB1*15:81	Unconfirmed
DRB1*15:01:01:03	Unconfirmed	DRB1*15:06:01	Confirmed	DRB1*15:43	Unconfirmed	DRB1*15:82	Unconfirmed
DRB1*15:01:01:04	Unconfirmed	DRB1*15:06:02	Unconfirmed	DRB1*15:44	Unconfirmed	DRB1*15:83	Unconfirmed
DRB1*15:01:02	Confirmed	DRB1*15:07:01	Confirmed	DRB1*15:45	Unconfirmed	DRB1*15:84	Unconfirmed
DRB1*15:01:03	Unconfirmed	DRB1*15:07:02	Unconfirmed	DRB1*15:46	Unconfirmed	DRB1*15:85	Unconfirmed
DRB1*15:01:04	Unconfirmed	DRB1*15:08	Unconfirmed	DRB1*15:47	Unconfirmed	DRB1*15:86	Unconfirmed
DRB1*15:01:05	Unconfirmed	DRB1*15:09	Unconfirmed	DRB1*15:48	Unconfirmed	DRB1*15:87	Unconfirmed
DRB1*15:01:06	Confirmed	DRB1*15:10	Confirmed	DRB1*15:49	Unconfirmed	DRB1*15:88	Unconfirmed
DRB1*15:01:07	Confirmed	DRB1*15:11	Confirmed	DRB1*15:50N	Unconfirmed	DRB1*15:89	Unconfirmed
DRB1*15:01:08	Confirmed	DRB1*15:12	Unconfirmed	DRB1*15:51	Unconfirmed	DRB1*15:90	Unconfirmed
DRB1*15:01:09	Unconfirmed	DRB1*15:13	Unconfirmed	DRB1*15:52	Unconfirmed	DRB1*15:91	Unconfirmed
DRB1*15:01:10	Unconfirmed	DRB1*15:14	Confirmed	DRB1*15:53	Unconfirmed	DRB1*15:92	Unconfirmed
DRB1*15:01:11	Confirmed	DRB1*15:15	Confirmed	DRB1*15:54	Unconfirmed	DRB1*15:93	Unconfirmed
DRB1*15:01:12	Confirmed	DRB1*15:16	Unconfirmed	DRB1*15:55	Confirmed	DRB1*15:94	Confirmed
DRB1*15:01:13	Confirmed	DRB1*15:17N	Unconfirmed	DRB1*15:56	Confirmed	DRB1*15:95	Unconfirmed
DRB1*15:01:14	Unconfirmed	DRB1*15:18	Confirmed	DRB1*15:57	Confirmed	DRB1*15:96	Unconfirmed
DRB1*15:01:15	Unconfirmed	DRB1*15:19	Unconfirmed	DRB1*15:58	Unconfirmed	DRB1*15:97	Unconfirmed
DRB1*15:01:16	Unconfirmed	DRB1*15:20	Confirmed	DRB1*15:59	Unconfirmed	DRB1*15:98	Unconfirmed
DRB1*15:01:17	Unconfirmed	DRB1*15:21	Unconfirmed	DRB1*15:60	Unconfirmed	DRB1*15:99	Unconfirmed
DRB1*15:01:18	Unconfirmed	DRB1*15:22	Confirmed	DRB1*15:61	Confirmed	DRB1*15:100	Unconfirmed
DRB1*15:01:19	Unconfirmed	DRB1*15:23	Confirmed	DRB1*15:62	Unconfirmed	DRB1*15:101	Unconfirmed
DRB1*15:01:20	Unconfirmed	DRB1*15:24	Confirmed	DRB1*15:63	Unconfirmed	DRB1*15:102	Unconfirmed
DRB1*15:01:21	Unconfirmed	DRB1*15:25	Unconfirmed	DRB1*15:64	Confirmed	DRB1*15:103	Unconfirmed
DRB1*15:01:22	Unconfirmed	DRB1*15:26	Unconfirmed	DRB1*15:65	Unconfirmed	DRB1*15:104	Unconfirmed
DRB1*15:01:23	Unconfirmed	DRB1*15:27	Confirmed	DRB1*15:66:01	Unconfirmed	DRB1*15:105	Unconfirmed
DRB1*15:01:24	Unconfirmed	DRB1*15:28	Unconfirmed	DRB1*15:66:02	Unconfirmed	DRB1*15:106	Unconfirmed
DRB1*15:02:01	Confirmed	DRB1*15:29	Unconfirmed	DRB1*15:67	Confirmed	DRB1*15:107	Unconfirmed
DRB1*15:02:02	Confirmed	DRB1*15:30	Unconfirmed	DRB1*15:68	Confirmed		
DRB1*15:02:03	Unconfirmed	DRB1*15:31	Unconfirmed	DRB1*15:69	Unconfirmed		
DRB1*15:02:04	Confirmed	DRB1*15:32	Unconfirmed	DRB1*15:70	Unconfirmed		
DRB1*15:02:05	Unconfirmed	DRB1*15:33	Confirmed	DRB1*15:71	Unconfirmed		
DRB1*15:02:06	Unconfirmed	DRB1*15:34	Unconfirmed	DRB1*15:72	Unconfirmed		
DRB1*15:02:07	Unconfirmed	DRB1*15:35	Confirmed	DRB1*15:73	Unconfirmed		
DRB1*15:02:08	Unconfirmed	DRB1*15:36	Confirmed	DRB1*15:74	Confirmed		
DRB1*15:02:09	Unconfirmed	DRB1*15:37:01	Confirmed	DRB1*15:75	Confirmed		
DRB1*15:02:10	Unconfirmed	DRB1*15:37:02	Confirmed	DRB1*15:76	Unconfirmed		
DRB1*15:03:01:01	Confirmed	DRB1*15:38	Confirmed	DRB1*15:77	Unconfirmed		
DRB1*15:03:01:02	Unconfirmed	DRB1*15:39	Unconfirmed	DRB1*15:78	Confirmed		
DRB1*15:03:02	Confirmed	DRB1*15:40	Confirmed	DRB1*15:79	Unconfirmed		

¹Allele status “confirmed” or “unconfirmed” as listed on the IMGT/HLA web page 2014-April-14, release 3.16.0, www.ebi.ac.uk/imgt/hla.

RESOLUTION IN HOMO- AND HETEROZYGOTES

Results file with resolution in DRB1*15 homo- and heterozygotes is available upon request.

101.125-24/06 – including *Taq* pol., IFU-01.
101.125-24u/06u – without *Taq* pol., IFU-02.

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **83V**

Lot-specific information
SPECIFICITY TABLE

DRB1*15 SSP subtyping

Specificities and sizes of the PCR products of the 31+1 primer mixes used for DRB1*15 SSP subtyping

Primer Mix	Size of spec. PCR product ¹	Size of control band ²	Amplified DRB1*15 alleles ³	Other amplified DRB1 alleles ⁴
1	140 bp	515 bp	*15:01:01:01-15:02:09, 15:03:01:01-15:06:02, 15:08, 15:10, 15:12-15:33, 15:35-15:47, 15:49-15:58, 15:60-15:68, 15:70-15:87, 15:89-15:95, 15:97-15:107	*16:09:01-16:10
2	200 bp	515 bp	*15:01:01:01-15:02:06, 15:02:08-15:03:02, 15:06:01-15:09, 15:11-15:14, 15:16, 15:18 ^w , 15:19-15:20, 15:22, 15:24, 15:26, 15:28-15:30, 15:32-15:33, 15:35-15:48, 15:51-15:53, 15:55-15:62, 15:64-15:65, 15:67-15:68, 15:71-15:72, 15:74-15:79, 15:81-15:83, 15:85-15:87, 15:90, 15:92-15:99, 15:101-15:102, 15:104-15:107	
3	150 bp	430 bp	*15:01:01:01-15:02:10, 15:06:01-15:09, 15:11-15:14, 15:16, 15:18 ^w , 15:19-15:20, 15:22, 15:24, 15:26, 15:28-15:30, 15:32-15:33, 15:35-15:48, 15:51-15:53, 15:55-15:56, 15:59-15:62, 15:64-15:65, 15:67-15:69, 15:71-15:72, 15:74-15:77, 15:79, 15:81-15:83, 15:85-15:87, 15:90, 15:92-15:93, 15:95-15:99, 15:101-15:102, 15:104-15:107	*04:73, 04:105:01-04:105:02, 11:82, 13:09, 13:129, 14:37
4	260 bp	430 bp	*15:01:01:01-15:01:24, 15:03:01:01-15:07:02, 15:09-15:10, 15:12-15:13, 15:16-15:18, 15:20-15:25, 15:28, 15:32-15:33, 15:35-15:37:02, 15:40-15:43, 15:45-15:46, 15:48-15:57, 15:59, 15:61-15:62, 15:64-15:67, 15:70-15:73, 15:75-15:77, 15:79, 15:81-15:84, 15:86-15:98, 15:100, 15:102, 15:106-15:107	*16:15, 16:23
5	260 bp	430 bp	*15:02:01-15:02:10, 15:08, 15:11, 15:14-15:15, 15:19, 15:26-15:27, 15:29-15:31, 15:34, 15:38-15:39, 15:44, 15:47, 15:58, 15:60, 15:63, 15:68, 15:78, 15:80N, 15:99, 15:101, 15:103-15:105	*16:01:01-16:05:02, 16:07-16:11-16:14, 16:16-16:22, 16:24
6	150 bp	430 bp	*15:03:01:01-15:03:02, 15:57, 15:78, 15:94	
	180 bp		*15:14	
7	135 bp	430 bp	*15:07:01-15:07:02, 15:11, 15:34, 15:56, 15:88, 15:96	*16:01:01-16:05:02, 16:07-16:08, 16:11-16:24

101.125-24/06 – including *Taq* pol., IFU-01.
101.125-24u/06u – without *Taq* pol., IFU-02.

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **83V**

Lot-specific information

8	200 bp 245 bp	430 bp	*15:05, 15:23, 15:25, 15:27, 15:31, 15:34, 15:54, 15:63, 15:66:01- 15:66:02, 15:73, 15:100 *15:36	
9⁵	90 bp 215 bp	430 bp	*15:06:01-15:06:02, 15:77, 15:94 *15:19, 15:69	
10	170 bp 200 bp	430 bp	*15:12 *15:04, 15:15, 15:88	
11	155 bp 200 bp	430 bp	*15:47, 15:65 *15:49	
12⁵	95 bp	430 bp	*15:09, 15:48	*04:162, 11:88, 13:177
13⁵	110 bp 215 bp	430 bp	*15:32 *15:10, 15:84, 15:103	
14	200 bp	430 bp	*15:13, 15:17N	
15	200 bp	515 bp	*15:10, 15:21	*16:05:01-16:05:02, 16:07
16	200 bp 260 bp	430 bp	*15:08 *15:74	
17	160 bp	430 bp	*15:18	
18	205 bp	515 bp	*15:20	
19⁷	175 bp 225 bp	430 bp	*15:30 *15:21, 15:25, , 15:37:01-15:37:02, 15:57, 15:100, 15:104	*16:04, 16:18
20	165 bp 200 bp	430 bp	*15:22 *15:27, 15:34, 15:54, 15:66:01- 15:66:02	
21	165 bp	430 bp	*15:03:01:01-15:03:02, 15:23, 15:78, 15:94	
22⁶	165 bp 200 bp	430 bp	*15:24 *15:28, 15:40	
23⁵	125 bp 170 bp 220 bp	430 bp	*15:55 *15:29 *15:25	
24	140 bp 200 bp	430 bp	*15:61 *15:26, 15:40 15:43	
25^{5,7}	70 bp 135 bp 195 bp	430 bp	*15:39 *15:61, 15:67 *15:33	
26^{5,7}	75 bp 125 bp	430 bp	*15:38, 15:63 *15:75	
27^{5,7}	110 bp 135 bp 175 bp	430 bp	*15:64 *15:67 *15:14, 15:42	
28⁷	135 bp 255 bp	430 bp	*15:16 *15:35, 15:96	
29	200 bp	430 bp	*15:41, 15:50N, 15:80N	
30	165 bp 220 bp	430 bp	*15:45 *15:44	
31	150 bp 215 bp	515bp	*15:68 *15:46	
32⁸			Negative Control	

101.125-24/06 – including *Taq* pol., IFU-01.
101.125-24u/06u – without *Taq* pol., IFU-02.

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: 83V

Lot-specific information

¹Alleles are assigned by the presence of specific PCR product(s). However, the sizes of the specific PCR products may be helpful in the interpretation of DRB1*15 SSP subtypings.

When the primers in a primer mix can give rise to HLA-specific PCR products of more than one length this is indicated if the size difference is more than 20 base pairs. Size differences of 20 base pairs or less are not given. For high resolution SSP kits, the alleles listed are specified according to amplicon length. Nonspecific amplifications, i.e. a ladder or a smear of bands, may sometimes be seen. GC-rich primers have a higher tendency of giving rise to nonspecific amplifications than other primers.

PCR fragments longer than the control bands may sometimes be observed. Such bands should be disregarded and do not influence the interpretation of the SSP typings.

PCR fragments migrating faster than the control bands, but slower than a 400 bp fragment may be seen in some gel read-outs. Such bands can be disregarded and do not influence the interpretation of the SSP typings.

Some primers may give rise to primer oligomer artifacts. Sometimes this phenomenon is an inherent feature of the primer pair(s) of a primer mix. More often it is due to other factors such as too low amount of DNA in the PCR reactions, taking too long time in setting up the PCR reactions, working at elevated room temperature or using thermal cyclers that are not pre-heated.

²The internal positive control primer pairs amplify segments of the human growth hormone gene. The internal positive control bands are 430 or 515 base pairs respectively, well distribution as outlined in the table. Well number 1 contains the longer, 515 bp, internal positive control band. The well distribution of the internal controls can help in orientation of the kit on gel photo, as well as allow for kit identification. In the presence of a specific amplification the intensity of the control band often decreases.

³For several DRB1 alleles 1st and/or 3rd exon(s) and beyond, as well as intron nucleotide sequences, are not available. In these instances it is not known whether some of the primers of the SSP sets are completely matched with the target sequences or not. Assumption is made that unknown sequences in these regions are conserved within allelic groups.

⁴Due to the sharing of sequence motifs between DRB1 alleles, non-DRB1*15 alleles will be amplified by primer mixes 1, 3, 4, 5, 7, 12, 15 and 19. Thus, the interpretation of DRB1*15 subtypings is not influenced by other DRB genes.

⁵HLA-specific PCR products shorter than 125 base pairs have a lower intensity and are less sharp than longer PCR products.

⁶Primer mix 22 may have a tendency of unspecific amplification.

⁷Primer mixes 19, 25, 26, 27 and 28 have a tendency to giving rise to primer oligomer formation.

⁸Primer mix 32 contains a negative control, which will amplify more than 95% of HLA amplicons as well as the amplicons generated by control primer pairs. PCR product sizes range from 75 to 200 base pairs. The PCR product generated by the control primer pair is 430 base pairs.

‘w’, may be weakly amplified.

101.125-24/06 – including *Taq* pol., IFU-01.
101.125-24u/06u – without *Taq* pol., IFU-02.

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **83V**

Lot-specific information

PRIMER SPECIFICATION

Well No.	1	2	3	4	5	6	7	8	9	10	11	12
Length of spec. PCR product	140	200	150	260	260	150	135	200	90	170	155	95
Length of int. pos. control ¹	515	515	430	430	430	430	430	430	430	430	430	430
5'-primer(s) ²	13(126) 5'-Agg 3'	13(126) 5'-Agg 3'	29(175) 5'-gAT 3'	13(126) 5'-Agg 3'	13(126) 5'-Agg 3'	20(146) 5'-TgC 3'	13(126) 5'-Agg 3'	13(126) 5'-Agg 3'	5(103) 5'-CAg 3'	13(126) 5'-Agg 3'	13(126) 5'-Agg 3'	29(175) 5'-gAT 3'
				13(126) 5'-AgA 3'		29(175) 5'-gAC 3'			11(121) 5'-CTg 3'			
									50(236) 5'-ggC 3'			
									50(236) 5'-ggg 3'			
3'-primer(s) ³	47(227) 5'-ggA 3'	66(286) 5'-gAT 3'	66(286) 5'-gAT 3'	86(344) 5'-CCA 3'	86(344) 5'-CAC 3'	66(286) 5'-gAT 3'	44(218) 5'-CCC 3'	66(286) 5'-gAg 3'	66(286) 5'-gAT 3'	56(256) 5'-gCT 3'	51(239) 5'-CCC 3'	48(230) 5'-CCT 3'
							47(227) 5'-ggT 3'	66(286) 5'-gAg 3'		66(286) 5'-gAA 3'	67(287) 5'-ggg 3'	
								81(329) 5'-TgC 3'				
Well No.	1	2	3	4	5	6	7	8	9	10	11	12

Well No.	13	14	15	16	17	18	19	20	21	22	23	24
Length of spec. PCR product	110	200	200	200	160	205	175	165	165	165	125	140
Length of int. pos. control ¹	430	430	515	430	430	515	430	430	430	430	430	430
5'-primer(s) ²	13(126) 5'-Agg 3'	13(126) 5'-Agg 3'	13(126) 5'-Agg 3'	13(126) 5'-Agg 3'	29(175) 5'-gAT 3'	29(175) 5'-gAT 3'	13(126) 5'-Agg 3'	13(126) 5'-Agg 3'	29(175) 5'-gAC 3'	13(126) 5'-AAg 3'	13(126) 5'-Agg 3'	15(131) 5'-gTA 3'
										15(131) 5'-gTA 3'		19(143) 5'-CAg 3'
										25(161) 5'-gCA 3'		20(148) 5'-ggT 3'
												37(199) 5'-CCA 3'
3'-primer(s) ³	36(196) 5'-ggT 3'	63(277) 5'-Tg 3'	66(286) 5'-gAT 3'	65(283) 5'-gTT 3'	69(295) 5'-CTC 3'	83(337) 5'-CCT 3'	57(258) 5'-gCC 3'	55(251) 5'-gCA 3'	70(298) 5'-CgC 3'	66(286) 5'-gAT 3'	40(208) 5'-gTg 3'	70(298) 5'-CgC 3'
	71(299) 5'-gCT 3'	69(294) 5'-TgT 3'	66(286) 5'-gAT 3'	87(347) 5'-TCC 3'			73(307) 5'-CAg 3'	66(286) 5'-gAg 3'			57(257) 5'-CgA 3'	
							77(317) 5'-AgT 3'				73(305) 5'-ggC 3'	
							77(317) 5'-AAT 3'					
Well No.	13	14	15	16	17	18	19	20	21	22	23	24

101.125-24/06 – including *Taq* pol., IFU-01.
101.125-24u/06u – without *Taq* pol., IFU-02.

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **83V**

Lot-specific information

Well No.	25	26	27	28	29	30	31
Length of spec.	70	75	110	135	200	165	150
PCR product	135	125	135	255		220	215
	195		175				
Length of int.	430	430	430	430	430	430	515
pos. control ¹							
5'-primer(s) ²	14(130)	43(216)	20(146)	13(126)	13(126)	13(126)	13(126)
	5' -AgA 3'	5' -gAg 3'	5' -TgC 3'	5' -Agg 3'	5' -Agg 3'	5' -Agg 3'	5' -Agg 3'
	36(194)	60(266)	23(155)				
	5' -ggg 3'	5' -gTC 3'	5' -gCA 3'				
	37(199)		36(194)				
	5' -CCA 3'		5' -ggg 3'				
	57(259)		43(217)				
	5' -ACA 3'		5' -ACA 3'				
3'-primer(s) ³	66(286)	70(298)	66(286)	44(220)	62(273)	54(250)	49(233)
	5' -gAT 3'	5' -CgC 3'	5' -gAT 3'	5' -CCT 3'	5' -CTC 3'	5' -CCA 3'	5' -CCT 3'
				85(341)	69(295)	73(305)	71(301)
				5' -CAg 3'	5' -.Tg 3'	5' -CgA 3'	5' -CCC 3'
Well No.	25	26	27	28	29	30	31

¹The internal positive control primer pairs amplify segments of the human growth hormone gene. The internal positive control bands are 430 or 515 base pairs respectively, well distribution as outlined in the table. Well number 1 contains the longer, 515 bp, internal positive control band. The well distribution of the internal controls can help in orientation of the kit on gel photo, as well as allow for kit identification. In the presence of a specific amplification the intensity of the control band often decreases

²The nucleotide position matching the specificity-determining 3'-end of the primer is given. Nucleotide numbering as on the www.ebi.ac.uk/imgt/hla web site. The sequence of the 3 terminal nucleotides of the primer is given.

³The nucleotide position matching the specificity-determining 3'-end of the primer is given in the anti-sense direction. Nucleotide numbering as on the www.ebi.ac.uk/imgt/hla web site. The sequence of the 3 terminal nucleotides of the primer is given.

101.125-24/06 – including *Taq* pol., IFU-01.
101.125-24u/06u – without *Taq* pol., IFU-02.

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **83V**

Lot-specific information

CELL LINE VALIDATION SHEET																				
DRB1*15 SSP subtyping kit²																				
				Well																
				1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	
				Prod. No.:	201436301	201436302	201436303	201436304	201436305	201436306	201436307	201436308	201436309	201436310	201436332	201436312	201436313	201436314	201436315	201436316
IHWC cell line¹		DRB1																		
1	9001	SA	*01:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
2	9280	LK707	*15:02	*04:05	+	+	+	-	+	-	-	-	-	-	-	-	-	-	-	-
3	9011	E4181324	*15:02		+	+	+	-	+	-	-	-	-	-	-	-	-	-	-	-
4	9275	GU373	*03:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
5	9009	KAS011	*16:01		-	-	-	-	+	-	+	-	-	-	-	-	-	-	-	-
6	9353	SM	*04:07	*08:03	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
7	9020	QBL	*03:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
8	9025	DEU	*04:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
9	9026	YAR	*04:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
10	9107	LKT3	*04:05		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
11	9051	PITOUT	*07:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
12	9052	DBB	*07:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
13	9004	JESTHOM	*01:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
14	9071	OLGA	*08:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
15	9075	DKB	*09:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
16	9037	SWEIG007	*11:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
17	9282	CTM3953540	*03:01	*13:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
18	9257	32367	*09:01	*11:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
19	9038	BM16	*12:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
20	9059	SLE005	*13:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
21	9064	AMALA	*14:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
22	9056	KOSE	*13:02	*14:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
23	9124	IHL	*08:03	*14:14	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
24	9035	JBUSH	*11:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
25	9049	IBW9	*07:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
26	9285	WT49	*03:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
27	9191	CH1007	*04:05	*10:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
28	9320	BEL5GB	*04:16	*07:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
29	9050	MOU	*07:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
30	9021	RSH	*03:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
31	9019	DUCAF	*03:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
32	9297	HAG	*13:03		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
33	9098	MT14B	*04:04		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
34	9104	DHIF	*11:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
35	9302	SSTO	*04:03		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
36	9024	KT17	*04:03	*04:06	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
37	9065	HHKB	*13:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
38	9099	LZL	*14:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
39	9315	CML	*03:01	*04:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
40	9134	WHONP199	*07:01	*09:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
41	9055	H0301	*13:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
42	9066	TAB089	*08:03		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
43	9076	T7526	*09:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
44	9057	TEM	*14:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
45	9239	SHJO	*07:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
46	9013	SCHU	*15:01		+	+	+	+	-	-	-	-	-	-	-	-	-	-	-	-
47	9045	TUBO	*11:04	*12:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
48	9303	TER-ND	*01:03		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-

101.125-24/06 – including *Taq* pol., IFU-01.
101.125-24u/06u – without *Taq* pol., IFU-02.

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **83V**

Lot-specific information

CELL LINE VALIDATION SHEET																		
DRB1*15 SSP subtyping kit²																		
				Well														
				17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
				Prod. No.:														
				201436317	201436318	201436319	201436320	201436321	201436322	201436323	201436324	201436325	201436326	201436327	201436328	201436329	201436330	201436331
	IHWG cell line¹	DRB1																
1	9001 SA	*01:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
2	9280 LK707	*15:02	*04:05	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
3	9011 E4181324	*15:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
4	9275 GU373	*03:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
5	9009 KAS011	*16:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
6	9353 SM	*04:07	*08:03	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
7	9020 QBL	*03:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
8	9025 DEU	*04:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
9	9026 YAR	*04:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
10	9107 LKT3	*04:05		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
11	9051 PITOUT	*07:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
12	9052 DBB	*07:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
13	9004 JESTHOM	*01:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
14	9071 OLGA	*08:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
15	9075 DKB	*09:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
16	9037 SWEIG007	*11:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
17	9282 CTM3953540	*03:01	*13:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
18	9257 32367	*09:01	*11:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
19	9038 BM16	*12:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
20	9059 SLE005	*13:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
21	9064 AMALA	*14:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
22	9056 KOSE	*13:02	*14:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
23	9124 IHL	*08:03	*14:14	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
24	9035 JBUSH	*11:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
25	9049 IBW9	*07:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
26	9285 WT49	*03:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
27	9191 CH1007	*04:05	*10:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
28	9320 BEL5GB	*04:16	*07:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
29	9050 MOU	*07:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
30	9021 RSH	*03:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
31	9019 DUCAF	*03:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
32	9297 HAG	*13:03		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
33	9098 MT14B	*04:04		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
34	9104 DHIF	*11:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
35	9302 SSTO	*04:03		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
36	9024 KT17	*04:03	*04:06	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
37	9065 HHKB	*13:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
38	9099 LZL	*14:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
39	9315 CML	*03:01	*04:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
40	9134 WHONP199	*07:01	*09:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
41	9055 H0301	*13:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
42	9066 TAB089	*08:03		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
43	9076 T7526	*09:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
44	9057 TEM	*14:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
45	9239 SHJO	*07:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
46	9013 SCHU	*15:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
47	9045 TUBO	*11:04	*12:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
48	9303 TER-ND	*01:03		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-

¹The provided cell line HLA specificities are retrieved from the <http://www.ihwg.org/hla> web site. The specificity of an individual cell line may thus be subject to change.

101.125-24/06 – including *Taq* pol., IFU-01.
101.125-24u/06u – without *Taq* pol., IFU-02.

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: 83V

Lot-specific information

²The specificity of each primer solution in the kit has been tested against 48 well characterized cell line DNAs and where applicable, additional cell line DNAs.

No DNAs carrying the alleles to be amplified by primer solutions 8, 9, 11 to 20 and 22 to 31 were available. The specificities of the primers in primer solutions 8, 11 to 13, 15, 19, 20, 22, 23 and 28 were tested by separately adding one 5'-primer, respectively one 3'-primer. In primer solutions 14, 16 to 18 and 29 to 31 it was only possible to test the 5'-primers, the 3'-primers were not possible to test. In primer solutions 9, 24 to 27 it was only possible to test the 3'-primers, the 5'-primers were not possible to test. In primer solutions 6 and 22 one respective two of the 5'-primers were not possible to test, and in primer solutions 2, 3, 7, 8, 11, 13, 19, 20, 23 and 28 one or two of the 3'-primers were not possible to test.

101.125-24/06 – including *Taq* pol., IFU-01.
101.125-24u/06u – without *Taq* pol., IFU-02.

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **83V**

Lot-specific information

101.125-24/06 – including *Taq* pol., IFU-01.
101.125-24u/06u – without *Taq* pol., IFU-02.

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **83V**

Lot-specific information

ADDRESSES:

Manufacturer:

Olerup SSP AB, Franzengatan 5, SE-112 51 Stockholm, Sweden.

Tel: +46-8-717 88 27

Fax: +46-8-717 88 18

E-mail: info-ssp@olerup.com

Web page: <http://www.olerup-ssp.com>

Distributed by:

Olerup GmbH, Löwengasse 47 / 6, AT-1030 Vienna, Austria.

Tel: +43-1-710 15 00

Fax: +43-1-710 15 00 10

E-mail: support-at@olerup.com

Web page: <http://www.olerup.com>

Olerup Inc., 901 S. Bolmar St., Suite R, West Chester, PA 19382

Tel: 1-877-OLERUP1

Fax: 610-344-7989

E-mail: info.us@olerup.com

Web page: <http://www.olerup.com>

For information on *Olerup* SSP distributors worldwide, contact **Olerup GmbH**.