

101.403.48/12 – including *Taq* pol., IFU-01
 101.403.48u/12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
 “Instructions for Use” (IFU)

Lot No.: **28V**

Lot-specific information

Olerup SSP® HLA-A low resolution screening

Product number:	101.403-48/12 – including <i>Taq</i> polymerase 101.403-48u/12u – without <i>Taq</i> polymerase
Lot number:	28V
Expiry date:	2016-July-01
Number of tests:	48 tests – Product No. 101.403-48/48u 12 tests – Product No. 101.403-12/12u
Number of wells per test:	23 +1
Storage - pre-aliquoted primers:	dark at -20°C
- PCR Master Mix:	-20°C
- Adhesive PCR seals	RT
- Product Insert	RT

This Product Description is only valid for Lot No. 28V.

Complete product documentation consists of generic Instructions for Use (IFU), lot specific Product Insert, Worksheet and Certificate.

**CHANGES COMPARED TO THE PREVIOUS OLERUP SSP®
 HLA-A LOW RESOLUTION SCREENING LOT (71N)**

The Lot-specific Information for HLA-A low resolution screening including and without *Taq* polymerase is described in one common Product Insert.

The format of the Product Insert and Worksheet have been changed.

The **HLA-A low resolution Screening** specificity and interpretation tables have been updated for the HLA-A alleles described since the previous *Olerup SSP®* HLA-A low resolution Screening lot was made (**Lot No. 71N**).

As of lot series V, the Specificity Table is included in the lot-specific Product Insert, and the Interpretation Table is included in the Worksheet.

101.403.48/12 – including *Taq* pol., IFU-01
 101.403.48u/12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
 “Instructions for Use” (IFU)

Lot No.: **28V**

Lot-specific information

The primers of the wells detailed below have been exchanged, modified or added compared to the previous lot.

Well	5'-primer	3'-primer	rationale
2	Exchanged	-	5'-primer exchanged for improved specificity of primer pair.
3	-	Added	3'-primer added for the A*03:05:02 allele, 3'-primer added for the A*03:167 allele.
5	-	Exchanged	3'-primer and positive control primer pair exchanged for improved resolution of HLA-specific PCR product and positive control.
6	-	Exchanged	3'-primers exchanged for resolution of the A*23:14:02 and A*23:34 alleles.
9	Exchanged	-	Exchanged 5'-primer for the A*25:17 allele.
10	Added	-	5'-primer added for the A*26:72 allele, 5'-primer added for the A*26:76 allele, positive control primer pair exchanged.
12	Added	Added	5'-primer added for the A*34:10N allele, 3'-primer added for the A*34:11 allele.
13	-	Added	3'-primer added for the A*25:18 allele.
14	Added	Exchanged	5'-primer added for the A*29:37 allele, 3'-primer exchanged for resolution of the A*29:48 allele.
15	Added	-	5'-primer added for the A*30:72 allele.
16	-	Added	3'-primer added for the A*31:01:07 allele.
18	Added	-	5'-primer added for the A*33:51 allele.
19	Exchanged	Exchanged	Exchanged primer pair for the A*74 alleles.
21	Added	Added	Primer pair added for the A*23:57 allele.

Change in revision R01 compared to R00:

- Primer mix 19 does not amplify the A*74:07 allele, and primer mix 11 does not amplify the A*31:03 and A*66:15 alleles. This has been corrected in the Specificity and Interpretation tables.
 The A*32:01:14, 32:03, 32:38 and the A*74:07 alleles give rise to identical amplification patterns with this lot of the HLA-A low resolution screening primer set. These alleles can be separated by the respective high resolution primer sets.

Change in revision R02 compared to R01:

- Primer mix 3 may have tendency of unspecific amplification. A footnote has been added in the Specificity Table.

101.403.48/12 – including *Taq* pol., IFU-01
 101.403.48u/12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
 “Instructions for Use” (IFU)

Lot No.: **28V**

Lot-specific information

Well **24** contains Negative Control primer pairs, that will amplify more than 95% of the *Olerup SSP*® HLA Class I, DRB, DQB1 and DPB1 amplicons as well as amplicons generated by a control primer pair.

PCR product sizes range from 75 to 430 base pairs.
 The PCR product generated by the control primer pair is 430 base pairs.

Length of PCR product	105	200	105	80	75	80
5'-primer¹	164	340	440	45	45	43
	5'-CAC ^{3'}	5'-Agg ^{3'}	5'-TTA ^{3'}	5'-Tgg ^{3'}	5'-Tgg ^{3'}	5'-Tgg ^{3'}
3'-primer²	231	2nd I	507	59	58	57
	5'-TgC ^{3'}	5'-AAA ^{3'}	5'-TTg ^{3'}	5'-CTC ^{3'}	5'-ggC ^{3'}	5'-CTC ^{3'}
A*	+	+	+			
B*	+	+	+			
C*	+	+	+			
DRB1				+	+	
DRB3				+	+	
DRB5				+		
DQB1					+	
DPB1						+

¹The nucleotide position for HLA class I genes and the codon for HLA class II genes, in the 2nd or 3rd exon, matching the specificity-determining 3'-end of the primer is given. Nucleotide and codon numbering as on the www.ebi.ac.uk/imgt/hla web site. The sequence of the 3 terminal nucleotides of the primer is given.

²The nucleotide position for HLA class I genes and the codon for HLA class II genes, in the 2nd or 3rd exon or the 2nd intron, matching the specificity-determining 3'-end of the primer is given in the anti-sense direction. Nucleotide and codon numbering as on the www.ebi.ac.uk/imgt/hla web site. The sequence of the 3 terminal nucleotides of the primer is given.

101.403.48/12 – including *Taq* pol., IFU-01
 101.403.48u/12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
 “Instructions for Use” (IFU)

Lot No.: **28V**

Lot-specific information

PRODUCT DESCRIPTION

HLA-A low resolution screening

CONTENT

The primer set contains 5'- and 3'-primers for grouping the HLA-A*01:01 to A*80:03 alleles into the corresponding serological groups A1 to A80.

PLATE LAYOUT

Each test consists of 24 PCR reactions in a 24 well PCR plate.

1	2	3	4	5	6	7	8
9	10	11	12	13	14	15	16
17	18	19	20	21	22	23	NC

The 24 well PCR plate is marked with ‘HLA-A low screening’ in silver/gray ink.

Well No. 1 is marked with the Lot No. ‘28V’.

Wells 1 to 23 – HLA-A low resolution screening primers.

Well 24 – Negative Control.

A faint row of numbers is seen between wells 1 and 2 or wells 7 and 8 of the PCR trays. These stem from the manufacture of the trays, and should be disregarded.

The PCR plates are covered with a PCR-compatible foil.

Please note: When removing each 24 well PCR plate, make sure that the remaining plates stay covered. Use a scalpel or a similar instrument to carefully cut the foil between the plates.

INTERPRETATION

Only HLA-A alleles will be amplified by the 24 wells of the HLA-A low resolution screening primer set, except that a few HLA-B and HLA-C alleles will be amplified by primer mixes 5, 6 and 9.

For further details see Specificity Table.

UNIQUELY IDENTIFIED ALLELES

All the HLA-A alleles, i.e. **A*01:01 to A*80:03**, recognized by the HLA Nomenclature Committee in October 2013^{1,2} will be amplified by the primers in the HLA-A low resolution screening primer set. The HLA-A alleles will be grouped into their corresponding serological specificities³.

The following HLA-A low alleles can be distinguished by the different sizes of the HLA-specific PCR product:

Alleles	Primer mix
A*01:136, 11:94, 11:112	11
A*03:88, 11:130	11
A*03:152, 24:92	5
A*23:14:01-23:14:02, 24:24, 24:71	5
A*24:14-24:15, 24:51-24:53, 24:57, 24:64, 24:94, 24:114, 24:138, 24:188, 24:222N, 24:228, C*04:01:03	5

101.403.48/12 – including *Taq* pol., IFU-01
101.403.48u/12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **28V**

Lot-specific information

¹HLA-A, HLA-B and HLA-C alleles listed on the IMGT/HLA web page 2013-October-11, release 3.14.0, www.ebi.ac.uk/imgt/hla.

²Alleles that have been deleted from or renamed in the official WHO HLA Nomenclature up to and including the last IMGT/HLA database release can be retrieved from web page <http://hla.alleles.org/alleles/deleted.html>.

³The A*32:01:14, 32:03, 32:38 and the A*74:07 alleles give rise to identical amplification patterns with this lot of the HLA-A low resolution primer set. These alleles can be separated by the respective high resolution primer sets.

101.403.48/12 – including *Taq* pol., IFU-01
 101.403.48u/12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
 “Instructions for Use” (IFU)

Lot No.: **28V**

Lot-specific information
SPECIFICITY TABLE

HLA-A low resolution screening primer set

Specificities and sizes of the PCR products of the 23+1 primer mixes used for HLA-A low resolution screening SSP typing

Primer Mix	Size of spec. PCR product ¹	Size of control band ²	HLA-A serology ³	Amplified HLA-A alleles ^{4,5}
1^{6,7}	120 bp, 145 bp, 225 bp	800 bp	A1, A36	*01:01:01:01-01:04N, 01:06-01:33, 01:35-01:142, 03:18, 03:135, 11:94, 11:112, 36:01-36:05
2⁷	175 bp, 215 bp, 255 bp, 365 bp, 415 bp	800 bp	A2, A203, A210, A19, A28	*02:01:01:01-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:22:02, 02:24:01-02:35:01, 02:35:03-02:47, 02:49-02:77, 02:78 ^w , 02:79:01-02:97:02, 02:99, 02:101:01-02:128, 02:130-02:454, 02:456
3^{8,14}	205 bp, 235 bp	1070 bp	A1, A3, A11, A32, A34, A36	*01:12, 01:19, 01:21, 01:126, 02:338, 03:01:01:01-03:17:02, 03:19-03:74, 03:76-03:94, 03:96-03:134, 03:136-03:175, 11:03, 11:20, 11:25, 11:60, 11:130, 11:158, 24:92, 32:04, 34:02:01-34:04, 34:07-34:10N, 36:02
4	190 bp	800 bp	A1, A2, A3, A11, A26, A30, A36	*01:01:01:01-01:01:22, 01:01:24-01:01:47, 01:01:49-01:04N, 01:06-01:07, 01:09-01:11N, 01:13, 01:16N-01:18N, 01:20-01:29, 01:31N-01:33, 01:35-01:78, 01:80-01:98, 01:100-01:142, 02:78, 02:169, 03:12, 03:18, 03:88, 03:135, 11:01:01-11:27, 11:29-11:52Q, 11:54-11:158, 26:19, 26:72, 30:08, 36:04, 68:13:01, 68:66, 74:19
5⁷	160 bp, 335 bp, 505 bp	1070 bp	A3, A9, A23, A24, A2403, A29, A31, A32	*03:15, 03:19, 03:30, 03:152, 11:139, 23:01:01-23:60, 24:02:01:01-24:11N, 24:13:01-24:15, 24:17-24:64, 24:66-24:210, 24:212-24:252N, 29:07, 29:49, 31:08, 31:29, 32:05, 33:19, 33:53, C*04:01:03
6^{9,11}	135 bp, 200 bp	800 bp	A9, A23, A24, A29, A80	*23:01:01-23:56, 23:58-23:60, 24:24, 24:71, 29:07, 29:49, 31:29, 80:01:01:01-80:03, B*18:27
7	175 bp, 205 bp	1070 bp	A2, A23, A24, A26, A33	*02:17:01 ^w -02:17:02 ^w , 11:139, 23:14:01-23:14:02, 24:02:01:01-24:11N, 24:13:01-24:13:02, 24:17-24:50, 24:54-24:56, 24:58-24:63, 24:66-24:91, 24:93, 24:95-24:113, 24:115-24:137, 24:139-24:187, 24:189-24:210, 24:212-24:221, 24:223-24:227, 24:229-24:252N, 26:16, 33:19, 68:45
8	165 bp, 200 bp	800 bp	A2, A3, A10, A11, A25, A26, A28,A32, A34, A66, A68, A69	*01:51, 02:55, 03:24, 03:50, 11:10, 25:01:01-25:16, 25:18-25:23, 26:01:01-26:06, 26:08-26:15, 26:17-26:18, 26:20-26:43:02, 26:45-26:63, 26:65-26:71N, 26:73-26:88, 26:90-26:91, 26:93-26:94, 29:28, 32:15, 33:51, 34:01:01-34:11, 66:01:01-66:19, 68:01:01-68:106, 69:01-69:02
9^{6,10}	75 bp	800 bp	A3, A25, A32	*25:01:01-25:23, 32:01:01-32:01:13, 32:01:15-32:02, 32:04, 32:06-32:37, 32:39-32:59, B*07:81, B*08:52, B*18:67, B*38:41, B*53:05, B*53:16

101.403.48/12 – including *Taq* pol., IFU-01
 101.403.48u/12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
 “Instructions for Use” (IFU)

Lot No.: **28V**

Lot-specific information

10 ^{6,7,11}	80 bp, 240 bp	800 bp	A10, A26, A43	*01:43, 01:51, 11:17, 11:40, 26:01:01-26:02:02, 26:04, 26:07:01-26:20, 26:22-26:29, 26:31-26:43:02, 26:45-26:77, 26:79-26:91, 26:93-26:94, 33:13, 33:48, 43:01, 68:84
11 ^{6,7}	80 bp, 175 bp, 500 bp	1070 bp	A1, A11, A24, A26, A31, A34, A66	*01:13, 01:28, 01:136, 03:63, 03:88, 11:01:01-11:27, 11:29-11:52Q, 11:54-11:158, 24:19, 24:44, 26:03:01- 26:03:02, 26:06, 26:21, 26:78, 26:92, 34:01:01-34:08, 34:10N-34:11, 66:01:01, 66:04-66:11, 66:13-66:14, 66:17-66:19, 69:02, 80:02
12 ⁶	125 bp, 190 bp	800 bp	A3, A10, A25, A26, A31, A34, A43, A66	*02:309, 02:454, 03:01:19, 11:11, 25:05-25:06, 26:09, 26:54, 26:91, 31:03-31:04, 34:01:01-34:11, 43:01, 66:02-66:03, 66:16
13	175 bp, 225 bp	1070 bp	A1, A2, A3, A10, A25, A26, A34, A43, A66	*01:01:56 ^w , 01:13, 02:34-02:35:03, 02:56:01- 02:56:02, 02:62, 02:103, 02:135, 03:01:01:01- 03:01:22, 03:01:24-03:07, 03:09-03:11N, 03:13- 03:31, 03:33-03:35, 03:37-03:40, 03:42-03:56, 03:58, 03:60-03:71, 03:73-03:87, 03:90-03:106, 03:109- 03:110, 03:112-03:141, 03:143-03:151, 03:153- 03:171, 03:174-03:175, 25:01:01-25:05, 25:07-25:23, 26:01:01-26:01:20, 26:01:22-26:01:29, 26:02:01 ^w , 26:03:01-26:03:02, 26:05-26:08, 26:10-26:28, 26:29 ^w , 26:30-26:33, 26:35-26:43:02, 26:45-26:48, 26:49 ^w , 26:50-26:72, 26:74-26:77, 26:79-26:90, 26:92-26:94, 30:55, 34:08, 43:01, 66:01:01, 66:04-66:09, 66:10 ^w , 66:11-66:15, 66:17-66:19, 68:71, 74:13
14 ^{6,7,11}	80 bp 115 bp, 200 bp, 240 bp 470 bp	1070 bp	A26, A29, A31, A34	*02:237, 02:309, 02:454, 03:95, 26:19, 26:22, 29:01:01:01-29:55, 31:03-31:04, 32:42, 34:04, 66:09
15 ^{6,7,13,14}	90 bp, 135 bp, 205 bp	1070 bp	A1, A30	*01:13, 01:28, 03:43, 03:82, 11:113, 30:01:01- 30:04:02, 30:06-30:20, 30:22-30:78N, 31:35
16	240 bp, 380 bp	1070 bp	A24, A31, A32	*02:237, 03:95, 29:14, 31:01:02-31:79, 32:05, 33:53
17	140 bp, 180 bp	1070 bp	A29, A32, A74	*01:95, 03:43, 03:82, 24:243, 29:13, 29:39, 31:35, 32:01:01-32:03, 32:05-32:59, 74:07
18	200 bp, 390 bp	1070 bp	A,32,A33, A68, A74	*02:243:01-02:243:02, 29:48, 32:15, 33:01:01- 33:01:07, 33:03:01-33:37, 33:39-33:52, 33:54- 33:74N, 68:29, 74:04
19	340 bp, 375 bp	800 bp	A2, A19, A68, A74	*01:121, 02:65, 02:407, 02:449, 68:25, 74:01-74:06, 74:08-74:19
20 ¹¹	210 bp, 240 bp	800 bp	A2, A210, A25, A28, A68	*02:34-02:35:03, 02:46, 02:48, 02:56:01-02:56:02, 02:62, 02:70, 02:78, 02:103, 02:129, 25:05, 26:54, 68:01:01:01-68:106
21	240 bp, 375 bp	800 bp	A2, A26, A28, A32, A66, A68, A69	*02:55, 02:149, 02:243:01-02:243:02, 02:309, 23:57, 24:82, 26:22, 31:41, 32:06, 33:22, 66:06, 66:09, 68:29, 68:105, 69:01-69:02
22 ^{6,12}	85 bp, 240 bp	800 bp	A2, A36	*02:34-02:35:03, 02:46, 02:48, 02:56:01-02:56:02, 02:62, 02:70, 02:78, 02:103, 02:129, 11:155, 31:62, 36:01-36:05
23 ^{6,11}	75 bp, 160 bp,	800 bp	A2, A26, A68, A80	*02:55, 03:41, 03:63, 03:75, 03:88, 11:130, 24:18, 24:204, 24:213, 26:03:01-26:03:02, 26:05-26:06,

101.403.48/12 – including *Taq* pol., IFU-01
 101.403.48u/12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
 “Instructions for Use” (IFU)

Lot No.: **28V**

Lot-specific information

	240 bp, 495 bp	26:21, 26:30, 26:78, 33:24, 36:02, 68:05, 68:15, 68:20, 80:01:01:01-80:01:01:02, 80:03
24¹⁵	- - -	Negative control

¹Alleles are assigned by the presence of specific PCR product(s). However, the sizes of the specific PCR products may be helpful in the interpretation of HLA-A low resolution screening SSP typings.

When the primers in a primer mix can give rise to HLA-specific PCR products of more than one length this is indicated if the size difference is more than 20 base pairs. Size differences of 20 base pairs or less are not given. For high resolution SSP kits, the alleles listed are specified according to amplicon length.

Nonspecific amplifications, i.e. a ladder or a smear of bands, may sometimes be seen. GC-rich primers have a higher tendency of giving rise to nonspecific amplifications than other primers.

PCR fragments longer than the control bands may sometimes be observed. Such bands should be disregarded and do not influence the interpretation of the SSP typings.

PCR fragments migrating faster than the control bands, but slower than a 400 bp fragment may be seen in some gel read-outs. Such bands can be disregarded and do not influence the interpretation of the SSP typings.

Some primers may give rise to primer oligomer artifacts. Sometimes this phenomenon is an inherent feature of the primer pair(s) of a primer mix. More often it is due to other factors such as too low amount of DNA in the PCR reactions, taking too long time in setting up the PCR reactions, working at elevated room temperature or using thermal cyclers that are not pre-heated.

²The internal positive control primer pairs amplify segments of the human growth hormone gene. The internal positive control bands are 1070 or 800 base pairs respectively, well distribution as outlined in the table. Well number 1 contains the shorter, 800 bp, internal positive control band. The well distribution of the internal controls can help in orientation of the kit on gel photo, as well as allow for kit identification. In the presence of a specific amplification the intensity of the control band often decreases.

³The serological reactivity of all HLA-A alleles is not known. In this table we use the expert-assigned serological grouping in Tissue Antigens (2009) 73:95-170 and the serological grouping of the sequence-defined allele.

⁴For several HLA Class I alleles 1st and/or 4th exon(s) and beyond, as well as intron nucleotide sequences, are not available. In these instances it is not known whether some of the primers of the SSP sets are completely matched with the target sequences or not. Assumption is made that unknown sequences in these regions are conserved within allelic groups.

⁵Due to the sharing of sequence motifs between HLA-C alleles, non-HLA-A alleles will be amplified by primer mixes 5, 6 and 9

⁶HLA-specific PCR products shorter than 125 base pairs have a lower intensity and are less sharp than longer PCR products.

⁷The primer pairs in wells 1, 2, 10, 11, 14 and 15 will in many samples give rise to two or three HLA-specific PCR fragments.

⁸Primer mix 3 may faintly amplify the A*30:04:01-30:04:02, 30:06, 30:17 and 30:29 alleles.

⁹Primer mix 6 may give rise to a lower yield of A*23 alleles than the other A low primer mixes.

¹⁰Primer mix 9 may weakly amplify the A*34 alleles.

¹¹Primer mixes 6, 10, 14, 20 and 23 have a tendency to giving rise to primer oligomer formation.

¹²Primer mix 22 might faintly amplify most A*11 alleles.

¹³Primer mix 15 may give rise to a lower yield of HLA-specific PCR product than the other HLA-A low primer mixes.

¹⁴Primer mix 3 and 15 may have tendency of unspecific amplification.

¹⁵Primer mix 24 contains a negative control, which will amplify more than 95% of HLA amplicons as well as the amplicons generated by control primer pairs. PCR product sizes range from 75 to 200 base pairs. The PCR product generated by the control primer pair is 430 base pairs.

'w', might be weakly amplified.

101.403.48/12 – including *Taq* pol., IFU-01
 101.403.48u/12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
 “Instructions for Use” (IFU)

Lot No.: **28V**

Lot-specific information

PRIMER SPECIFICATION

Well No.	1	2	3	4	5	6	7	8	9	10	11	12
Length of spec.	120	175	205	190	160	135	175	165	75	80	80	125
PCR product	145	215	235		335	200	205	200		240	175	190
		225	255		505						500	
		365										
		415										
Length of int. pos. control ¹	800	800	1070	800	1070	800	1070	800	800	800	1070	800
5'-primer(s) ²	98 5'-CTT 3'	48 5'-gCT 3'	363 5'-ATA 3'	98 5'-CTA 3'	144 5'-gCC 3'	176 5'-gCA 3'	98 5'-CTC 3'	98 5'-CTA 3'	261 5'-AAC 3'	98 5'-CTA 3'	301 5'-Cgg 3'	103 5'-CCT 3'
	103 5'-CCT 3'	78 5'-TCT 3'		413 5'-CCg 3'	317 5'-gCT 3'	368 5'-gTT 3'	368 5'-gTT 3'	102 5'-ACA 3'	266 5'-ACg 3'	261 5'-AAC 3'	302 5'-ggA 3'	415 5'-ggT 3'
	123 5'-AgT 3'	106 5'-CCA 3'						413 5'-CCg 3'			385 5'-ggC 3'	423 5'-gCT 3'
	363 5'-ATA 3'											
3'-primer(s) ³	203 5'-TCT 3'	240 5'-ggA 3'	527 5'-CCA 3'	256 5'-CTg 3'	265 5'-CCC 3'	270 5'-ACA 3'	259 5'-gTT 3'	259 5'-gTT 3'	302 5'-ggC 3'	299 5'-TCg 3'	341 5'-CgT 3'	257 5'-gCA 3'
	545 5'-AgA 3'	292 5'-gTg 3'	527 5'-CCT 3'	559 5'-CCg 3'	368 5'-CAA 3'	521 5'-ggg 3'	502 5'-CTT 3'	259 5'-gTT 3'			521 5'-ggg 3'	506 5'-TgT 3'
			527 5'-CCT 3'			534 5'-CgT 3'	539 5'-TCT 3'	538 5'-CCA 3'				559 5'-CTC 3'
			555 5'-CCA 3'									559 5'-CgT 3'
			555 5'-gCA 3'									559 5'-CgC 3'
Well No.	1	2	3	4	5	6	7	8	9	10	11	12

Well No.	13	14	15	16	17	18	19	20	21	22	23
Length of spec.	175	80	90	240	140	200	340	210	240	85	75
PCR product	225	115	135	380	180	390	375	240	375	240	160
		200	205								240
		240									495
		470									
Length of int. pos. control ¹	1070	1070	1070	1070	1070	1070	800	800	800	800	800
5'-primer(s) ²	98 5'-CTT 3'	98 5'-CAC 3'	203 5'-gAA 3'	41 5'-CTT 3'	180 5'-TTT 3'	41 5'-CTT 3'	302 5'-ggA 3'	78 5'-TCT 3'	28 5'-TCg 3'	78 5'-TCT 3'	176 5'-gCA 3'
	423 5'-gCT 3'	219 5'-gCA 3'	362 5'-ggT 3'	355 5'-CCg 3'	203 5'-gAA 3'	98 5'-CAC 3'	341 5'-ggA 3'	106 5'-CCA 3'	261 5'-AAC 3'	527 5'-TgC 3'	261 5'-AAC 3'
		238 5'-AgA 3'	363 5'-ATA 3'		418 5'-AgC 3'			2 nd 5'-CCT 3'	502 5'-CCC 3'		341 5'-ggA 3'
		355 5'-CCg 3'	363 5'-ATA 3'								355 5'-CCC 3'
			363 5'-ATA 3'								
			369 5'-TAC 3'								
3'-primer(s) ³	282 5'-gAC 3'	180 5'-TCA 3'	299 5'-CCA 3'	238 5'-CCT 3'	290 5'-CAA 3'	256 5'-CCC 3'	397 5'-gAg 3'	265 5'-CCC 3'	97 5'-ggT 3'	265 5'-CCC 3'	292 5'-gTg 3'
	282 5'-gAC 3'	257 5'-gCA 3'	411 5'-TCA 3'	238 5'-CCT 3'	317 5'-ggA 3'	256 5'-CTC 3'		282 5'-gAC 3'	355 5'-gAC 3'	282 5'-gAC 3'	292 5'-gTT 3'
	559 5'-CCC 3'	418 5'-gTC 3'	526 5'-CCA 3'	243 5'-TCA 3'	555 5'-CCA 3'	259 5'-gTT 3'		282 5'-gAC 3'	524 5'-CAT 3'	282 5'-gAC 3'	299 5'-TCT 3'
	559 5'-CCg 3'	555 5'-CCA 3'		265 5'-CCC 3'				502 5'-CTT 3'		570 5'-CAC 3'	555 5'-CCA 3'
				555 5'-CCA 3'				506 5'-TgT 3'			
Well No.	13	14	15	16	17	18	19	20	21	22	23

101.403.48/12 – including *Taq* pol., IFU-01
101.403.48u/12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **28V**

Lot-specific information

¹The internal positive control primer pairs amplify segments of the human growth hormone gene. The internal positive control bands are 1070 or 800 base pairs respectively, well distribution as outlined in the table. Well number 1 contains the shorter, 800 bp, internal positive control band. The well distribution of the internal controls can help in orientation of the kit on gel photo, as well as allow for kit identification. In the presence of a specific amplification the intensity of the control band often decreases.

²The nucleotide position matching the specificity-determining 3'-end of the primer is given. Nucleotide numbering as on the www.ebi.ac.uk/imgt/hla web site. The sequence of the 3 terminal nucleotides of the primer is given.

³The nucleotide position matching the specificity-determining 3'-end of the primer is given in the anti-sense direction. Nucleotide numbering as on the www.ebi.ac.uk/imgt/hla web site. The sequence of the 3 terminal nucleotides of the primer is given.

101.403.48/12 – including *Taq* pol., IFU-01
 101.403.48u/12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
 “Instructions for Use” (IFU)

Lot No.: **28V**

Lot-specific information

CELL LINE VALIDATION SHEET																			
HLA-A low resolution screening primer set²																			
				Well															
				1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
			Lot No.:	201321701	201211702	201326503	201211704	201326505	201211706	201211707	201211708	201211709	201211710	201211711	201322012	201211713	201321714	201321715	201211716
	IHWC cell line ¹	A*	A*																
1	9001 SA	*24:02		-	-	-	-	+	-	+	-	-	-	-	-	-	-	-	-
2	9280 LK707	*02:01		-	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
3	9011 E4181324	*01:01		+	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
4	9275 GU373	*30:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	+	-
5	9009 KAS011	*01:01		+	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
6	9353 SM	*02:01	*26:03	-	+	-	-	-	-	-	+	-	-	+	-	+	-	-	-
7	9020 QBL	*26:01		-	-	-	-	-	-	-	+	-	+	-	-	+	-	-	-
8	9025 DEU	*31:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	+
9	9026 YAR	*26:01		-	-	-	-	-	-	-	+	-	+	-	-	+	-	-	-
10	9107 LKT3	*24:02		-	-	-	-	+	-	+	-	-	-	-	-	-	-	-	-
11	9051 PITOUT	*29:02		-	-	-	-	-	-	-	-	-	-	-	-	-	+	-	-
12	9052 DBB	*02:01		-	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
13	9004 JESTHOM	*02:01		-	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
14	9071 OLGA	*31:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	+
15	9075 DKB	*24:02		-	-	-	-	+	-	+	-	-	-	-	-	-	-	-	-
16	9037 SWEIG007	*29:02		-	-	-	-	-	-	-	-	-	-	-	-	-	+	-	-
17	9282 CTM3953540	*03:01	*80:01	-	-	+	-	-	+	-	-	-	-	-	-	+	-	-	-
18	9257 32367	*33:03	*74:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
19	9038 BM16	*02:01		-	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
20	9059 SLE005	*02:01		-	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
21	9064 AMALA	*02:17		-	+	-	-	-	-	W	-	-	-	-	-	-	-	-	-
22	9056 KOSE	*02:01		-	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
23	9124 IHL	*02:01	*34:01	-	+	-	-	-	-	-	+	-	-	+	+	-	-	-	-
24	9035 JBUSH	*32:01		-	-	-	-	-	-	-	-	+	-	-	-	-	-	-	-
25	9049 IBW9	*33:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
26	9285 WT49	*02:05		-	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
27	9191 CH1007	*24:10	*29:01	-	-	-	-	+	-	+	-	-	-	-	-	-	+	-	-
28	9320 BEL5GB	*02:01	*29:02	-	+	-	-	-	-	-	-	-	-	-	-	-	+	-	-
29	9050 MOU	*29:02		-	-	-	-	-	-	-	-	-	-	-	-	-	+	-	-
30	9021 RSH	*30:01	*68:02	-	-	-	-	-	-	-	+	-	-	-	-	-	-	+	-
31	9019 DUCAF	*30:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	+	-
32	9297 HAG	*02:01		-	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
33	9098 MT14B	*31:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	+
34	9104 DHIF	*31:01		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	+
35	9302 SSTO	*32:01		-	-	-	-	-	-	-	-	+	-	-	-	-	-	-	-
36	9024 KT17	*02:06	*11:01	-	+	-	+	-	-	-	-	-	-	+	-	-	-	-	-
37	9065 HHKB	*03:01		-	-	+	-	-	-	-	-	-	-	-	-	+	-	-	-
38	9099 LZL	*02:17		-	+	-	-	-	-	W	-	-	-	-	-	-	-	-	-
39	9315 CML	*01:01	*03:01	+	-	+	+	-	-	-	-	-	-	-	-	+	-	-	-
40	9134 WHONP199	*02:07	*30:01	-	+	-	-	-	-	-	-	-	-	-	-	-	-	+	-
41	9055 H0301	*03:01		-	-	+	-	-	-	-	-	-	-	-	-	+	-	-	-
42	9066 TAB089	*02:07		-	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
43	9076 T7526	*02:06	*02:07	-	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
44	9057 TEM	*66:01		-	-	-	-	-	-	-	+	-	-	+	-	+	-	-	-
45	9239 SHJO	*23:01	*24:02	-	-	-	-	+	+	+	-	-	-	-	-	-	-	-	-
46	9013 SCHU	*03:01		-	-	+	-	-	-	-	-	-	-	-	-	+	-	-	-
47	9045 TUBO	*02:16	*03:01	-	+	+	-	-	-	-	-	-	-	-	-	+	-	-	-
48	9303 TER-ND	*02:01	*11:01	-	+	-	+	-	-	-	-	-	-	+	-	-	-	-	-

101.403.48/12 – including *Taq* pol., IFU-01
 101.403.48u/12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
 “Instructions for Use” (IFU)

Lot No.: **28V**

Lot-specific information

CELL LINE VALIDATION SHEET										
HLA-A low resolution screening primer set ²										
				Well						
				17	18	19	20	21	22	23
				201211717	201321718	201321719	201211720	201321721	201211722	201211723
				Lot No.:						
	IHWC cell line ¹	A*	A*							
1	9001 SA	*24:02		-	-	-	-	-	-	-
2	9280 LK707	*02:01		-	-	-	-	-	-	-
3	9011 E4181324	*01:01		-	-	-	-	-	-	-
4	9275 GU373	*30:01		-	-	-	-	-	-	-
5	9009 KAS011	*01:01		-	-	-	-	-	-	-
6	9353 SM	*02:01	*26:03	-	-	-	-	-	-	+
7	9020 QBL	*26:01		-	-	-	-	-	-	-
8	9025 DEU	*31:01		-	-	-	-	-	-	-
9	9026 YAR	*26:01		-	-	-	-	-	-	-
10	9107 LKT3	*24:02		-	-	-	-	-	-	-
11	9051 PITOUT	*29:02		-	-	-	-	-	-	-
12	9052 DBB	*02:01		-	-	-	-	-	-	-
13	9004 JESTHOM	*02:01		-	-	-	-	-	-	-
14	9071 OLGA	*31:01		-	-	-	-	-	-	-
15	9075 DKB	*24:02		-	-	-	-	-	-	-
16	9037 SWEIG007	*29:02		-	-	-	-	-	-	-
17	9282 CTM3953540	*03:01	*80:01	-	-	-	-	-	-	+
18	9257 32367	*33:03	*74:01	-	+	+	-	-	-	-
19	9038 BM16	*02:01		-	-	-	-	-	-	-
20	9059 SLE005	*02:01		-	-	-	-	-	-	-
21	9064 AMALA	*02:17		-	-	-	-	-	-	-
22	9056 KOSE	*02:01		-	-	-	-	-	-	-
23	9124 IHL	*02:01	*34:01	-	-	-	-	-	-	-
24	9035 JBUSH	*32:01		+	-	-	-	-	-	-
25	9049 IBW9	*33:01		-	+	-	-	-	-	-
26	9285 WT49	*02:05		-	-	-	-	-	-	-
27	9191 CH1007	*24:10	*29:01	-	-	-	-	-	-	-
28	9320 BEL5GB	*02:01	*29:02	-	-	-	-	-	-	-
29	9050 MOU	*29:02		-	-	-	-	-	-	-
30	9021 RSH	*30:01	*68:02	-	-	-	+	-	-	-
31	9019 DUCAF	*30:02		-	-	-	-	-	-	-
32	9297 HAG	*02:01		-	-	-	-	-	-	-
33	9098 MT14B	*31:01		-	-	-	-	-	-	-
34	9104 DHIF	*31:01		-	-	-	-	-	-	-
35	9302 SSTO	*32:01		+	-	-	-	-	-	-
36	9024 KT17	*02:06	*11:01	-	-	-	-	-	-	-
37	9065 HHKB	*03:01		-	-	-	-	-	-	-
38	9099 LZL	*02:17		-	-	-	-	-	-	-
39	9315 CML	*01:01	*03:01	-	-	-	-	-	-	-
40	9134 WHONP199	*02:07	*30:01	-	-	-	-	-	-	-
41	9055 H0301	*03:01		-	-	-	-	-	-	-
42	9066 TAB089	*02:07		-	-	-	-	-	-	-
43	9076 T7526	*02:06	*02:07	-	-	-	-	-	-	-
44	9057 TEM	*66:01		-	-	-	-	-	-	-
45	9239 SHJO	*23:01	*24:02	-	-	-	-	-	-	-
46	9013 SCHU	*03:01		-	-	-	-	-	-	-
47	9045 TUBO	*02:16	*03:01	-	-	-	-	-	-	-
48	9303 TER-ND	*02:01	*11:01	-	-	-	-	-	-	-

¹The provided cell line HLA specificities are retrieved from the <http://www.ihwg.org/hla> web site. The specificity of an individual cell line may thus be subject to change.

101.403.48/12 – including *Taq* pol., IFU-01
101.403.48u/12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: 28V**Lot-specific information**

²The specificity of each primer solution in the kit has been tested against 48 well characterized cell line DNAs and where applicable, additional cell line DNAs.

Additional 5'- and 3'-primers in primer solutions 4, 5, 7, 8, 12, 14 to 17 and 20 to 23 were tested by separately adding one 3'-primer, respectively one 5'-primer. Additional 5'-primers in primer solution 1 and 19 were tested by separately adding one 3'-primer. Additional 3'-primers in primer solutions 3 and 18 were tested by separately adding one 5'-primer.

In primer solutions 2, 11, 12 and 15 one or two 5'-primers were not possible to test, and in primer solutions 3, 6, 8, 12, 13, 16 and 18 one 3'-primer was not possible to test.

101.403.48/12 – including *Taq* pol., IFU-01
101.403.48u/12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **28V**

Lot-specific information

101.403.48/12 – including *Taq* pol., IFU-01
101.403.48u/12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **28V**

Lot-specific information

101.403.48/12 – including *Taq* pol., IFU-01
101.403.48u/12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **28V**

Lot-specific information

ADDRESSES:

Manufacturer:

Olerup SSP AB, Franzengatan 5, SE-112 51 Stockholm, Sweden.

Tel: +46-8-717 88 27

Fax: +46-8-717 88 18

E-mail: info-ssp@olerup.com

Web page: <http://www.olerup-ssp.com>

Distributed by:

Olerup GmbH, Löwengasse 47 / 6, AT-1030 Vienna, Austria.

Tel: +43-1-710 15 00

Fax: +43-1-710 15 00 10

E-mail: support-at@olerup.com

Web page: <http://www.olerup.com>

Olerup Inc., 901 S. Bolmar St., Suite R, West Chester, PA 19382

Tel: 1-877-OLERUP1

Fax: 610-344-7989

E-mail: info.us@olerup.com

Web page: <http://www.olerup.com>

For information on *Olerup* SSP distributors worldwide, contact **Olerup GmbH**.