

101.851-12 – including *Taq* polymerase  
101.851-12u – without *Taq* polymerase

Visit [www.olerup-ssp.com](http://www.olerup-ssp.com) for  
“Instructions for Use” (IFU)

Lot No.: **35V**

Lot-specific Information

[www.olerup-ssp.com](http://www.olerup-ssp.com)

## **Olerup SSP® HLA-B\*51:11N**

<b>Product number:</b>	101.851-12 – including <i>Taq</i> polymerase 101.851-12u - without <i>Taq</i> polymerase
<b>Lot number:</b>	35V
<b>Expiry date:</b>	2016-August-01
<b>Number of tests:</b>	12
<b>Number of wells per test:</b>	2+1
<b>Storage - pre-aliquoted primers:</b>	dark at -20°C
- PCR Master Mix:	-20°C
- Adhesive PCR seals	RT
- Product Insert	RT

**This Product Description is only valid for Lot No. 35V.**

Complete product documentation consists of generic Instructions for Use (IFU), lot specific Product Insert, Worksheet and Certificate.

### **CHANGES COMPARED TO THE PREVIOUS OLERUP SSP® HLA-B\*51:11N LOT (96R)**

A well containing Negative Control primer pairs has been added.

The format of the Product Insert and Worksheet have been changed.

One well has been added to HLA-B\*51:11N, well **3**.

The HLA-B\*51:11N specificity and interpretation tables have been updated compared the previous *Olerup SSP®* HLA-B\*51:11N lot (**Lot No. 96R**).

As of lot series V, the Specificity Table is included in the lot-specific Product Insert, and the Interpretation Table is included in the Worksheet.

The primers of the wells detailed below have been exchanged, added or modified compared to the previous lot.

Well	5'-primer	3'-primer	rationale
3	-	-	Negative Control

101.851-12 – including *Taq* polymerase  
101.851-12u – without *Taq* polymerase

Visit [www.olerup-ssp.com](http://www.olerup-ssp.com) for  
“Instructions for Use” (IFU)

Lot No.: **35V**

Lot-specific Information

[www.olerup-ssp.com](http://www.olerup-ssp.com)

Well **3** contains Negative Control primer pairs, that will amplify more than 95% of the *Olerup SSP®* HLA Class I, DRB, DQB1 and DPB1 amplicons as well as amplicons generated by a control primer pair.

PCR product sizes range from 75 to 430 base pairs.  
The PCR product generated by the control primer pair is 430 base pairs.

Length of PCR product	105	200	105	80	75	80
<b>5'-primer<sup>1</sup></b>	<b>164</b>	<b>340</b>	<b>440</b>	<b>45</b>	<b>45</b>	<b>43</b>
	5'-CAC <sup>3'</sup>	5'-Agg <sup>3'</sup>	5'-TTA <sup>3'</sup>	5'-Tgg <sup>3'</sup>	5'-Tgg <sup>3'</sup>	5'-Tgg <sup>3'</sup>
<b>3'-primer<sup>2</sup></b>	<b>231</b>	<b>2<sup>nd</sup> I</b>	<b>507</b>	<b>59</b>	<b>58</b>	<b>57</b>
	5'-TgC <sup>3'</sup>	5'-AAA <sup>3'</sup>	5'-TTg <sup>3'</sup>	5'-CTC <sup>3'</sup>	5'-ggC <sup>3'</sup>	5'-CTC <sup>3'</sup>
<b>A*</b>	+	+	+			
<b>B*</b>	+	+	+			
<b>C*</b>	+	+	+			
<b>DRB1</b>				+	+	
<b>DRB3</b>				+	+	
<b>DRB5</b>				+		
<b>DQB1</b>					+	
<b>DPB1</b>						+

<sup>1</sup>The nucleotide position for HLA class I genes and the codon for HLA class II genes, in the 2<sup>nd</sup> or 3<sup>rd</sup> exon, matching the specificity-determining 3'-end of the primer is given. Nucleotide and codon numbering as on the [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla) web site. The sequence of the 3 terminal nucleotides of the primer is given.

<sup>2</sup>The nucleotide position for HLA class I genes and the codon for HLA class II genes, in the 2<sup>nd</sup> or 3<sup>rd</sup> exon or the 2<sup>nd</sup> intron, matching the specificity-determining 3'-end of the primer is given in the anti-sense direction. Nucleotide and codon numbering as on the [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla) web site. The sequence of the 3 terminal nucleotides of the primer is given.

101.851-12 – including *Taq* polymerase  
101.851-12u – without *Taq* polymerase

Visit [www.olerup-ssp.com](http://www.olerup-ssp.com) for  
“Instructions for Use” (IFU)

Lot No.: **35V**

Lot-specific Information

[www.olerup-ssp.com](http://www.olerup-ssp.com)

## PRODUCT DESCRIPTION

### HLA-B\*51:11N SSP subtyping

#### CONTENT

The primer set contains 5'- and 3'-primers for identifying the HLA-B\*51:11N allele.

#### PLATE LAYOUT

Each test consists of 3 PCR reactions in an 8 well cut PCR plate. Wells 4 to 8 are empty.

1	2	NC	empty	empty	empty	empty	empty
---	---	----	-------	-------	-------	-------	-------

The 8 well cut PCR plate is marked with the Lot No. '35V' in silver/gray ink.

Well No. 1 is marked with the Lot No. '35V'.

Wells 1 to 2– HLA-B\*51:11N high resolution primers.

Well 3 – Negative Control (NC).

A faint row of numbers is seen between wells 1 and 2 or wells 7 and 8 of the PCR trays. These stem from the manufacture of the trays, and should be disregarded.

The PCR plates are heat-sealed with a PCR-compatible foil.

**Please note:** When removing each 8 well PCR plate, make sure that the remaining plates stay sealed. Use a scalpel or a similar instrument to carefully cut the foil between the plates.

#### INTERPRETATION

Due to the sharing of sequence motifs between HLA-B alleles non-HLA-B\*51:11N alleles will be amplified by primer mix 1.

For further details see Specificity Table.

#### UNIQUELY IDENTIFIED ALLELES

The HLA-B\*51:11N allele will give rise to a unique amplification pattern by the primers in the HLA-B\*51:11N kit<sup>1,2</sup>.

<sup>1</sup>HLA-B alleles listed on the IMGT/HLA web page 2013-October-11, release 3.14.0, [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla).

<sup>2</sup>Alleles that have been deleted from or renamed in the official WHO HLA Nomenclature up to and including the last IMGT/HLA database release can be retrieved from web page <http://hla.alleles.org/alleles/deleted.html>.

101.851-12 – including *Taq* polymerase  
101.851-12u – without *Taq* polymerase

Visit [www.olerup-ssp.com](http://www.olerup-ssp.com) for  
“Instructions for Use” (IFU)

Lot No.: **35V**

Lot-specific Information  
**SPECIFICITY TABLE**

[www.olerup-ssp.com](http://www.olerup-ssp.com)

**HLA-B\*51:11N SSP subtyping**

**Specificities and sizes of the PCR products of the 2+1 primer mixes used for HLA-B\*51:11N SSP subtyping**

Primer Mix	Size of spec. PCR product <sup>1</sup>	Size of control band <sup>2</sup>	Amplified HLA-B alleles
1 <sup>3</sup>	95 bp	800 bp	*07:65 <sup>w</sup> , 07:134 <sup>w</sup> , 08:32, 18:01:01:01-18:11, 18:13-18:15, 18:17N-18:36, 18:38-18:47, 18:49-18:65, 18:67-18:94N, 35:01:01:01-35:08:04, 35:08:06-35:09:03, 35:11:01-35:12:03, 35:14:01-35:15, 35:17:01-35:18, 35:20:01-35:24:02, 35:27, 35:29:01-35:45, 35:48, 35:50-35:62, 35:64:01-35:68:02, 35:70-35:72, 35:74-35:75, 35:76 <sup>w</sup> , 35:77-35:79, 35:81-35:153, 35:155-35:185, 35:187-35:190, 35:192-35:197, 35:199-35:206, 35:208-35:209, 35:211-35:216N, 35:218-35:225, 35:227-35:240, 37:08, 38:06-38:07, 39:19:01-39:19:02, 44:06, 51:01:01-51:24:05, 51:26-51:46, 51:48-51:103, 51:105-51:111, 51:113-51:117, 51:119-51:146, 51:148-51:154, 53:01:01-53:16, 53:18-53:27, 53:29, 53:31, 56:06 <sup>w</sup> , 78:01:01-78:04, 78:07
2	495 bp	1070 bp	*51:11N
3 <sup>4</sup>			<b>Negative Control</b>

<sup>1</sup>Alleles are assigned by the presence of specific PCR product(s). However, the sizes of the specific PCR products may be helpful in the interpretation of HLA-B\*51:11N SSP typings.

Nonspecific amplifications, i.e. a ladder or a smear of bands, may sometimes be seen. GC-rich primers have a higher tendency of giving rise to nonspecific amplifications than other primers.

PCR fragments longer than the control bands may sometimes be observed. Such bands should be disregarded and do not influence the interpretation of the SSP typings.

PCR fragments migrating faster than the control bands, but slower than a 400 bp fragment may be seen in some gel read-outs. Such bands can be disregarded and do not influence the interpretation of the SSP typings.

Some primers may give rise to primer oligomer artifacts. Sometimes this phenomenon is an inherent feature of the primer pair(s) of a primer mix. More often it is due to other factors such as too low amount of DNA in the PCR reactions, taking too long time in setting up the PCR reactions, working at elevated room temperature or using thermal cyclers that are not pre-heated.

<sup>2</sup>The internal positive control primer pairs amplify segments of the human growth hormone gene. The internal positive control bands are 1070 or 800 base pairs respectively, well distribution as outlined in the table. Well number 1 contains the shorter, 800 bp, internal positive control band. The well distribution of the internal controls can help in orientation of the kit on gel photo, as well as allow for kit identification. In the presence of a specific amplification the intensity of the control band often decreases.

<sup>3</sup>HLA-specific PCR products shorter than 125 base pairs have a lower intensity and are less sharp than longer PCR products.

<sup>4</sup>Primer mix 3 contains a negative control, which will amplify more than 95% of HLA amplicons as well as the amplicons generated by control primer pairs. PCR product sizes range from 75 to 200 base pairs. The PCR product generated by the control primer pair is 430 base pairs.

'w', might be weakly amplified.

101.851-12 – including *Taq* polymerase  
101.851-12u – without *Taq* polymerase

Visit [www.olerup-ssp.com](http://www.olerup-ssp.com) for  
“Instructions for Use” (IFU)

Lot No.: **35V**

Lot-specific Information

[www.olerup-ssp.com](http://www.olerup-ssp.com)

## PRIMER SPECIFICATION

Well No.	1	2
Length of spec.	95	495
PCR product		
Length of int.	800	1070
pos. control <sup>1</sup>		
5'-primer(s) <sup>2</sup>	206 5' -gAC 3'	3 <sup>rd</sup> I 5' -CTT 3'
3'-primer(s) <sup>3</sup>	259 5' -gTT 3'	621 5' -ggg 3'
Well No.	1	2

<sup>1</sup>The internal positive control primer pairs amplify segments of the human growth hormone gene. The internal positive control bands are 1070 or 800 base pairs respectively, well distribution as outlined in the table. Well number 1 contains the shorter, 800 bp, internal positive control band. The well distribution of the internal controls can help in orientation of the kit on gel photo, as well as allow for kit identification. In the presence of a specific amplification the intensity of the control band often decreases.

<sup>2</sup>The nucleotide position matching the specificity-determining 3'-end of the primer is given. Nucleotide numbering as on the [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla) web site. The sequence of the 3 terminal nucleotides of the primer is given.

<sup>3</sup>The nucleotide position matching the specificity-determining 3'-end of the primer is given in the anti-sense direction. Nucleotide numbering as on the [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla) web site. The sequence of the 3 terminal nucleotides of the primer is given.

101.851-12 – including *Taq* polymerase  
101.851-12u – without *Taq* polymerase

Visit [www.olerup-ssp.com](http://www.olerup-ssp.com) for  
“Instructions for Use” (IFU)

Lot No.: **35V**

Lot-specific Information

[www.olerup-ssp.com](http://www.olerup-ssp.com)

## CELL LINE VALIDATION SHEET

### HLA-B\*51:11N SSP kit<sup>2</sup>

					Well	
					1	2
				Lot No.:	201189901	201189902
	IHWC cell line <sup>1</sup>	HLA-B				
1	9001 SA	*07:02			-	-
2	9280 LK707	*52:01	*73:01		-	-
3	9011 E4181324	*52:01			-	-
4	9275 GU373	*15:10	*53:01		+	-
5	9009 KAS011	*37:01			-	-
6	9353 SM	*39:01	*51:01		+	-
7	9020 QBL	*18:01			+	-
8	9025 DEU	*35:01			+	-
9	9026 YAR	*38:01			-	-
10	9107 LKT3	*54:01			-	-
11	9051 PITOUT	*44:03			-	-
12	9052 DBB	*57:01			-	-
13	9004 JESTHOM	*27:05			-	-
14	9071 OLGA	*15:01	*15:20		-	-
15	9075 DKB	*40:01			-	-
16	9037 SWEIG007	*40:02			-	-
17	9282 CTM3953540	*08:01	*55:01		-	-
18	9257 32367	*14:01	*56:01		-	-
19	9038 BM16	*18:01			+	-
20	9059 SLE005	*40:01			-	-
21	9064 AMALA	*15:01			-	-
22	9056 KOSE	*35:03			+	-
23	9124 IHL	*40:02	*56:02		-	-
24	9035 JBUSH	*38:01			-	-
25	9049 IBW9	*14:02			-	-
26	9285 WT49	*58:01			-	-
27	9191 CH1007	*07:05	*51:01		+	-
28	9320 BEL5GB	*44:02	*44:03		-	-
29	9050 MOU	*44:03			-	-
30	9021 RSH	*42:01			-	-
31	9019 DUCAF	*18:01			+	-
32	9297 HAG	*41:02			-	-
33	9098 MT14B	*40:01			-	-
34	9104 DHIF	*38:01			-	-
35	9302 SСТО	*44:02			-	-
36	9024 KT17	*15:01	*35:01		+	-
37	9065 HHKB	*07:02			-	-
38	9099 LZL	*15:01			-	-
39	9315 CML	*08:01	*27:05		-	-
40	9134 WHONP199	*13:02	*46:01		-	-
41	9055 H0301	*14:02			-	-
42	9066 TAB089	*46:01			-	-
43	9076 T7526	*46:01			-	-
44	9057 TEM	*38:01			-	-
45	9239 SHJO	*42:01	*50:01		-	-
46	9013 SCHU	*07:02			-	-
47	9045 TUBO	*51:01			+	-
48	9303 TER-ND	*35:01	*44:03		+	-

**101.851-12 – including *Taq* polymerase**  
**101.851-12u – without *Taq* polymerase**

Visit [www.olerup-ssp.com](http://www.olerup-ssp.com) for  
**“Instructions for Use” (IFU)**

**Lot No.: 35V**

**Lot-specific Information**

**[www.olerup-ssp.com](http://www.olerup-ssp.com)**

<sup>1</sup>The provided cell line HLA specificities are retrieved from the <http://www.ihwg.org/hla> web site. The specificity of an individual cell line may thus be subject to change.

<sup>2</sup>The specificity of each primer solution in the kit has been tested against 48 well characterized cell line DNAs and where applicable, additional cell line DNAs.

101.851-12 – including *Taq* polymerase  
101.851-12u – without *Taq* polymerase

Visit [www.olerup-ssp.com](http://www.olerup-ssp.com) for  
“Instructions for Use” (IFU)

Lot No.: **35V**

Lot-specific Information

[www.olerup-ssp.com](http://www.olerup-ssp.com)

**ADDRESSES:**

**Manufacturer:**

**Olerup SSP AB**, Franzengatan 5, SE-112 51 Stockholm, Sweden.

**Tel:** +46-8-717 88 27

**Fax:** +46-8-717 88 18

**E-mail:** [info-ssp@olerup.com](mailto:info-ssp@olerup.com)

**Web page:** <http://www.olerup-ssp.com>

**Distributed by:**

**Olerup GmbH**, Löwengasse 47 / 6, AT-1030 Vienna, Austria.

**Tel:** +43-1-710 15 00

**Fax:** +43-1-710 15 00 10

**E-mail:** [support-at@olerup.com](mailto:support-at@olerup.com)

**Web page:** <http://www.olerup.com>

**Olerup Inc.**, 901 S. Bolmar St., Suite R, West Chester, PA 19382

**Tel:** 1-877-OLERUP1

**Fax:** 610-344-7989

**E-mail:** [info.us@olerup.com](mailto:info.us@olerup.com)

**Web page:** <http://www.olerup.com>

For information on *Olerup* SSP distributors worldwide, contact **Olerup GmbH**.