

Lot No.: **50G**

Lot-specific information

## **Olerup SSP<sup>®</sup> HLA-Cw\*14**

**Product number:** 101.625-06u – without *Taq* polymerase  
**Lot number:** 50G  
**Expiry date:** 2011-November-01  
**Number of tests:** 6  
**Number of wells per test:** 16  
**Storage - pre-aliquoted primers:** dark at -20°C  
- PCR Master Mix: -20°C  
- Adhesive PCR seals RT  
- Product Insert RT

**This Product Description is only valid for Lot No. 50G.**

### **CHANGES COMPARED TO THE PREVIOUS *OLERUP SSP*<sup>®</sup> HLA-Cw\*14 LOT**

The HLA-Cw\*14 specificity and interpretation tables have been updated for the HLA-Cw alleles described since the previous *Olerup SSP*<sup>®</sup> HLA-Cw\*14 lot was made (**Lot No. 32F**).

Two wells have been added to the HLA-Cw\*14 kit,  
wells **15 and 16**.

Primers of the wells detailed below have been exchanged, added or modified compared to the previous lot.

Well	5'-primer	3'-primer	rationale
4	-	Modified	Improved specificity of amplification.
5	Removed	-	One 5'-primer removed for increased specificity of primer mix.
13	-	Added	Primer added for the Cw*1414 allele, exchanged positive control primer pair.
15	New	New	New primer pair for the Cw*1412 allele.
16	New	New	New primer pair for the Cw*1413 allele.

Lot No.: **50G**

Lot-specific information

Lot No.: **50G**

Lot-specific information

## PRODUCT DESCRIPTION

### HLA-Cw\*14 SSP typing

#### CONTENT

The primer set contains 5'- and 3'-primers for identifying the Cw\*1402 to Cw\*1414 alleles.

#### PLATE LAYOUT

Each HLA-Cw\*14 test consists of 16 PCR reactions in a 16 well cut PCR plate.

1	2	3	4	5	6	7	8
9	10	11	12	13	14	15	16

The 16 well cut PCR plate is marked with 'Cw\*14' in silver/gray ink .

Well No. 1 is marked with the Lot No. '50G'.

A faint row of numbers is seen between wells 1 and 2 or wells 7 and 8 of the PCR trays. These stem from the manufacture of the trays, and should be disregarded.

The PCR plates are heat-sealed with a PCR-compatible foil.

**Please note:** When removing each 16 well PCR plate, make sure that the remaining plates stay sealed. Use a scalpel or a similar instrument to carefully cut the foil between the plates.

#### INTERPRETATION

The interpretation of HLA-Cw\*14 SSP subtypings will be influenced by other HLA-Cw alleles, as in particular primer mixes 2 to 4, 6 to 8, 15 and 16 amplify many non-HLA-Cw\*14 alleles. In addition, primer mix 4 will amplify the B\*350802 and B\*6702 alleles, primer mix 7 will amplify the B\*0713 and B\*0715 alleles and primer mix 8 will amplify the B\*5530 allele.

#### UNIQUELY IDENTIFIED ALLELES

All the HLA-Cw\*14 alleles, i.e. **Cw\*1402 to Cw\*1414**, recognized by the HLA Nomenclature Committee in October 2009<sup>1</sup> will be amplified by the primers in the HLA-Cw\*14 SSP kit.

The HLA-Cw\*14 subtyping kit cannot distinguish the Cw\*140201 and Cw\*140204 alleles.

<sup>1</sup>HLA-Cw alleles listed on the IMGT/HLA web page 2009-October-19, release 2.27.0, [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla).

Lot No.: **50G**

Lot-specific information

### RESOLUTION IN HOMO- AND HETEROZYGOTES

The thirteen phenotypically different HLA-Cw\*14 alleles give rise to 15 amplification patterns that can be combined in 120 homozygous and heterozygous combinations. Sixty-four of these genotypes do not give rise to unique amplification patterns. (The different product sizes generated by primer mix 13 were not considered in these calculations.)

+++++--+	+----+---	1405,1410 = 1409,1410
+++++--+	+-----	1403,1405 = 1403,1409
++-+++-	+-----	140203,1405 = 140203,1409
++-+++-	+----+---	1402,1410 = 140202,1410 = 1403,1414 = 1410,1414
++-+++-	+-----	1402,1403 = 140202,1403
++-+++-	+---+---	1402,1408 = 140202,1408
++-+++-	+-----	1402,140203 = 140202,140203
++-+-+--	+---+---	1405,1408 = 1408,1409
+--+++-	+-----	1404,1405 = 1404,1409
+--+++-	+-----+	1405,1412 = 1409,1412
+--+++-	+-----	1402,1404 = 140202,1404 = 1404,1404
+--+++-	+-----+	1402,1412 = 140202,1412 = 1412,1412
+---++-	+---+---	1405,1407N = 1407N,1409
+---++-	+----+---	1405,1414 = 1409,1414
+---++-	+-----+	1405,1411 = 1409,1411
+---++-	+-----+	1405,1413 = 1409,1413
+---++-	+-----	1402,1405 = 1402,1409 = 140202,1405
+---+---	++-----	1402,1406 = 140202,1406
+---+---	+---+---	1402,1407N = 140202,1407N = 1407N,1407N
+---+---	+----+---	1402,1414 = 140202,1414 = 1414,1414
+---+---	+-----+	1402,1411 = 140202,1411 = 1411,1411
+---+---	+-----+	1402,1413 = 140202,1413 = 1413,1413
+---+---	+-----	1402,1402 = 1402,140202
+---++-	++-----	1405,1406 = 1406,1409
+---++-	+-----	1405,1405 = 1405,1409
-++-+-	+----+---	140203,1410 = 1403,1410 = 1410,1410
-++-+-	+-----	140203,1403 = 1403,1403

1402 = 140201 and 140204

Lot No.: **50G**

Lot-specific information

## SPECIFICITY TABLE

### HLA-Cw\*14 SSP subtyping

Specificities and sizes of the PCR products of the 16 primer mixes used for HLA-Cw\*14 SSP subtyping

Primer Mix	Size of spec. PCR product <sup>1</sup>	Size of control band <sup>2</sup>	Amplified HLA-Cw*14 alleles	Other amplified HLA Class I alleles <sup>3</sup>
<b>1</b>	145 bp	<b>800 bp</b>	*140201, 140202, 140204, 1404-1407N, 1409, 1411-1414	*0764
<b>2<sup>5</sup></b>	145 bp	1070 bp	*140203, 1403, 1408, 1410	*04010101-040112, 040401-0405, 0407-041502, 0417-0420, 0423-0425, 0426 <sup>w</sup> , 0427-0441, 0443-0448
<b>3</b>	245 bp	<b>800 bp</b>	*1404, 1412	*04010101-040112, 040401-0405, 0407-0410, 0412-041502, 0417-0420, 0423-0425, 0426 <sup>w</sup> , 0427, 0428, 0430-0435, 0437-0441, 0443-0448
<b>4</b>	140 bp	1070 bp	*140203, 1403, 1408, 1410	*0121, 0212 <sup>w</sup> , 0227, 0411, 0429, 0436, 070209, 080101-0809, 0811-0829, 120201-120303, 120305-120308, 1206-1208, 1210-1220, 1222-1226, 1228, 1507, 1521 <sup>w</sup> , 160101, 160103, 160401, 1606-1608, 1610, 1611, 1613-1617, <b>B*350802, B*6702</b>
<b>5</b>	210 bp	1070 bp	*140201-1404, 1407N, 1410-1414	
<b>6<sup>4</sup></b>	130 bp	1070 bp	*1405, 1409	*030201-030414, 0306-0313, 0315-0324, 0326, 0328-034301, 0344-0357, 0359-0376, 041501, 041502, 0417
<b>7<sup>4</sup></b>	130 bp	1070 bp	*1404	*0117, 0123, 0315, 0327, 033801, 033802, 0369, *06020101, 06020102,

Lot No.: **50G**

Lot-specific information

				060203-0604, 0606-0628, *070101-0720, 0722- 0733N, *0735-0738, *0741- 0768, 0770, 0771, 0773- 0791, 120304, 120401, 17010101-1705, 1801- 1803, <b>B*0713, B*0715</b>
<b>8</b>	210 bp	1070 bp	*1403, 1410	*0212 <sup>w</sup> , 0227, 030201- 0306, 0308, 0309, 0310 <sup>w</sup> , *0312-0314, 0316, 0317, 0319-0328, 0329 <sup>w</sup> , 0330- 0336, 033801, 033802, 0340-0344, 0346-0349, 0351-0376, 0720, 1507, 1521 <sup>w</sup> , <b>B*5530</b>
<b>9</b>	140 bp	1070 bp	*140201, 140203-1408, 1410-1414	*0317, 0371, 0437
<b>10</b>	210 bp	1070 bp	*1406	
<b>11</b>	205 bp	1070 bp	*1407N	
<b>12</b>	225 bp	1070 bp	*1408	
<b>13<sup>4,6</sup></b>	125 bp, 285 bp	1070 bp	*1410, 1414	
<b>14</b>	290 bp	1070 bp	*1411	
<b>15</b>	150 bp	<b>800 bp</b>	*1412	*0114, 0307, 0310, 0315, 0329, 0345, 06020101, 06020102, 060203-0604, 0606-0610, 0612-0628, 0707, 0709, 0776, 120401, 17010101-1705, 1801-1803
<b>16</b>	410 bp	1070 bp	*1413	*0315, 0332, 0345, 0360, 04010101-040104, 040106, 040108, 040110-040112, 0403, 040401, 0405-0410, 0412-0420, 0423-0432, 0434-0448, 1801-1803

**Lot No.: 50G**

**Lot-specific information**

<sup>1</sup>Alleles are assigned by the presence of specific PCR product(s). However, the sizes of the specific PCR products may be helpful in the interpretation of HLA-Cw\*14 resolution SSP typings. When the primers in a primer mix can give rise to specific PCR products of more than one length this is indicated if the size difference is 20 base pairs or more. Size differences shorter than 20 base pairs are not given. For high resolution SSP kits the respective length of the specific PCR product(s) of the alleles amplified by these primer mixes is given.

Nonspecific amplifications, i.e. a ladder or a smear of bands, may sometimes be seen. GC-rich primers have a higher tendency of giving rise to nonspecific amplifications than other primers.

PCR fragments longer than the control bands may sometimes be observed. Such bands should be disregarded and do not influence the interpretation of the SSP typings.

PCR fragments migrating faster than the control bands, but slower than a 400 bp fragment may be seen in some gel read-outs. Such bands can be disregarded and do not influence the interpretation of the SSP typings.

Some primers may give rise to primer oligomer artifacts. Sometimes this phenomenon is an inherent feature of the primer pair(s) of a primer mix. More often it is due to other factors such as too low amount of DNA in the PCR reactions, taking too long time in setting up the PCR reactions, working at elevated room temperature or using thermal cyclers that are not pre-heated.

<sup>2</sup>The internal positive control primer pairs amplify segments of the human growth hormone gene. The two different control primer pairs give rise to either an internal positive control band of 1070 base pairs, for most wells, or a band of 800 base pairs, for some wells.

Well number 1 contains the primer pair giving rise to the shorter, 800 bp, internal positive control band in order to help in the correct orientation of the HLA-Cw\*14 SSP subtyping.

In addition, wells number 3 and 15 contain the primer pair giving rise to the shorter, 800 bp, internal positive control band in order to allow kit identification.

In the presence of a specific amplification the intensity of the control band often decreases.

<sup>3</sup>Due to the sharing of sequence motifs between HLA-Cw alleles non-HLA-Cw\*14 alleles will be amplified by primer mixes 1 to 4, 6 to 9, 15 and 16. In addition, primer mix 4 will amplify the B\*350802 and B\*6702 alleles, primer mix 7 will amplify the B\*0713 and B\*0715 alleles and primer mix 8 will amplify the B\*5530 allele.

<sup>4</sup>Specific PCR fragments shorter than 130 base pairs have a lower intensity and are less sharp than longer PCR bands.

<sup>5</sup>Primer mix 2 has a tendency of giving a lower yield of specific PCR product than the other HLA-Cw\*14 primer mixes.

<sup>6</sup>Primer mix 13: specific PCR product of 125 bp in the Cw\*1414 allele. Specific PCR product of 285 bp in the Cw\*1410 allele.

'w', may be weakly amplified.

Lot No.: **50G**

Lot-specific information

<b>INTERPRETATION TABLE</b>								
<b>HLA-Cw*14 SSP subtyping</b>								
<b>Amplification patterns of the HLA-Cw*1402 to 1414 alleles</b>								
	<b>Well<sup>5</sup></b>							
	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>
<b>Length of spec.</b>	<b>145</b>	<b>145</b>	<b>245</b>	<b>140</b>	<b>210</b>	<b>130</b>	<b>130</b>	<b>210</b>
<b>PCR product</b>								
<b>Length of int.</b>	<b>800</b>	<b>1070</b>	<b>800</b>	<b>1070</b>	<b>1070</b>	<b>1070</b>	<b>1070</b>	<b>1070</b>
<b>pos. control<sup>1</sup></b>								
<b>5'-primer<sup>2</sup></b>	<b>98</b>	<b>98</b>	<b>98</b>	<b>201</b>	<b>368</b>	<b>368</b>	<b>201</b>	<b>134</b>
	5' -CTC 3'5' -CTC 3'5' -CTC 3'5' -CCA 3'5' -gTT 3'5' -gTA 3'5' -CCg 3'5' -CCA 3'							
<b>3'-primer(s)<sup>3</sup></b>	<b>201</b>	<b>201</b>	<b>302</b>	<b>302</b>	<b>538</b>	<b>459</b>	<b>289</b>	<b>302</b>
	5' -CTC 3'5' -CTT 3'5' -ggT 3'5' -ggC 3'5' -CCg 3'5' -AgA 3'5' -AgC 3'5' -ggC 3'							
							<b>289</b>	
							5' -AgC 3'	
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>
<b>HLA-Cw allele<sup>4</sup></b>								
<b>*140201, 140204</b>	<b>1</b>				<b>5</b>			
<b>*140202</b>	<b>1</b>				<b>5</b>			
<b>*140203</b>		<b>2</b>		<b>4</b>	<b>5</b>			
<b>*1403</b>		<b>2</b>		<b>4</b>	<b>5</b>			<b>8</b>
<b>*1404</b>	<b>1</b>		<b>3</b>		<b>5</b>		<b>7</b>	
<b>*1405</b>	<b>1</b>					<b>6</b>		
<b>*1406</b>	<b>1</b>							
<b>*1407N</b>	<b>1</b>				<b>5</b>			
<b>*1408</b>		<b>2</b>		<b>4</b>				
<b>*1409</b>	<b>1</b>					<b>6</b>		
<b>*1410</b>		<b>2</b>		<b>4</b>	<b>5</b>			<b>8</b>
<b>*1411</b>	<b>1</b>				<b>5</b>			
<b>*1412</b>	<b>1</b>		<b>3</b>		<b>5</b>			
<b>*1413</b>	<b>1</b>				<b>5</b>			
<b>*1414</b>	<b>1</b>				<b>5</b>			
<b>*0114</b>								
<b>*0117, 0123, 0611, 070101-070207, 070210- 0706, 0708, 0710-0719, 0722-0733N, 0735-0738, 0741-0763, 0765-0768, 0770, 0771, 0773-0775, 0777-0791, 120304, B*0713, B*0715</b>							<b>7</b>	
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>



Lot No.: **50G**

Lot-specific information

INTERPRETATION TABLE								
HLA-Cw*14 SSP subtyping								
Amplification patterns of the HLA-Cw*1402 to 1414 alleles								
Well <sup>5</sup>								
9	10	11	12	13	14	15	16	
140	210	205	225	125	290	150	410	Length of spec. PCR product
				285				
1070	1070	1070	1070	1070	1070	<b>800</b>	1070	Length of int. pos. control <sup>1</sup>
361	368	419	356	98	368	201	341	5'-primer <sup>2</sup>
5' -AgT 3' 5' -gTT 3' 5' -gTC 3' 5' -CCC 3' 5' -CTC 3' 5' -gTT 3' 5' -CCg 3' 5' -ggA 3'								
459	539	585	538	180	619	312	459	3'-primer(s) <sup>3</sup>
5' -AgA 3' 5' -TCA 3' 5' -AgT 3' 5' -CCg 3' 5' -TCC 3' 5' -TTT 3' 5' -AgT 3' 5' -AgA 3'								
				343				
				5' -ACT 3'				
9	10	11	12	13	14	15	16	Well No.
								HLA-Cw allele <sup>4</sup>
9								*140201, 140204
								*140202
9								*140203
9								*1403
9								*1404
9								*1405
9	10							*1406
9		11						*1407N
9			12					*1408
								*1409
9				13				*1410
9					14			*1411
9						15		*1412
9							16	*1413
9				13				*1414
						15		*0114
								*0117, 0123, 0611, 070101-070207, 070210- 0706, 0708, 0710-0719, 0722-0733N, 0735-0738, 0741-0763, 0765-0768, 0770, 0771, 0773-0775, 0777-0791, 120304, B*0713, B*0715
9	10	11	12	13	14	15	16	Well No.

Lot No.: **50G**

Lot-specific information

Length of spec.	145	145	245	140	210	130	130	210
PCR product								
Well No.	1	2	3	4	5	6	7	8
*0121, 070209, 080101-0809, 0811-0829, 120201-120303, 120305-120308, 1206-1208, 1210-1220, 1222-1226, 1228, 160101, 160103, 160401, 1606-1608, 1610, 1611, 1613-1617, <i>B*350802</i> , <i>B*6702</i>				4				
*0212, 1521				w				w
*0227, 1507				4				8
*030201-030414, 0306, 0308, 0309, 0312, 0313, 0316, 0319-0324, 0326, 0328, 0330, 0331, 0333-0336, 0340-034301, 0344, 0346-0349, 0351-0357, 0359, 0361-0363, 0365-0368, 0370, 0372-0376						6		8
*0305, 0314, 0325, 034302, 0358, <i>B*5530</i>								8
*0307						6		
*0310, 0329						6		w
*031101, 031102, 0318, 0337, 0339, 0350, 0364						6		
*0315						6	7	
*0317, 0371						6		8
*0327, 0720							7	8
*0332, 0360						6		8
*033801, 033802, 0369						6	7	8
*0345						6		
*04010101-040104, 040106, 040108, 040110-040112, 040401, 0405, 0407-0410, 0412-0414, 0418-0420, 0423-0425, 0427, 0428, 0430-0432, 0434, 0435, 0438-0441, 0443-0448		2	3					
*040105, 040107, 040109, 040402, 0433		2	3					
Well No.	1	2	3	4	5	6	7	8

Lot No.: **50G**

Lot-specific information

140	210	205	225	125	290	150	410	Length of spec. PCR product
9	10	11	12	13	14	15	16	Well No.
				285				
								*0121, 070209, 080101-0809, 0811-0829, 120201-120303, 120305-120308, 1206-1208, 1210-1220, 1222-1226, 1228, 160101, 160103, 160401, 1606-1608, 1610, 1611, 1613-1617, <i>B*350802, B*6702</i>
								*0212, 1521
								*0227, 1507
								*030201-030414, 0306, 0308, 0309, 0312, 0313, 0316, 0319-0324, 0326, 0328, 0330, 0331, 0333-0336, 0340-034301, 0344, 0346-0349, 0351-0357, 0359, 0361-0363, 0365-0368, 0370, 0372-0376
								*0305, 0314, 0325, 034302, 0358, <i>B*5530</i>
						15		*0307
						15		*0310, 0329
								*031101, 031102, 0318, 0337, 0339, 0350, 0364
						15	16	*0315
9								*0317, 0371
								*0327, 0720
							16	*0332, 0360
								*033801, 033802, 0369
						15	16	*0345
							16	*04010101-040104, 040106, 040108, 040110-040112, 040401, 0405, 0407-0410, 0412-0414, 0418-0420, 0423-0425, 0427, 0428, 0430-0432, 0434, 0435, 0438-0441, 0443-0448
								*040105, 040107, 040109, 040402, 0433
9	10	11	12	13	14	15	16	Well No.

Lot No.: **50G**

Lot-specific information

Length of spec.	145	145	245	140	210	130	130	210
PCR product								
Well No.	1	2	3	4	5	6	7	8
*0403, 0406, 0416, 0442								
*0411		2		4				
*041501, 041502, 0417		2	3			6		
*0426		w	w					
*0429, 0436		2		4				
*0437		2	3					
*06020101, 06020102, 060203-0604, 0606-0610, 0612-0628, 0707, 0709, 0776, 120401, 17010101- 1705							7	
*0764	1						7	
*1801-1803							7	
Well No.	1	2	3	4	5	6	7	8

Lot No.: **50G**

Lot-specific information

140	210	205	225	125	290	150	410	Length of spec. PCR product
				285				
9	10	11	12	13	14	15	16	Well No.
							16	*0403, 0406, 0416, 0442
								*0411
							16	*041501, 041502, 0417
							16	*0426
							16	*0429, 0436
9							16	*0437
						15		*06020101, 06020102, 060203-0604, 0606-0610, 0612-0628, 0707, 0709, 0776, 120401, 17010101- 1705
								*0764
						15	16	*1801-1803
9	10	11	12	13	14	15	16	Well No.

<sup>1</sup>The internal positive control primer pairs amplify segments of the human growth hormone gene. The two different control primer pairs give rise to either an internal positive control band of 1070 base pairs, for most wells, or a band of 800 base pairs, for some wells.

Well number 1 contains the primer pair giving rise to the shorter, 800 bp, internal positive control band in order to help in the correct orientation of the HLA-Cw\*14 SSP subtyping.

In addition, wells number 3 and 15 contain the primer pair giving rise to the shorter, 800 bp, internal positive control band in order to allow kit identification.

<sup>2</sup>The nucleotide position, in the 2<sup>nd</sup> or 3<sup>rd</sup> exon, matching the specificity-determining 3'-end of the primer is given. Nucleotide numbering as on the [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla) web site. The sequence of the 3 terminal nucleotides of the primer is given.

<sup>3</sup>The nucleotide position, in the 2<sup>nd</sup> or 3<sup>rd</sup> exon, matching the specificity-determining 3'-end of the primer is given in the anti-sense direction. Nucleotide numbering as on the [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla) web site. The sequence of the 3 terminal nucleotides of the primer is given.

<sup>4</sup>The HLA-Cw\*1401 nucleotide sequence has been shown to be identical to Cw\*1402.

<sup>5</sup>Primer mix 13: specific PCR product of 125 bp in the Cw\*1414 allele. Specific PCR product of 285 bp in the Cw\*1410 allele.

'w', may be weakly amplified.

Lot No.: **50G**

Lot-specific information

<b>CELL LINE VALIDATION SHEET</b>																				
<b>HLA-Cw*14 SSP primer set</b>																				
				Well																
				1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	
				Prod. No.:	200853501	200965802	200853503	200965804	200965805	200853506	200853507	200853508	200853509	200853510	200853511	200853512	200965813	200853514	200965815	200965816
IHCW cell line			Cw*																	
1	9001	SA	*0702		-	-	-	-	-	-	+	-	-	-	-	-	-	-	-	-
2	9280	LK707	*0701	*1505	-	-	-	-	-	-	+	-	-	-	-	-	-	-	-	-
3	9011	E4181324	*1202		-	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
4	9275	GU373	*0304	*0401	-	+	+	-	-	+	-	+	-	-	-	-	-	-	-	+
5	9009	KAS011	*0602		-	-	-	-	-	-	+	-	-	-	-	-	-	-	-	+
6	9353	SM	*0304	*0702	-	-	-	-	-	+	+	+	-	-	-	-	-	-	-	-
7	9020	QBL	*0501		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
8	9025	DEU	*0401		-	+	+	-	-	-	+	-	-	-	-	-	-	-	-	+
9	9026	YAR	*1203		-	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
10	9107	LKT3	*0102		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
11	9051	PITOUT	*1601		-	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
12	9052	DBB	*0602		-	-	-	-	-	-	+	-	-	-	-	-	-	-	+	-
13	9004	JESTHOM	*0102		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
14	9071	OLGA	*0102	*0304	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
15	9075	DKB	*0304		-	-	-	-	-	+	-	+	-	-	-	-	-	-	-	-
16	9037	SWEIG007	*0202		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
17	9282	CTM3953540	*0303	*0701	-	-	-	-	-	+	+	+	-	-	-	-	-	-	-	-
18	9257	32367	*0102	*0705	-	-	-	-	-	-	+	-	-	-	-	-	-	-	-	-
19	9038	BM16	*0701		-	-	-	-	-	-	+	-	-	-	-	-	-	-	-	-
20	9059	SLE005	*0304		-	-	-	-	-	+	-	+	-	-	-	-	-	-	-	-
21	9064	AMALA	*0303		-	-	-	-	-	+	-	+	-	-	-	-	-	-	-	-
22	9056	KOSE	*1203		-	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
23	9124	IHL	*0102	*1502	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
24	9035	JBUSH	*1203		-	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
25	9049	IBW9	*0802		-	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
26	9285	WT49	*0701		-	-	-	-	-	-	+	-	-	-	-	-	-	-	-	-
27	9191	CH1007	*0704	*1505	-	-	-	-	-	-	+	-	-	-	-	-	-	-	-	-
28	9320	BEL5GB	*0501	*1601	-	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
29	9050	MOU	*1601		-	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
30	9021	RSH	*1701		-	-	-	-	-	-	+	-	-	-	-	-	-	-	+	-
31	9019	DUCAF	*0501		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
32	9297	HAG	*1701	*1703	-	-	-	-	-	-	+	-	-	-	-	-	-	-	+	-
33	9098	MT14B	*0304		-	-	-	-	-	+	-	+	-	-	-	-	-	-	-	-
34	9104	DHIF	*1203		-	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
35	9302	SSTO	*0501		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
36	9024	KT17	*0303	*0401	-	+	+	-	-	+	-	+	-	-	-	-	-	-	-	+
37	9065	HHKB	*0702		-	-	-	-	-	-	+	-	-	-	-	-	-	-	-	-
38	9099	LZL	*0303		-	-	-	-	-	+	-	+	-	-	-	-	-	-	-	-
39	9315	CML	*0202	*0701	-	-	-	-	-	-	+	-	-	-	-	-	-	-	-	-
40	9134	WHONP199	*0602		-	-	-	-	-	-	+	-	-	-	-	-	-	-	+	-
41	9055	H0301	*0802		-	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
42	9066	TAB089	*0102		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
43	9076	T7526	*0102	*0801	-	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
44	9057	TEM	*1203		-	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
45	9239	SHJO	*0602	*1701	-	-	-	-	-	-	+	-	-	-	-	-	-	-	+	-
46	9013	SCHU	*0702		-	-	-	-	-	-	+	-	-	-	-	-	-	-	-	-
47	9045	TUBO	*0704	*1502	-	-	-	-	-	-	+	-	-	-	-	-	-	-	-	-
48	9303	TER-ND	*0401	*1601	-	+	+	+	-	-	-	-	-	-	-	-	-	-	-	+



Lot No.: **50G**

Lot-specific information

## CERTIFICATE OF ANALYSIS

### **Olerup SSP<sup>®</sup> HLA-Cw\*14 SSP**

**Product number:** 101.625-06u – without *Taq* polymerase  
**Lot number:** 50G  
**Expiry date:** 2011-November-01  
**Number of tests:** 6  
**Number of wells per test:** 16

#### **Well specifications:**

Well No.	Production No.	Well No.	Production No.
1	2008-535-01	9	2008-535-09
2	2009-658-02	10	2008-535-10
3	2008-535-03	11	2008-535-11
4	2009-658-04	12	2008-535-12
5	2009-658-05	13	2009-658-13
6	2008-535-06	14	2008-535-14
7	2008-535-07	15	2009-658-15
8	2008-535-08	16	2009-658-16

The specificity of each primer solution of the HLA-Cw\*14 primer set has been tested against 48 well characterized IHWC cell line DNAs.

No DNAs carrying the alleles to be amplified by primer mixes 10 to 14 were available. The specificities of the primers in primer solutions 10 and 13 were tested by separately adding one additional 5'-primer, respectively one additional 3'-primer. In primer solutions 11 and 14, it was only possible to test the 5'-primer, the 3'-primer was not possible to test. In primer solution 12, it was only possible to test the 3'-primer, the 5'-primer was not possible to test. In primer solution 13, one of the 3'-primers was not possible to test. Finally, one additional 5'-primer in primer solution 6 was tested by adding one additional 3'-primer.

**Results:** No false positive or false negative amplifications were obtained.

**Date of approval:** 2009-November-18

**Approved by:**

**Quality Control, Supervisor**

Lot No.: **50G**

Lot-specific information

## Declaration of Conformity

**Product name:** *Olerup* SSP® HLA-Cw\*14  
**Product number:** 101.625-06u  
**Lot number:** 50G

**Intended use:** HLA-Cw\*14 high resolution histocompatibility testing

**Manufacturer:** *Olerup* SSP AB  
Hasselstigen 1  
SE-133 33 Saltsjöbaden, Sweden  
**Phone:** +46-8-717 88 27  
**Fax:** +46-8-717 88 18

We, *Olerup* SSP AB, hereby declare that this product, to which this Declaration of Conformity relates is in conformity with the following Standard(s) and other normative document(s) ISO 9001:2008 and ISO 13485:2003, following the provisions of the 98/79/EC Directive on *in vitro* diagnostic medical devices, Annex III, as transposed into the national laws of the Member States of the European Union.

The Technical Documentation File is maintained at *Olerup* SSP AB, Hasselstigen 1, SE-133 33 Saltsjöbaden, Sweden.

The Authorized Representative located within the Community is: *Olerup* SSP AB.

Saltsjöbaden, Sweden  
2009- November-18

Olle Olerup  
Managing Director



Lot No.: **50G**

Lot-specific information

Lot No.: **50G**

Lot-specific information

Lot No.: **50G**

Lot-specific information

Lot No.: **50G**

Lot-specific information

**ADDRESSES:**

**Manufacturer:**

**Olerup SSP AB**, Hasselstigen 1, SE-133 33 Saltsjöbaden, Sweden.

**Tel:** +46-8-717 88 27

**Fax:** +46-8-717 88 18

**E-mail:** [info-ssp@olerup.com](mailto:info-ssp@olerup.com)

**Web page:** <http://www.olerup.com>

**Distributed by:**

**Olerup GmbH**, Löwengasse 47 / 6, AT-1030 Vienna, Austria.

**Tel:** +43-1-710 15 00

**Fax:** +43-1-710 15 00 10

**E-mail:** [support-at@olerup.com](mailto:support-at@olerup.com)

**Web page:** <http://www.olerup.com>

**Olerup Inc.**, 901 S. Bolmar St., Suite R, West Chester, PA 19382

**Tel:** 1-877-OLERUP1

**Fax:** 610-344-7989

**E-mail:** [info.us@olerup.com](mailto:info.us@olerup.com)

**Web page:** <http://www.olerup.com>

For information on *Olerup* SSP distributors worldwide, contact **Olerup GmbH**.