

Olerup SSP™ HLA-Cw*06

Product number: 101.614-12 – licensed for PCR
101.614-12u – not licensed for PCR
Lot number: X85
Expiry date: 2009-June-01
Number of tests: 12
Number of tubes per test: 16
Storage - pre-aliquoted primers: dark at -20°C
- PCR Master Mix: -20°C

This Product Description is only valid for Lot No. X85.

CHANGES COMPARED TO THE PREVIOUS OLERUP SSP™ HLA-Cw*06 LOT

The HLA-Cw*06 specificity and interpretation tables have been updated for the HLA-Cw alleles described since the previous *Olerup SSP™* HLA-Cw*06 lot was made (**Lot No. V92**).

Three tubes have been added to the HLA-Cw*06 kit,
wells **14 to 16**.

The primers of the tubes detailed below have been exchanged, added or modified compared to the previous lot.

Tube	5'-primer	3'-primer	rationale
4	Modified	-	Increased specificity of specific primer pair.
8	-	Modified	Increased specificity of specific primer pair.
11	Modified	-	Increased specificity of specific primer pair.
14	New	New	New primer pair for the Cw*0614 allele.
15	New	New	New primer pair for the Cw*0615 allele.
16	New	New	New primer pair for the Cw*0616N allele.

PRODUCT DESCRIPTION

HLA-Cw*06 SSP subtyping

CONTENT

The primer set contains 5'- and 3'-primers for identifying the HLA-Cw*0602 to HLA-Cw*0616N alleles.

The primer solutions are pre-aliquoted into 0.2 ml PCR tubes. Each tube in the set contains a dried primer solution consisting of a specific primer mix, i.e. allele- and group-specific primers as well as a **control primer pair** matching non-allelic sequences.

PCR Master Mix complete with Taq, Taq polymerase, nucleotides, buffer, glycerol and cresol red, as well as PCR lids are included in the licensed kit.

PCR Master Mix without Taq, nucleotides, buffer, glycerol and cresol red, as well as PCR lids are included in the unlicensed kit.

16 PCR reactions with a reaction volume of 10 µl are performed per sample.

Note: The pellets in the tubes may vary in form and colour. This does not affect the performance of the product.

PLATE LAYOUT

Each test consists of 16 PCR reactions in a 16 well PCR plate.

1	2	3	4	5	6	7	8
9	10	11	12	13	14	15	16

The 16 well cut PCR plate is marked with 'Cw*06 X85'.

Well No. 1 is marked with '1'.

The PCR plates are covered with a PCR-compatible foil.

Please note: When removing each 16 well PCR plate, make sure that the remaining plates stay covered. Use a scalpel or a similar instrument to carefully cut the foil between the plates.

INTERPRETATION

The interpretation of HLA-Cw*06 SSP subtypings will be influenced by the Cw*01 alleles, five Cw*02 alleles, the Cw*03 alleles, the Cw*04 alleles, the Cw*05 alleles, the Cw*08 alleles, most Cw*12 alleles, the Cw*14 alleles, the Cw*15 alleles, the Cw*160401 allele, the Cw*17 alleles and the Cw*18 alleles when present on the other haplotype.

UNIQUELY IDENTIFIED ALLELES

All the HLA-C*06 alleles, i.e. **Cw*0602 to Cw*0616N**, recognized by the HLA Nomenclature Committee in July 2007¹ will give rise to unique amplification patterns by the primers in the HLA-Cw*06 subtyping kit.

The HLA-Cw*06 SSP subtyping kit cannot distinguish the Cw*06020101 to Cw*060202 alleles.

¹**Nomenclature for factors of the HLA system, 1998.** *Tissue Antigens* 1999; **53**: 407-446.

HLA-Cw alleles listed on the IMGT/HLA web page 2007-July-09, release 2.18.0, www.ebi.ac.uk/imgt/hla.

RESOLUTION IN HOMO- AND HETEROZYGOTES

The 15 HLA-Cw*06 alleles can be combined in 120 homozygous and heterozygous combinations. Twenty of these genotypes do not give rise to unique amplification patterns.

++-----+	-----	0602,0607 = 0607,0607
++-----+	-----	0602,0608 = 0608,0608
++-----	+-----	0602,0609 = 0609,0609
++-----	-+-----	0602,0610 = 0610,0610
++-----	--+-----	0602,0611 = 0611,0611
++-----	---+-----	0602,0612 = 0612,0612
++-----	----+-----	0602,0613 = 0613,0613
++-----	-----+ -	0602,0615 = 0615,0615
++-----	-----+ +	0602,0616N = 0616N,0616N
+---++--	-----	0605,0605 = 0605,0606

LICENSES**101.614-12 – licensed for PCR.****Notice to purchaser: Limited License.**

The purchase price of this product includes limited, non-transferable rights under U.S. Patents 4,683,202, 4,683,195 and 4,965,188 and their foreign counterparts, owned by Roche Molecular Systems, Inc. and F. Hoffman-La Roche Ltd ("Roche"), to use only this amount of the product to practice the Polymerase Chain Reaction ("PCR") Process described in said patents solely for the HLA Typing applications of the purchaser solely for organ or tissue or bone marrow transplantation, and explicitly excludes analysis of forensic evidence or parentage determination. The rights to use this product to perform and to offer commercial service for HLA Typing for organ or tissue transplantation using PCR, including reporting the results of the purchaser's activities for a fee or other commercial consideration, is also hereby granted. Further information on purchasing licenses to practice PCR may be obtained by contacting in the United States, the Director of Licensing at Roche Molecular Systems, inc., 1145 Atlantic Avenue, Alameda, California 94501, and outside the United States, the PCR Licensing Manager, F. Hoffmann-La Roche Ltd, Grenzacherstr. 124, CH-4070 Basel, Switzerland.

101.614-12u – not licensed for PCR.**Notice to purchaser: Disclaimer of License.**

This product is optimized for use in the Polymerase Chain Reaction ("PCR") Process which is covered by patents owned by Roche Molecular Systems, Inc. and F. Hoffmann-La Roche Ltd ("Roche"). No license under these patents to use the PCR Process is conveyed expressly or by implication to the purchaser of this product. Further information on purchasing licenses to practice PCR may be obtained by contacting in the United States, the Director of Licensing at Roche Molecular Systems, inc., 1145 Atlantic Avenue, Alameda, California 94501.

101.614-12 and 101.614-12u

These products use ARMSTM technology and is sold under license from Zeneca Limited. ARMS is the subject of European Patent No. 0332435, US Patent No. 5595890 and corresponding world-wide patents. ARMS is a trademark of Zeneca Limited.

GUARANTEE

Olerup SSP AB guarantees that the primers in the HLA-Cw*06 subtyping kit have the specificities given in the Specificity and Interpretation Tables of the product insert and in the GenoVision version of the HELMBERG-SCORETM software.

When stored at –20°C, the dried primers are stable for 22 months from the date of manufacture.

When stored at –20°C, the PCR Master Mix complete with *Taq* and the PCR Master Mix without *Taq* are stable for 24 months from the date of manufacture.

The kit is shipped at ambient temperature.

PROTOCOL

DNA EXTRACTION

Extracted, highly pure DNA is needed for SSP typings. We recommend isolation of DNA using GenoPrep B200 or GenoPrep B350 cartridges on the GenoM™-6 robotic workstation (GenoVision Europe Tel: +43 1 710 15 00 or GenoVision Inc. USA Tel: +1 610 430 88 41; <http://www.genovision.com>). Using GenoM™-6-extracted DNA ACD, EDTA and heparinised blood can be used as starting material. Because of its high purity, GenoM™-6-extracted DNA can be diluted when used in combination with Olerup SSP™ products. The recommended DNA concentration is 15 ng/μl.

Alternatively – BUT DO NOT USE HEPARINISED BLOOD WITH THESE METHODS - the DNA can be extracted using trimethylammoniumbromide salts (DTAB/CTAB) or by salting out. Dissolve the extracted DNA in dH₂O.

IMPORTANT:

Optimal DNA concentration using: GenoM™-6-extracted DNA, 15 ng/μl.

DNA extracted by other methods, 30 ng/μl.

Concentration exceeding 50 ng/μl will increase the risk for nonspecific amplifications.

PCR AMPLIFICATION

101.614-12 – licensed for PCR

For one HLA-Cw*06 subtyping, add at room temperature in a 0.5 ml tube:

19 x 2 μl = 38 μl DNA (30 ng/μl)

19 x 3 μl = 57 μl PCR Master Mix complete with *Taq* – mix well before taking your aliquot

19 x 5 μl = 95 μl dH₂O

Mix well, dispense 10 μl of the DNA-PCR Master Mix-H₂O mixture into each of the 16 wells of an HLA-Cw*06 subtyping. **Well No. 1 of the 16 well PCR plate is marked with '1'**. Close the 16 well PCR plate with the provided lids.

101.614-12u – not licensed for PCR

For one HLA-Cw*06 subtyping, add at room temperature in a 0.5 ml tube:

19 x 2 μl = 38 μl DNA (30 ng/μl)

19 x 3 μl = 57 μl PCR Master Mix without *Taq* – mix well before taking your aliquot

1.5 μl *Taq* polymerase (5 units/μl)

19 x 5 μl – 1.5 μl = 93.5 μl dH₂O

Mix well, dispense 10 μl of the DNA-PCR Master Mix-*Taq*-H₂O mixture into each of the 16 wells of an HLA-Cw*06 subtyping. **Well No. 1 of the 16 well PCR plate is marked with '1'**. Close the 16 well PCR plate with the provided lids.

Use a 96 well thermal cycler with a heated lid. The temperature gradient across the heating block should be < 1°C.

PCR cycling parameters:

1. 1 cycle	94°C	2 min	denaturation
2. 10 cycles	94°C	10 sec.	denaturation
	65°C	60 sec.	annealing and extension
3. 20 cycles	94°C	10 sec.	denaturation
	61°C	50 sec.	annealing
	72°C	30 sec.	extension

The same PCR cycling parameters are used for all the Olerup SSP kits.

AGAROSE GEL ELECTROPHORESIS

Prepare a 2% (w/v) agarose gel in 0.5 x TBE buffer. Dissolve the agarose by boiling in a microwave oven. Let the gel solution cool to 60°C. Stain the gel prior to casting with ethidium bromide (10 mg/ml), 5 µl per 100 ml gel solution. For maximal ease of handling use our ethidium bromide dropper bottles (Product No. 103.301-10), 1 drop of ethidium bromide solution per 50-75 ml of gel. **Note:** Ethidium bromide is a powerful carcinogen.

Load the PCR products, preferably using an 8-channel pipette. Load a DNA size marker (100 base pair ladder, Product No. 103.201-100) in one well per row.

Run the gel in 0.5 x TBE buffer, without re-circulation of the buffer, for 15-20 minutes at 8-10 V/cm.

DOCUMENTATION AND INTERPRETATION

Put the gel on a UV transilluminator and document by photography.

Record the presence and absence of specific PCR products. The relative lengths of the specific PCR products are helpful in the interpretation of the results.

Record the presence and relative lengths of the internal positive control bands. The differently sized control bands will help in the correct orientation of the typing as well as in kit identification.

Lanes without either control band or specific PCR products should be repeated.

Interpret the typings with the ***lot-specific Interpretation and Specificity Tables***.

INTERPRETATION SOFTWARE

The interpretation software (Product No. 110.101) can be helpful in the interpretation of the typings.

PCR MASTER MIXES

The PCR Master Mix complete with *Taq* contains:

<i>Taq</i> polymerase	0.4 unit per 10 µl SSP reaction
nucleotides	final concentration of each dNTP is 200 µM
PCR buffer	final concentrations: 50 mM KCl, 1.5 mM MgCl ₂ , 10 mM Tris-HCl pH 8.3, 0.001% w/v gelatin
glycerol	final concentration of glycerol is 5%
cresol red	final concentration of cresol red is 100 µg/ml

The same PCR Master Mix complete with *Taq* is used for all the licensed *Olerup* SSP kits.

The PCR Master Mix without *Taq* contains:

nucleotides	final concentration of each dNTP is 200 µM
PCR buffer	final concentrations: 50 mM KCl, 1.5 mM MgCl ₂ , 10 mM Tris-HCl pH 8.3, 0.001% w/v gelatin
glycerol	final concentration of glycerol is 5%
cresol red	final concentration of cresol red is 100 µg/ml

The same PCR Master Mix without *Taq* is used for all the unlicensed *Olerup* SSP kits.

The PCR Master Mix complete with *Taq* and the PCR Master Mix without *Taq* can be shipped at ambient temperature.

When stored at –20°C, the PCR Master Mix complete with *Taq* and the PCR Master Mix without *Taq* are stable for 24 months from the date of manufacture.

Vials with the PCR Master Mixes can be kept at +4°C for 4 weeks, but the PCR Master Mixes are then no longer stable for 24 months.

SPECIFICITY TABLE

HLA-Cw*06 SSP subtyping

Specificities and sizes of the PCR products of the 16 primer mixes used for HLA-Cw*06 SSP subtyping

Primer Mix	Approx. Size of spec. PCR product ¹	Size of control band ²	Amplified HLA-Cw*06 alleles	Other amplified HLA-Cw alleles ³
1	240 bp	800 bp	06020101-060202, 0604-0616N	0339, 1216
2	220 bp	800 bp	06020101-0603, 0607- 0613, 0615, 0616N	0104, 0109, 0205, 0217, 120301-1207, 1211-1213, 1215, 160401
3	135 bp	1070 bp	0603	030201-0317, 0319-0338, 0340, 0341
4⁴	240 bp	1070 bp	0604	0203, 0216, 0218, 040401-040402, 0406, 0413, 0511, 0517, 080101-080102, 0803, 0804, 0806, 0808-0811, 0813, 0814, 1214, 1218, 1220, 1406, 150201-1507, 1509-1521, 1701-1704
5	165 bp	800 bp	0605	050101-0517, 0810, 1221
6	240 bp	1070 bp	0605, 0606	010201-0103, 0105-0108, 0110-0119, 04010101-040104, 0403, 0405, 0407-0412, 0415-0421, 0423-0427, 050101-0507N, 0509, 0510, 0512-0516, 0802, 0805, 0807, 0812, 1209, 140201-1405, 1407N, 1408, 1508, 1801, 1802
7	110 bp	1070 bp	0607	
8	240 bp	1070 bp	0608	0110, 0205, 0217
9	165 bp	1070 bp	0609	0508, 1803

10	200 bp	800 bp	0610	
11	130 bp	1070 bp	0611	070401-070402, 0711, 0712, 0745
12	210 bp	1070 bp	0612	
13	155 bp	1070 bp	0613	
14	295 bp	1070 bp	0614	0332, 0710, 0743
15	365 bp	800 bp	0615	
16	240 bp	1070 bp	0616N	

¹ Alleles are assigned by the presence of specific PCR product(s). However, the sizes of the specific PCR products may be helpful in the interpretation of HLA-Cw*06 SSP subtypings. Nonspecific amplifications, i.e. a ladder or a smear of bands, may sometimes be seen. GC-rich primers have a higher tendency of giving rise to nonspecific amplifications than other primers. PCR fragments longer than the control band may sometimes be observed. Such bands can be disregarded and do not influence the interpretation of the SSP typings.

² The internal positive control primer pairs amplify segments of the human growth hormone gene. The two different control primer pairs give rise to either an internal positive control band of 1070 base pairs, for most tubes, or a band of 800 base pairs, for some tubes.

Tube number 1 contains the primer pair giving rise to the shorter, 800 bp, internal positive control band in order to help in the correct orientation of the HLA-Cw*06 subtyping.

In addition, tubes number 2, 5, 10 and 15 contain the primer pair giving rise to the shorter, 800 bp, internal positive control in order to allow kit identification.

PLEASE NOTE: All the SSP kits, except the B*37, B*41, B*42, B*46, B*47, B*48, B*49, B*50, B*53, B*67, B*78, B*81 and B*82 kits and the Cw*01, Cw*02, Cw*08, Cw*12, Cw*14, Cw*15, Cw*16, Cw*17 and Cw*18 kits, from *Olerup* SSP AB can be uniquely identified by the number of tubes and the kit-specific pattern of the two differently sized control bands.

In the presence of a specific amplification the intensity of the control band often decreases.

³ Due to the sharing of sequence motifs between HLA-Cw alleles non-HLA-Cw*06 alleles will be amplified by primer mixes 1 to 6, 8, 9, 11 and 14.

⁴ Primer mix 4 will amplify some B*5802-positive samples.

INTERPRETATION TABLE								
HLA-Cw*06 SSP subtyping								
Amplification patterns of the Cw*0602 to Cw*0616N alleles								
	Tube⁵							
	1	2	3	4	5	6	7	8
Length of spec.	240	220	135	240	165	240	110	240
PCR product								
Length of int.	800	800	1070	1070	800	1070	1070	1070
pos. control¹								
5'-primer²	28	361	105	2nd In	176	2nd In	232	361
	5'-TCA ^{3'}	5'-AgT ^{3'}	5'-gCT ^{3'}	5'-CCA ^{3'}	5'-gCA ^{3'}	5'-CCA ^{3'}	5'-AgA ^{3'}	5'-AgT ^{3'}
3'-primer³	97	538	201	539	302	538	302	539
	5'-gTC ^{3'}	5'-CCA ^{3'}	5'-CTC ^{3'}	5'-TCA ^{3'}	5'-ggT ^{3'}	5'-CCg ^{3'}	5'-ggT ^{3'}	5'-CTC ^{3'}
Tube No.	1	2	3	4	5	6	7	8
HLA-Cw allele⁴								
*06020101-060202	+	+						
*0603		+	+					
*0604	+			+				
*0605	+				+	+		
*0606	+					+		
*0607	+	+					+	
*0608	+	+						+
*0609	+	+						
*0610	+	+						
*0611	+	+						
*0612	+	+						
*0613	+	+						
*0614	+							
*0615	+	+						
*0616N	+	+						
*010201-0103, 0105-0108, 0111-0119, 04010101-040104, 0403, 0405, 0407-0412, 0415-0421, 0423-0427, 0802, 0805, 0807, 0812, 1209, 140201-1405, 1407N, 1408, 1508, 1801, 1802						+		
*0104, 0109, 12030101-1207, 1211-1213, 1215, 160401		+						
*0110						+		+
Tube No.	1	2	3	4	5	6	7	8

INTERPRETATION TABLE								
HLA-Cw*06 SSP subtyping								
Amplification patterns of the Cw*0602 to 0616N alleles								
Tube ⁵								
9	10	11	12	13	14	15	16	
165	200	130	210	155	295	365	240	Length of spec.
								PCR product
1070	800	1070	1070	1070	1070	800	1070	Length of int.
								pos. control ¹
412	142	213	529	361	341	376	501	5'-primer ²
5'-ATA ^{3'}	5'-TCC ^{3'}	5'-CCC ^{3'}	5'-AgA ^{3'}	5'-AgT ^{3'}	5'-ggA ^{3'}	5'-gCT ^{3'}	5'-ggC ^{3'}	
538	302	302	3 rd I	475	353	3 rd I	3 rd I	3'-primer ³
5'-CCA ^{3'}	5'-ggT ^{3'}	5'-ggC ^{3'}	5'-CTC ^{3'}	5'-ggT ^{3'}	5'-TgA ^{3'}	5'-CTC ^{3'}	5'-CTC ^{3'}	
9	10	11	12	13	14	15	16	Tube No.
								HLA-Cw allele ⁴
								*06020101-060202
								*0603
								*0604
								*0605
								*0606
								*0607
								*0608
+								*0609
	+							*0610
		+						*0611
			+					*0612
				+				*0613
					+			*0614
						+		*0615
							+	*0616N
								*010201-0103, 0105-0108, 0111-0119, 04010101-040104, 0403, 0405, 0407-0412, 0415- 0421, 0423-0427, 0802, 0805, 0807, 0812, 1209, 140201- 1405, 1407N, 1408, 1508, 1801, 1802
								*0104, 0109, 12030101-1207, 1211-1213, 1215, 160401
								*0110
9	10	11	12	13	14	15	16	Tube No.

Length of spec.	240	220	135	240	165	240	110	240
PCR product								
Tube No.	1	2	3	4	5	6	7	8
*0203, 0216, 0218, 040401-040402, 0406, 0413, 080101-080102, 0803, 0804, 0806, 0808, 0809, 0811, 0813, 0814, 1214, 1218, 1220, 1406, 150201-1507, 1509-1521, 1701-1704				+				
*0205, 0217		+						+
*030201-0317, 0319-0331, 0333-0338, 0340, 0341			+					
*0332			+					
*0339, 1216	+							
*050101-0507N, 0509, 0510, 0512-0516					+	+		
*0508					+			
*0511, 0517, 0810				+	+			
*070401-070402, 0711, 0712, 0745								
*0710, 0743								
*1221					+			
*1803								
HLA-Cw allele ⁴								
Tube No.	1	2	3	4	5	6	7	8

¹The internal positive control primer pairs amplify segments of the human growth hormone gene. The two different control primer pairs give rise to either an internal positive control band of 1070 base pairs, for most tubes, or a band of 800 base pairs, for some tubes.

Tube number 1 contains the primer pair giving rise to the shorter, 800 bp, internal positive control band in order to help in the correct orientation of the HLA-Cw*06 subtyping.

In addition, tubes number 2, 5, 10 and 15 contain the primer pair giving rise to the shorter, 800 bp, internal positive control in order to allow kit identification.

PLEASE NOTE: All the SSP kits, except the B*37, B*41, B*42, B*46, B*47, B*48, B*49, B*50, B*53, B*67, B*78, B*81 and B*82 kits and the Cw*01, Cw*02, Cw*08, Cw*12, Cw*14, Cw*15, Cw*16, Cw*17 and Cw*18 kits, from *Olerup* SSP AB can be uniquely identified by the number of tubes and the kit-specific pattern of the two differently sized control bands.

165	200	130	210	155	295	365	240	Length of spec. PCR product
9	10	11	12	13	14	15	16	Tube No.
								*0203, 0216, 0218, 040401- 040402, 0406, 0413, 080101- 080102, 0803, 0804, 0806, 0808, 0809, 0811, 0813, 0814, 1214, 1218, 1220, 1406, 150201-1507, 1509-1521, 1701-1704
								*0205, 0217
								*030201-0317, 0319-0331, 0333-0338, 0340, 0341
					+			*0332
								*0339, 1216
								*050101-0507N, 0509, 0510, 0512-0516
+								*0508
								*0511, 0517, 0810
		+						*070401-070402, 0711, 0712, 0745
					+			*0710, 0743
								*1221
+								*1803
								HLA-Cw allele ⁴
9	10	11	12	13	14	15	16	Tube No.

²The nucleotide position, in the 1st, 2nd or 3rd exons or the 2nd intron, matching the specificity-determining 3'-end of the primer is given. Nucleotide numbering as in *Tissue Antigens* 1998, **51:II**, 417-466. The sequence of the 3 terminal nucleotides of the primer is given.

³The nucleotide position, in the 2nd or 3rd exons or the 3rd intron, matching the specificity-determining 3'-end of the primer is given in the anti-sense direction. Nucleotide numbering as in *Tissue Antigens* 1998, **51:II**, 417-466. The sequence of the 3 terminal nucleotides of the primer is given.

⁴The sequence of the Cw*0601 allele has been shown to be identical to Cw*0602.

⁵Primer mix 4 will amplify some B*5802-positive samples.

CELL LINE VALIDATION SHEET																				
HLA-Cw*06 SSP subtyping kit																				
					Tube															
					1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
				Prod. No.:	200734401	200734402	200734403	200734404	200734405	200734406	200734407	200734408	200734409	200734410	200734411	200734412	200734413	200734414	200734415	200734416
	cell line		Cw*																	
1	9001	SA	*0702		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
2	9280	LK707	*0701	*1505	-	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
3	9011	E4181324	*1202		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
4	9275	GU373	*0304	*0401	-	-	+	-	-	+	-	-	-	-	-	-	-	-	-	-
5	9009	KAS011	*0602		+	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
6	9353	SM	*0304	*0702	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-	-
7	9020	QBL	*0501		-	-	-	-	+	+	-	-	-	-	-	-	-	-	-	-
8	9007	DEM	*0602		+	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
9	9026	YAR	*1203		-	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
10	9107	LKT3	*0102		-	-	-	-	-	+	-	-	-	-	-	-	-	-	-	-
11	9051	PITOUT	*1601		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
12	9052	DBB	*0602		+	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
13	9067	BTB	*0102		-	-	-	-	-	+	-	-	-	-	-	-	-	-	-	-
14	9071	OLGA	*0102	*0304	-	-	+	-	-	+	-	-	-	-	-	-	-	-	-	-
15	9075	DKB	*0304		-	-	+	-	-	-	-	-	-	-	-	-	-	-	-	-
16	9037	SWEIG007	*0202		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
17	9008	WILJON	*1203		-	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
18	9257	32367	*0102	*0705	-	-	-	-	-	+	-	-	-	-	-	-	-	-	-	-
19	9038	BM16	*0701		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
20	9059	SLE005	*0304		-	-	+	-	-	-	-	-	-	-	-	-	-	-	-	-
21	9064	AMALA	*0303		-	-	+	-	-	-	-	-	-	-	-	-	-	-	-	-
22	9056	KOSE	*1203		-	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
23	9124	IHL	*0102	*1502	-	-	-	+	-	+	-	-	-	-	-	-	-	-	-	-
24	9035	JBUSH	*1203		-	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
25	9049	IBW9	*0802		-	-	-	-	-	+	-	-	-	-	-	-	-	-	-	-
26	9285	WT49	*0701		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
27	9191	CH1007	*0704	*1505	-	-	-	+	-	-	-	-	-	-	+	-	-	-	-	-
28	9320	BEL5GB	*0501	*1601	-	-	-	-	+	+	-	-	-	-	-	-	-	-	-	-
29	9050	MOU	*1601		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
30	9021	RSH	*1701		-	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
31	9019	DUCAF	*0501		-	-	-	-	+	+	-	-	-	-	-	-	-	-	-	-
32	9297	HAG	*1701	*1703	-	-	-	+	-	-	-	-	-	-	-	-	-	-	-	-
33	9098	MT14B	*0304		-	-	+	-	-	-	-	-	-	-	-	-	-	-	-	-
34	9104	DHIF	*1203		-	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
35	9302	SSTO	*0501		-	-	-	-	+	+	-	-	-	-	-	-	-	-	-	-
36	9024	KT17	*0303	*0401	-	-	+	-	-	+	-	-	-	-	-	-	-	-	-	-
37	9065	HHKB	*0702		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
38	9099	LZL	*0303		-	-	+	-	-	-	-	-	-	-	-	-	-	-	-	-
39	9315	CML	*0202	*0701	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
40	9134	WHONP199	*0602		+	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
41	9055	H0301	*0802		-	-	-	-	-	+	-	-	-	-	-	-	-	-	-	-
42	9066	TAB089	*0102		-	-	-	-	-	+	-	-	-	-	-	-	-	-	-	-
43	9076	T7526	*0102	*0801	-	-	-	+	-	+	-	-	-	-	-	-	-	-	-	-
44	9057	TEM	*1203		-	+	-	-	-	-	-	-	-	-	-	-	-	-	-	-
45	9239	SHJO	*0602	*17	+	+	-	+	-	-	-	-	-	-	-	-	-	-	-	-
46	9013	SCHU	*0702		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
47	9045	TUBO	*0704	*1502	-	-	-	+	-	-	-	-	-	-	+	-	-	-	-	-
48	9303	TER-ND	*0401	*1601	-	-	-	-	-	+	-	-	-	-	-	-	-	-	-	-

CERTIFICATE OF ANALYSIS

Olerup SSP™ HLA-Cw*06 SSP

Product number: 101.614-12 – licensed for PCR
101.614-12u – not licensed for PCR
Lot number: X85
Expiry date: 2009-June-01
Number of tests: 12
Number of tubes per test: 16

Tube specifications:

Tube No.	Production No.	Tube No.	Production No.
1	2007-344-01	9	2007-344-09
2	2007-344-02	10	2007-344-10
3	2007-344-03	11	2007-344-11
4	2007-344-04	12	2007-344-12
5	2007-344-05	13	2007-344-13
6	2007-344-06	14	2007-344-14
7	2007-344-07	15	2007-344-15
8	2007-344-08	16	2007-344-16

The specificity of each primer solution of the kit has been tested against 48 well characterized cell line DNAs.

No DNAs carrying the alleles to be amplified by primer solutions 7, 8, 9, 10, 12 to 16 were available. It was only possible to test the 3'-primer in primer solutions 7, 10, 12, 15 and 16 by separately adding another 5'-primer, the 5'-primer was not possible to test. In primer solutions 8, 9 and 14 it was possible to test both the 5'-primer and the 3'-primer. In primer solution 13 it was only possible to test the 5'-primer, the 3'-primer was not possible to test.

Results: No false positive or false negative amplifications were obtained.

Date of approval: 2007-August-07

Approved by:

Quality Control, Supervisor

Declaration of Conformity

Product name: Olerup SSP™ HLA-Cw*06
Product number: 101.614-12, 101.614-12u
Lot number: X85

Intended use: HLA-Cw*06 high resolution histocompatibility testing

Manufacturer: Olerup SSP AB
Hasselstigen 1
SE-133 33 Saltsjöbaden, Sweden
Phone: +46-8-717 88 27
Fax: +46-8-717 88 18

We, Olerup SSP AB, hereby declare that this product, to which this Declaration of Conformity relates is in conformity with the following Standard(s) and other normative document(s) ISO 9001:2000, ISO 17025:1999 and ISO 13485:2000, following the provisions of the 98/79/EC Directive on *in vitro* diagnostic medical devices, Annex III.

The Technical Construction File is maintained at Olerup SSP AB, Hasselstigen 1, SE-133 33 Saltsjöbaden, Sweden.

The Authorized Representative located within the Community is: Olerup SSP AB.

Saltsjöbaden, Sweden
2007-August-07

Olle Olerup

HLA-Cw*06

101.614-12 – licensed for PCR

101.614-12u – not licensed for PCR

Lot No.: **X85**

17

www.olerup.com

HLA-Cw*06

101.614-12 – licensed for PCR

101.614-12u – not licensed for PCR

Lot No.: **X85**

18

www.olerup.com

WARRANTY

Olerup SSP AB warrants its products to the original purchaser against defects in materials and workmanship under normal use and application. *Olerup* SSP AB's sole obligation under this warranty shall be to replace, at no charge, any product that does not meet the performance standards stated on the product specification sheet.

This warranty applies only to products that have been handled and stored in accordance with *Olerup* SSP AB's recommendations, and does not apply to products that have been the subject of alternation, misuse, or abuse.

All claims under this warranty must be directed to *Olerup* SSP AB in writing and must be accompanied by a copy of the purchaser's invoice. This warranty is in lieu of all other warranties, expressed or implied, including the warranties of merchantability and fitness for a particular purpose. In no case shall *Olerup* SSP AB be liable for incidental or consequential damages.

This product may not be reformulated, repacked or resold in any form without the written consent of *Olerup* SSP AB, Hasselstigen 1, SE-133 33 Saltsjöbaden, Sweden.

Handle all samples as if capable of transmitting disease. All work should be performed wearing gloves and appropriate protection.

Olerup SSPTM is a trademark of *Olerup* SSP AB.
PCRTM is a trademark of F. Hoffmann-La Roche Ltd.
ARMSTM is a trademark of Zeneca Ltd.

HLA-Cw*06
101.614-12 – licensed for PCR
101.614-12u – not licensed for PCR
Lot No.: **X85**

20

www.olerup.com

ADDRESSES:

Manufacturer:

Olerup SSP AB, Hasselstigen 1, SE-133 33 Saltsjöbaden, Sweden.

Tel: +46-8-717 88 27

Fax: +46-8-717 88 18

E-mail: info-ssp@olerup.com

Web page: <http://www.olerup.com>

Distributed by:

Olerup GmbH, Löwengasse 47 / 6, AT-1030 Vienna, Austria.

Tel: +43-1-710 15 00

Fax: +43-1-710 15 00 10

E-mail: support-at@olerup.com

Web page: <http://www.olerup.com>

Olerup Inc., 901 S. Bolmar St., Suite R, West Chester, PA 19382

Tel: 1-877-OLERUP1

Fax: 610-344-7989

E-mail: info.us@olerup.com

Web page: <http://www.olerup.com>

For information on *Olerup* SSP distributors worldwide, contact **Olerup GmbH**.