

101.128-12 – including *Taq* polymerase101.128-12u – without *Taq* polymeraseLot No.: **51K****Olerup SSP® DRB1*12**

Product number: 101.128-12 – including *Taq* polymerase
101.128-12u – without *Taq* polymerase
Lot number: 51K
Expiry date: 2013-July-01
Number of tests: 12
Number of wells per test: 24

CHANGES COMPARED TO THE PREVIOUS DRB1*12 LOT (89G):

Well	5'-primer	3'-primer	rationale
13	Added	-	Primer added for the DRB1*12:24N allele.
16	Added	-	Primer added for the DRB1*12:23 allele.
17	Added	Added	Primer pair added for the DRB1*12:25 allele.
18	-	Added	Primer added for the DRB1*12:26 allele.
22	-	Added	Primer added for the DRB1*12:21 allele.
23	Added	Added	Primer pair added for the DRB1*12:25 allele.

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

DRB1*12:01 to DRB1*12:26, i.e. all the currently recognized DRB1*12 alleles, give rise to unique amplification patterns; www.ebi.ac.uk/imgt/hla, 2010-October-15, release 3.2.0.

RESOLUTION IN DRB1*12 HOMOZYGOTES:

Excellent.

INFLUENCE ON THE INTERPRETATION OF DRB1*12 SUBTYPINGS BY NON-DRB1*12 ALLELES:

None frequently occurring.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.