

Olerup SSP[®] DRB1*12

Product number: 101.128-12 – including *Taq* polymerase
101.128-12u – without *Taq* polymerase
Lot number: 83M
Expiry date: 2014-April-01
Number of tests: 12
Number of wells per test: 24

CHANGES COMPARED TO THE PREVIOUS DRB1*12 LOT (51K):

Well	5'-primer	3'-primer	rationale
17	-	Modified	3'-primer modified for improved amplification of the DRB1*12:10 allele.
20	Added	-	Primer added for resolution of the DRB1*12:27 allele.

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

DRB1*12:01 to DRB1*12:27, i.e. all the currently recognized DRB1*12 alleles, give rise to unique amplification patterns; www.ebi.ac.uk/imgt/hla, 2011-April-08, release 3.4.0.

RESOLUTION IN DRB1*12 HOMOZYGOTES:

Excellent.

INFLUENCE ON THE INTERPRETATION OF DRB1*12 SUBTYPINGS BY NON-DRB1*12 ALLELES:

None frequently occurring.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

Primer mix 17 modified for improved amplification of the DRB1*12:10 allele.