

Olerup SSP[®] DRB1*12 Add-on

Product number:	101.815-12 – including <i>Taq</i> polymerase 101.815-12u – without <i>Taq</i> polymerase
Lot number:	31N
Expiry date:	2014-August-01
Number of tests:	12
Number of wells per test:	2

THE NUMBER OF WELLS is 2 in an 8 well plate.

ALLELE COVERAGE:

The DRB1*12:01 and DRB1*12:10 alleles give different patterns in the DRB1*12 Add-on subtyping kit¹; www.ebi.ac.uk/imgt/hla, 2012-January-12, release 3.7.0.

¹The DRB1*12 Add-on subtyping kit cannot distinguish the following silent mutations: the DRB1*12:01:01-12:01:04 alleles.

RESOLUTION IN HOMO- AND HETEROZYGOTES:

Excellent.

INFLUENCE ON THE INTERPRETATION OF DRB1*12 ADD-ON SUBTYPINGS BY NON-DRB1*12 ALLELES:

None.