

101.118-24 – including *Taq* polymerase

101.118-24u – without *Taq* polymerase

Lot No.: 3N6

Olerup SSP® DRB1*07

Product numbers: 101.118-24 – including *Taq* polymerase
 101.118-24u – without *Taq* polymerase
Lot number: 3N6
Expiry date: 2025-07-01
Number of tests: 24
Number of wells per test: 21+1

CHANGES COMPARED TO THE PREVIOUS DRB1*07 LOT (6K9):

Well	5'-primer	3'-primer	rationale
8	-	Exchanged	3'-primer exchanged due to unspecific amplification.

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

DRB1*07:01 to DRB1*07:123, i.e. all the currently recognized DRB1*07 alleles, will be amplified by the primers in the DRB1*07 subtyping kit; www.ebi.ac.uk/imgt/hla, 2021-April-20¹, release 3.44.0.

The DRB1*07 kit enables separation of the confirmed DRB1*07 alleles as listed in the IMGT/HLA database 3.25.0. An HLA allele is listed as confirmed by IMGT/HLA if it has been sequenced by more than a single laboratory or from multiple sources.

The DRB1*07 kit also enables identification of many null and alternatively expressed alleles.

The following DRB1*07 alleles can be distinguished by the different sizes of the specific PCR product:

Alleles	Primer mix	Alleles	Primer mix
DRB1*07:06, 07:32	6	DRB1*07:39, 07:68N	20
DRB1*07:14, 07:24	11	DRB1*07:40, 07:56	17
DRB1*07:17, 07:27	14	DRB1*07:44, 07:48	21
DRB1*07:30, 07:42	19		

¹Alleles that have been deleted from or renamed in the official WHO HLA Nomenclature up to and including the last IMGT/HLA database release can be retrieved from web page <http://hla.alleles.org/alleles/deleted.html>.



0197

DRB1*07

Release Note

Page 2 of 2

101.118-24 – including *Taq* polymerase

101.118-24u – without *Taq* polymerase

Lot No.: 3N6

RESOLUTION IN DRB1*07 HOMOZYGOTES:

The DRB1*07:01,07:01 genotype gives rise to a unique amplification pattern.

INFLUENCE ON THE INTERPRETATION OF DRB1*07 SUBTYPINGS BY NON-DRB1*07 ALLELES:

None of importance.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No suggestions received.



0197

For *In Vitro* Diagnostic Use

MA125 v01 SSP Release Note Template

Date: July 2021, Rev. No: 00