

## Olerup SSP<sup>®</sup> DRB1\*09

Product number: 101.120-06 – including *Taq* polymerase  
101.120-06u – without *Taq* polymerase  
Lot number: 07G  
Expiry date: 2011-July-01  
Number of tests: 6 tests – Product No. 101.120-06  
Number of wells per test: 8

### Changes compared to the previous DRB1\*09 Lot (18F):

Well	5'-primer	3'-primer	rationale
3	Exchanged	Exchanged	New primer pair for the DRB1*090201 and *090202 alleles.
5	-	Added	New primer added for the DRB1*0908 allele.

THE NUMBER OF WELLS is unchanged.

### ALLELE COVERAGE:

DRB1\*0901 to DRB1\*0908, i.e. all the currently recognized DRB1\*09 alleles, give rise to unique amplification patterns; [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla), 2009-May-11, release 2.25.2.

### RESOLUTION IN DRB1\*09 HOMOZYGOTES:

Very good.

### INFLUENCE ON THE INTERPRETATION OF DRB1\*09 SUBTYPINGS BY NON-DRB1\*09 ALLELES:

None.

### MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No suggestions received.