

Olerup SSP[®] DRB3

Product number: 101.121-24/04 – including *Taq* pol.
101.121-24u/04u – without *Taq* pol.
Lot number: 52R
Expiry date: 2015-April-01
Number of tests: 24 tests – Product No. 101.121-24/24u
4 tests – Product No. 101.121-04/04u
Number of wells per test: 32

CHANGES COMPARED TO THE PREVIOUS DRB3 LOT (71M):

Well	5'-primer	3'-primer	rationale
31	New	New	New primer pair for the DRB3*02:29N allele.
32	New	New	New primer pair for improved allelic resolution.

THE NUMBER OF WELLS is increased from 30 to 32.

ALLELE COVERAGE:

DRB3*01:01 to DRB3*01:15, DRB3*02:01 to DRB3*02:29N and DRB3*03:01 to DRB3*03:03, i.e. all the currently recognized DRB3 alleles, give rise to unique amplification patterns; www.ebi.ac.uk/imgt/hla, 2012-July-12, release 3.9.0.

RESOLUTION IN DRB3 HOMOZYGOTES:

Good.

INFLUENCE ON THE INTERPRETATION OF DRB3 SUBTYPINGS BY NON-DRB3 ALLELES:

None frequently occurring.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.