

101.847-12 – including *Taq* polymerase101.847-12u – without *Taq* polymerase

Lot No.: 2E5

**Olerup SSP® HLA-A\*03 Add-on**

<b>Product number:</b>	101.847-12 – including <i>Taq</i> polymerase 101.847-12u – without <i>Taq</i> polymerase
<b>Lot number:</b>	2E5
<b>Expiry date:</b>	2019-02-01
<b>Number of tests:</b>	12
<b>Number of wells per test:</b>	2+1

The HLA-A\*03 Add-on primer set is unchanged compared to the previous *Olerup SSP® HLA-A\*03 Add-on (Lot No. 33Y)*.

**THE NUMBER OF WELLS** is 3 in an 8 well plate.

**ALLELE COVERAGE:**

The HLA-A\*03:01 and A\*03:21N alleles give different patterns in the HLA-A\*03 Add-on kit<sup>1</sup>; [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla), 2016-April-15, release 3.24.0.

The HLA-A\*03 add-on kit cannot distinguish the silent mutations in the A\*03:01:01:01-03:01:29, 03:01:31-03:01:47, 03:01:51-03:01:57 alleles.

<sup>1</sup>Alleles that have been deleted from or renamed in the official WHO HLA Nomenclature up to and including the last IMGT/HLA database release can be retrieved from web page <http://hla.alleles.org/alleles/deleted.html>.

**RESOLUTION IN HOMO- AND HETEROZYGOTES:**

Excellent.

**INFLUENCE ON THE INTERPRETATION OF HLA-A\*03 ADD-ON SUBTYPINGS BY NON-HLA-A\*03 ALLELES:**

None of importance.