

HLA-A*01

101.411-24/06 – including *Taq* polymerase

101.411-24/06u – without *Taq* polymerase

Lot No.: **3R6**

Olerup SSP® HLA-A*01

Product number: 101.411-24/06 – including *Taq* pol.
101.411-24u/06u – without *Taq* pol.

Lot number: 3R6

Expiry date: 2026-09-01

Number of tests: 24 tests – Product No. 101.411-24/24u
6 tests – Product No. 101.411-06/06u

Number of wells per test: 62+1

The HLA-A*01 primer set is unchanged compared to the previous lot (**5K8**).

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

A*01:01 to A*01:419, i.e. all the currently recognized HLA-A*01 alleles, will be amplified by the primers in the HLA-A*01 SSP kit^{1,2}; www.ebi.ac.uk/imgt/hla, 2022-July-12, release 3.49.0.

The HLA-A*01 kit enables separation of the confirmed HLA-A*01 alleles as listed in the IMGT/HLA database 3.24.0. An HLA allele is listed as confirmed by IMGT/HLA if it has been sequenced by more than a single laboratory or from multiple sources.

The HLA-A*01 kit also enables identification of many null and alternatively expressed alleles.

The following HLA-A*01 alleles can be distinguished by the different sizes of the HLA-specific PCR product:

Alleles	Primer mix	Alleles	Primer mix
A*01:31N, 01:173	25	A*01:53N-01:54	36
A*01:32, 01:104	26	A*01:65, 01:92	37
A*01:47, 01:49	35	A*01:87N, 01:141	27, 42
A*01:48, 01:123N	36	A*01:137, 01:150	10, 35
A*01:50, 01:62	38		

¹Alleles that have been deleted from or renamed in the official WHO HLA Nomenclature up to and including the last IMGT/HLA database release can be retrieved from web page <http://hla.alleles.org/alleles/deleted.html>.

²The A*01 primer set cannot separate the following alleles. These alleles can be distinguished by the HLA-A low resolution and/or HLA-A*25 kits.

Alleles

A*01:69:01-01:69:03, 25:59



0197

For *In Vitro* Diagnostic Use
MA125 v01 SSP Release Note Template
Date: September 2022, Rev. No: 00

HLA-A*01

101.411-24/06 – including *Taq* polymerase

101.411-24/06u – without *Taq* polymerase

Lot No.: **3R6**

RESOLUTION IN HLA-A*01 HOMO- AND HETEROZYGOTES:

The A*01:01,01:01, A*01:01,01:02 and A*01:02,01:02 genotypes give rise to unique amplification patterns with regard to other confirmed HLA-A*01 alleles.

INFLUENCE ON THE INTERPRETATION OF HLA-A*01 SUBTYPINGS BY NON-HLA-A*01 ALLELES:

None frequently occurring.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No suggestions received.



0197

For *In Vitro* Diagnostic Use

MA125 v01 SSP Release Note Template

Date: September 2022, Rev. No: 00