

101.842-12 – including *Taq* polymerase

101.842-12u – without *Taq* polymerase

Lot No.: **90X**

Olerup SSP® HLA-A*11 Add-on

Product number:	101.842-12 – including <i>Taq</i> polymerase 101.842-12u – without <i>Taq</i> polymerase
Lot number:	90X
Expiry date:	2017-August-01
Number of tests:	12
Number of wells per test:	2+1

CHANGES COMPARED TO THE PREVIOUS HLA-A*11 ADD-ON LOT (43S):

Well	5'-primer	3'-primer	rationale
3	-	-	Negative Control.

THE NUMBER OF WELLS is increased from 2 to 3 wells.

ALLELE COVERAGE:

The HLA-A*11:02 and 11:110 alleles give different patterns in the HLA-A*11 Add-on kit¹; www.ebi.ac.uk/imgt/hla, 2014-October-10, release 3.18.0.

The HLA-A*11 add-on kit cannot distinguish the silent mutations in the A*11:02:01-11:02:06 alleles.

¹Alleles that have been deleted from or renamed in the official WHO HLA Nomenclature up to and including the last IMGT/HLA database release can be retrieved from web page <http://hla.alleles.org/alleles/deleted.html>.

RESOLUTION IN HOMO- AND HETEROZYGOTES:

Excellent.

INFLUENCE ON THE INTERPRETATION OF HLA-A*11 ADD-ON SUBTYPINGS BY NON-HLA-A*11 ALLELES:

None of importance.