

Lot No.: **48S**

## **Olerup SSP<sup>®</sup> HLA-A\*23 Add-on**

**Product number:** 101.843-12 – including *Taq* polymerase  
101.843-12u – without *Taq* polymerase  
**Lot number:** 48S  
**Expiry date:** 2015-December-01  
**Number of tests:** 12  
**Number of wells per test:** 3

### **CHANGES COMPARED TO THE PREVIOUS HLA-A\*23 ADD-ON LOT (55N):**

Well	5'-primer	3'-primer	rationale
2	Added	Added	Primer pair added for the A*23:05 allele

**THE NUMBER OF WELLS** is 3 in an 8 well plate.

#### **ALLELE COVERAGE:**

The HLA-A\*23:01 and 23:17 alleles give different patterns in the HLA-A\*23 Add-on kit<sup>1</sup>; [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla), 2013-April-17, release 3.12.0.

<sup>1</sup>The HLA-A\*23 add-on kit cannot distinguish the silent mutations in the A\*23:01:01-23:01:11 alleles.

#### **RESOLUTION IN HOMO- AND HETEROZYGOTES:**

Excellent.

#### **INFLUENCE ON THE INTERPRETATION OF HLA-A\*23 ADD-ON SUBTYPINGS BY NON-HLA-A\*23 ALLELES:**

None of importance.

#### **MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:**

No comments received.