

## Release Note - Olerup SSP™ HLA-A\*32 – Lot No. Y13

Product number: 101.431-12 – licensed for PCR  
101.431-12u – not licensed for PCR  
Lot number: Y13  
Expiry date: 2009-July-01  
Number of tests: 12  
Number of tubes per test: 16

### CHANGES COMPARED TO THE PREVIOUS HLA-A\*32 LOT (V65):

Tube	5'-primer	3'-primer	rationale
13	Added	Added	Primer pair for the A*3215 allele.
14	New	New	New primer pair for the A*3212 allele.
15	New	New	New primer pair for the A*3213 allele.
16	New	New	New primer pair for the A*3214 allele.

**THE NUMBER OF WELLS** has been increased from 13 to 16.

### ALLELE COVERAGE:

A\*3201 to A\*3215, i.e. all the currently recognized HLA-A\*32 alleles, give rise to unique amplification patterns; [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla), 2007-July-09, release 2.18.0.

### RESOLUTION IN HLA-A\*32 HOMO- AND HETEROZYGOTES:

The A\*3201,3201 genotype gives rise to a unique amplification pattern.

### INFLUENCE ON THE INTERPRETATION OF HLA-A\*32 SUBTYPINGS BY NON-HLA-A\*32 ALLELES:

None frequently occurring.

### MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No suggestions received.