

101.435-06 – including *Taq* polymerase101.435-06u – without *Taq* polymeraseLot No.: **54X****Olerup SSP® HLA-A*69**

Product number: 101.435-06 – including *Taq* polymerase
101.435-06u – without *Taq* polymerase

Lot number: 54X

Expiry date: 2017-April-01

Number of tests: 6

Number of wells per test: 5+1

THE NUMBER OF WELLS is 6 in an 8 well plate.

ALLELE COVERAGE:

All the HLA-A*69 alleles, i.e. **A*69:01 to A*69:03**, recognized by the HLA Nomenclature Committee in July 2014¹ will be amplified by the primers in the HLA-A*69 SSP kit.

The HLA-A*69 kit enables separation of the confirmed HLA-A*69 alleles as listed in the IMGT/HLA database. An HLA allele is listed as confirmed by IMGT/HLA if it has been sequenced by more than a single laboratory or from multiple sources.

The HLA-A*69 kit also enables identification of polymorphisms in exons outside of the region encoding the peptide binding domain and of null and alternatively expressed alleles.

¹Alleles that have been deleted from or renamed in the official WHO HLA Nomenclature up to and including the last IMGT/HLA database release can be retrieved from web page <http://hla.alleles.org/alleles/deleted.html>.

RESOLUTION IN HLA-A*69 HOMO- AND HETEROZYGOTES:

Excellent.

INFLUENCE ON THE INTERPRETATION OF HLA-A*69 SUBTYPINGS BY NON-HLA-A*69 ALLELES:

None of importance