

## **Olerup SSP<sup>®</sup> HLA-B\*13**

**Product number:** 101.515-12 – including *Taq* polymerase  
101.515-12u - without *Taq* polymerase  
**Lot number:** 96E  
**Expiry date:** 2010-July-01  
**Number of tests:** 12  
**Number of wells per test:** 16

### **CHANGES COMPARED TO THE PREVIOUS HLA-B\*13 LOT (X63):**

<b>Well</b>	<b>5'-primer</b>	<b>3'-primer</b>	<b>rationale</b>
8	Added	Added	Primer pair added for the B*1318 allele.
10	Added	Added	Primer pair added for the B*1321 allele.
13	Added	Added	Primer pair added for the B*1319 allele.

**THE NUMBER OF WELLS** is unchanged.

#### **ALLELE COVERAGE:**

B\*1301 to B\*1321, i.e. all the currently recognized HLA-B\*13 alleles, give rise to unique amplification patterns; [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla), 2008-April-08, release 2.21.0.

#### **RESOLUTION IN HLA-B\*13 HOMO- AND HETEROZYGOTES:**

Good.

#### **INFLUENCE ON THE INTERPRETATION OF HLA-B\*13 SUBTYPINGS BY NON-HLA-B\*13 ALLELES:**

None frequently occurring.

#### **MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:**

No modifications made.