

101.516-24/04 – including *Taq* polymerase
101.516-24u/04u – without *Taq* polymerase

Lot No.: 1L5

Olerup SSP[®] HLA-B*15

Product number: 101.516-24/04 – including *Taq* pol.
 101.516-24u/04u – without *Taq* pol.

Lot number: 1L5

Expiry date: 2024-05-01

Number of tests: 24 tests – Product No. 101.516-24/24u
 4 tests – Product No. 101.516-04/04u

Number of wells per test: 95+1

CHANGES COMPARED TO THE PREVIOUS HLA-B*15 LOT (8G5):

The HLA-B*15 primer set is unchanged compared to the previous *Olerup SSP[®]* HLA-B*15 (Lot No. 8G5).

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

B*15:01 to B*15:558, i.e. all the currently recognized HLA-B*15 alleles, will be amplified by the primers in the HLA-B*15 subtyping kit¹; www.ebi.ac.uk/imgt/hla, 2020-January-20, release 3.39.0.

The HLA-B*15 kit enables separation of the confirmed HLA-B*15 alleles as listed in the IMGT/HLA database 3.25.0. An HLA allele is listed as confirmed by IMGT/HLA if it has been sequenced by more than a single laboratory or from multiple sources.

The HLA-B*15 kit also enables identification of null and alternatively expressed alleles.

The following HLA-B*15 alleles can be distinguished by the different sizes of the HLA-specific PCR product:

Alleles	Primer mix	Alleles	Primer mix
B*15:81, 15:430	82	B*15:166, 15:193	86
B*15:82, 15:256	36	B*15:169, 15:225	87
B*15:92, 15:104	52	B*15:202, 15:239, 15:477	90
B*15:120, 15:258N	93	B*15:206, 15:343	50
B*15:142, 15:147, 15:429	78	B*15:208, 15:230	70
B*15:146, 15:441	75	B*15:301, 15:302N	64

The HLA-B*15 primer set cannot separate the following alleles. These alleles can be distinguished by the HLA-B low resolution kit and/or the respective high resolution kits:

Alleles

B*15:180, 46:71
B*15:515, 35:43:01, 35:135, 35:185, 51:220

101.516-24/04 – including *Taq* polymerase
101.516-24u/04u – without *Taq* polymerase

Lot No.: **1L5**

¹Alleles that have been deleted from or renamed in the official WHO HLA Nomenclature up to and including the last IMGT/HLA database release can be retrieved from web page <http://hla.alleles.org/alleles/deleted.html>.

RESOLUTION IN HLA-B*15 HOMO- AND HETEROZYGOTES:

The B*15:01,15:01, genotype gives rise to a unique amplification pattern.

INFLUENCE ON THE INTERPRETATION OF HLA-B*15 SUBTYPINGS BY NON-HLA-B*15 ALLELES:

None frequently occurring.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.