

101.854-12 – including *Taq* polymerase

101.854-12u – without *Taq* polymerase

Lot No.: **0E1**

## **Olerup SSP® HLA-B\*15 Add-on**

<b>Product number:</b>	101.854-12 – including <i>Taq</i> polymerase 101.854-12u – without <i>Taq</i> polymerase
<b>Lot number:</b>	0E1
<b>Expiry date:</b>	2018-11-01
<b>Number of tests:</b>	12
<b>Number of wells per test:</b>	2+1

**THE NUMBER OF WELLS** is 3 in an 8 well plate.

### **ALLELE COVERAGE:**

The HLA-B\*15:01:01:02N and 15:01:01G alleles give different patterns in the HLA-B\*15 Add-on kit<sup>1</sup>; [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla), 2016-January-19, release 3.23.0.

The HLA-B\*15 Add-on kit cannot distinguish these silent mutations: the B\*15:01:01:01, 15:01:01:03-15:01:04 and 15:01:06-15:01:39 alleles.

<sup>1</sup>Alleles that have been deleted from or renamed in the official WHO HLA Nomenclature up to and including the last IMGT/HLA database release can be retrieved from web page <http://hla.alleles.org/alleles/deleted.html>.

### **RESOLUTION IN HOMO- AND HETEROZYGOTES:**

Excellent.

### **INFLUENCE ON THE INTERPRETATION OF B\*15 ADD-ON SUBTYPINGS BY NON-HLA-B\*15 ALLELES:**

None of importance.