

## **Olerup SSP<sup>®</sup> HLA-B\*18**

**Product number:** 101.519-12 – including *Taq* polymerase  
101.519-12u – without *Taq* polymerase  
**Lot number:** 7R0  
**Expiry date:** 2027-01-01  
**Number of tests:** 12  
**Number of wells per test:** 41+1

**CHANGES COMPARED TO THE PREVIOUS HLA-B\*18 LOT (8L6):**

Well	5'-primer	3'-primer	rationale
9	Removed	-	Redundant 5'-primer removed.

**THE NUMBER OF WELLS** is unchanged.

**ALLELE COVERAGE:**

B\*18:01 to B\*18:226, i.e. all the currently recognized HLA-B\*18 alleles, will be amplified by the primers in the HLA-B\*18 subtyping kit<sup>1</sup>; [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla), 2022-October-12 release 3.50.0.

The HLA-B\*18 kit enables separation of the confirmed HLA-B\*18 alleles as listed in the IMGT/HLA database 3.29.0. An HLA allele is listed as confirmed by IMGT/HLA if it has been sequenced by more than a single laboratory or from multiple sources.

The HLA-B\*18 kit also enables identification of many null and alternatively expressed alleles.

The following HLA-B\*18 alleles can be distinguished by the different sizes of the HLA-specific PCR product:

Alleles	Primer mix	Alleles	Primer mix
B*18:05:01:01-18:05:01:02, 18:60	6	B*18:36, 18:67	11
B*18:18:01:01-18:18:01:02, 18:47	29	B*18:37:02, 18:46	27
B*18:23N, 18:28	10	B*18:38, 18:45	28
B*18:34, 18:52	30	B*18:40, 18:51, 18:217	23

<sup>1</sup>Alleles that have been deleted from or renamed in the official WHO HLA Nomenclature up to and including the last IMGT/HLA database release can be retrieved from web page <http://hla.alleles.org/alleles/deleted.html>.

**RESOLUTION IN HLA-B\*18 HOMO- AND HETEROZYGOTES:**

Good.



**0197**

101.519-12 – including *Taq* polymerase

101.519-12u – without *Taq* polymerase

Lot No.: **7R0**

**INFLUENCE ON THE INTERPRETATION OF HLA-B\*18 SUBTYPINGS BY NON-HLA-B\*18 ALLELES:**

The B\*08:xx,18:01 and B\*08:xx,18:26 genotypes give rise to the same amplification patterns.

**MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:**

No comments received.



**0197**

For *In Vitro* Diagnostic Use

MA125 v01 SSP Release Note Template

Date: March 2023, Rev. No: 00