

Olerup SSP[®] HLA-B*27

Product number: 101.521-24/04 – including *Taq* pol.
101.521-24u/04u – without *Taq* pol.
Lot number: 09F
Expiry date: 2010-September-01
Number of tests: 24 test – Product No. 101.521-24
4 tests – Product No. 101.521-04

CHANGES COMPARED TO THE PREVIOUS HLA-B*27 LOT (X94):

Well	5'-primer	3'-primer	rationale
2	-	Added	Primer added for the B*2739 allele.
10	-	Added	Primer added for the B*2741 allele.
12	Added	Added	Primer pair added for the B*2743 allele.

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

B*2701 to B*2743, i.e. all the currently recognized HLA-B*27 alleles, give rise to unique amplification patterns; www.ebi.ac.uk/imgt/hla, 2008 July-11, release 2.22.0.

The HLA-B*27 subtyping kit cannot distinguish the B*270502, B*270503 and B*270505 to B*270510 alleles.

RESOLUTION IN HLA-B*27 HOMO- AND HETEROZYGOTES:

The B*2705,2705 genotype gives rise to a unique amplification pattern.

INFLUENCE ON THE INTERPRETATION OF HLA-B*27 SUBTYPINGS BY NON-HLA-B*27 ALLELES:

None frequently occurring.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.