

Olerup SSP[®] HLA-B*37

Product number: 101.541-06 – including *Taq* polymerase
101.541-06u – without *Taq* polymerase
Lot number: 98F
Expiry date: 2011-July-01
Number of tests: 6
Number of wells per test: 16

CHANGES COMPARED TO THE PREVIOUS HLA-B*37 LOT (53E):

Well	5'-primer	3'-primer	rationale
8	-	Exchanged	Exchanged 3'-primer for increased specificity of specific primer pair.
9	Exchanged	Exchanged	Exchanged primer pair for the B*3708 allele.
10	Removed, added	Removed, added	Primer pair for the B*3708 allele removed, new primer pair for the B*3714 allele.

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

B*3701 to B*3714, i.e. all the currently recognized HLA-B*37 alleles, give rise to unique amplification patterns; www.ebi.ac.uk/imgt/hla, 2009-May-11, release 2.25.2.

RESOLUTION IN HLA-B*37 HOMO- AND HETEROZYGOTES:

The B*3701,3701 genotype gives rise to a unique amplification pattern.

INFLUENCE ON THE INTERPRETATION OF HLA-B*37 SUBTYPINGS BY NON-HLA-B*37 ALLELES:

None frequently occurring.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No suggestions received.