

101.853-12 – including *Taq* polymerase

101.853-12u – without *Taq* polymerase

Lot No.: 8G0

Olerup SSP® HLA-B*39 Add-on

Product number:	101.853-12 – including <i>Taq</i> polymerase 101.853-12u – without <i>Taq</i> polymerase
Lot number:	8G0
Expiry date:	2021-03-01
Number of tests:	12
Number of wells per test:	2+1

The HLA-B*39 Add-on primer set is unchanged compared to the previous *Olerup SSP® HLA-B*39 Add-on (Lot No. 4E2)*.

THE NUMBER OF WELLS is 3 in an 8 well plate.

ALLELE COVERAGE:

The HLA-B*39:01 and 39:01:02:01L alleles give different patterns in the HLA-B*39 Add-on kit¹; www.ebi.ac.uk/imgt/hla, 2018-April-16, release 3.32.0.

¹Alleles that have been deleted from or renamed in the official WHO HLA Nomenclature up to and including the last IMGT/HLA database release can be retrieved from web page <http://hla.alleles.org/alleles/deleted.html>.

RESOLUTION IN HOMO- AND HETEROZYGOTES:

Excellent.

INFLUENCE ON THE INTERPRETATION OF B*39 ADD-ON SUBTYPINGS BY NON-HLA-B*39

ALLELES:

None of importance.