

Olerup SSP® HLA-B*42

Product number: 101.543-06 – including *Taq* polymerase
101.543-06u – without *Taq* polymerase
Lot number: 62K
Expiry date: 2013-April-01
Number of tests: 6
Number of wells per test: 15

CHANGES COMPARED TO THE PREVIOUS HLA-B*42 LOT (96F):

Well	5'-primer	3'-primer	rationale
10	-	-	Exchanged positive control primer pair.
12	New	New	New primer pair for the HLA-B*42:11 allele.
13	New	New	New primer pair for the HLA-B*42:12 allele.
14	New	New	New primer pair for the HLA-B*42:13 allele.
15	New	New	New primer pair for the HLA-B*42:14 allele.

THE NUMBER OF WELLS has been increased from 11 to 15.

ALLELE COVERAGE:

B*42:01 to B*42:14, i.e. all the currently recognized HLA-B*42 alleles, give rise to unique amplification patterns; www.ebi.ac.uk/imgt/hla, 2010-July-16, release 3.1.0.

RESOLUTION IN HLA-B*42 HOMO- AND HETEROZYGOTES:

Good.

INFLUENCE ON THE INTERPRETATION OF HLA-B*42 SUBTYPINGS BY NON-HLA-B*42 ALLELES:

None frequently occurring.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.