

101.543-06 – including *Taq* polymerase101.543-06u – without *Taq* polymeraseLot No.: **57M****Olerup SSP® HLA-B\*42**

Product number: 101.543-06 – including *Taq* polymerase  
101.543-06u – without *Taq* polymerase  
Lot number: 57M  
Expiry date: 2014-February-01  
Number of tests: 6  
Number of wells per test: 15

**CHANGES COMPARED TO THE PREVIOUS HLA-B\*42 LOT (62K):**

Well	5'-primer	3'-primer	rationale
7	Added	-	5'-primer added for the HLA-B*42:16 allele
8	-	Added	Increased yield of HLA-specific PCR product.
10	-	-	Exchanged positive control primer pair.
14	Added	Added	Primer pair added for the HLA-B*42:15 allele.

**THE NUMBER OF WELLS** is unchanged.

**ALLELE COVERAGE:**

B\*42:01 to B\*42:16, i.e. all the currently recognized HLA-B\*42 alleles, give rise to unique amplification patterns; [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla), 2011-April-08, release 3.4.0.

**RESOLUTION IN HLA-B\*42 HOMO- AND HETEROZYGOTES:**

Good.

**INFLUENCE ON THE INTERPRETATION OF HLA-B\*42 SUBTYPINGS BY NON-HLA-B\*42 ALLELES:**

None frequently occurring.

**MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:**

No comments received.