

101.563-24/03 – including *Taq* polymerase
101.563-24u/03u – without *Taq* polymerase

Lot No.: **73F**

Olerup SSP® HLA-B*44

Product number: 101.563-24/03 – including *Taq* pol.
101.563-24u/03u - without *Taq* pol.
Lot number: 73F
Expiry date: 2011-March-01
Number of tests: 24 tests – Product No. 101.563-24
3 tests – Product No. 101.563-03
Number of wells per test: 48

CHANGES COMPARED TO THE PREVIOUS HLA-B*44 LOT (34E):

Well	5'-primer	3'-primer	rationale
10	Moved	Moved	Primer pair moved to well 27.
27	Added	Added	Primer pair from well 10.
35	Added	Added	Primer pair for the B*4463 allele.
46	Added	Added	Primer pair for the B*4465 allele.
48	Exchanged, added	Added	Decreased formation of primer dimers, new primer pair for the B*4461N allele.

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

B*4401 to B*4465, i.e. all the currently recognized HLA-B*44 alleles, give rise to unique amplification patterns; www.ebi.ac.uk/imgt/hla, 2009-January-16, release 2.24.0.

RESOLUTION IN HLA-B*44 HOMO- AND HETEROZYGOTES:

The B*4403,4403 and B*4403,4460 genotypes give rise to identical amplification pattern.

INFLUENCE ON THE INTERPRETATION OF HLA-B*44 SUBTYPINGS BY NON-HLA-B*44 ALLELES:

None frequently occurring.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.