

101.545-06 – including *Taq* polymerase101.545-06u – without *Taq* polymeraseLot No.: **68M****Olerup SSP[®] HLA-B*47**

Product number: 101.545-06 – including *Taq* polymerase
101.545-06u – without *Taq* polymerase
Lot number: 68M
Expiry date: 2014-March-01
Number of tests: 6
Number of wells per test: 8

CHANGES COMPARED TO THE PREVIOUS HLA-B*47 LOT (10K):

Well	5'-primer	3'-primer	rationale
4	Added	Added	Primer pair added for the B*47:08 allele, exchanged positive control primer pair.
8	Added	Added	Primer pair added for the B*47:08 allele.

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

B*47:01 to B*47:08, i.e. all the currently recognized HLA-B*47 alleles, give rise to unique amplification patterns; www.ebi.ac.uk/imgt/hla, 2011-July-14, release 3.5.0.

RESOLUTION IN HLA-B*47 HOMO- AND HETEROZYGOTES:

Excellent.

INFLUENCE ON THE INTERPRETATION OF HLA-B*47 SUBTYPINGS BY NON-HLA-B*47 ALLELES:

None frequently occurring.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.