

Olerup SSP[®] HLA-B*50

Product number: 101.548-06 – including *Taq* polymerase
101.548-06u – without *Taq* polymerase
Lot number: 49K
Expiry date: 2012-September-01
Number of tests: 6
Number of wells per test: 12

CHANGES COMPARED TO THE PREVIOUS HLA-B*50 LOT (89F):

Well	5'-primer	3'-primer	rationale
8	New	New	New primer pair for the B*50:05 allele.
9	New	New	New primer pair for the B*50:06 allele.
10	New	New	New primer pair for the B*50:07 allele.
11	New	New	New primer pair for the B*50:08 allele.
12	New	New	New primer pair for the B*50:09 allele.

THE NUMBER OF WELLS has been increased from 7 to 12.

ALLELE COVERAGE:

B*50:01 to B*50:09, i.e. all the currently recognized HLA-B*50 alleles, give rise to unique amplification patterns; www.ebi.ac.uk/imgt/hla, 2010-July-16, release 3.1.0.

RESOLUTION IN HLA-B*50 HOMO- AND HETEROZYGOTES:

Very good.

INFLUENCE ON THE INTERPRETATION OF HLA-B*50 SUBTYPINGS BY NON-HLA-B*50 ALLELES:

None frequently occurring.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.