

101.562-06 – including *Taq* polymerase101.562-06u – without *Taq* polymeraseLot No.: **90E****Olerup SSP® HLA-B\*52**

**Product number:** 101.562-06 – including *Taq* polymerase  
101.562-06u – without *Taq* polymerase  
**Lot number:** 90E  
**Expiry date:** 2010-June-01  
**Number of tests:** 6  
**Number of wells per test:** 15

**CHANGES COMPARED TO THE PREVIOUS HLA-B\*52 LOT (Y31):**

Well	5'-primer	3'-primer	rationale
4	Exchanged	Exchanged	Exchanged primer pair for less primer dimer formation tendency.
15	New	New	New primer pair for the B*5212 allele.

**THE NUMBER OF WELLS** has been increased from 14 to 15.

**ALLELE COVERAGE:**

B\*5201 to B\*5212, i.e. all the currently recognized HLA-B\*52 alleles, give rise to unique amplification patterns; [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla), 2008-April-08, release 2.21.0.

**RESOLUTION IN HLA-B\*52 HOMO- AND HETEROZYGOTES:**

The B\*5201,5201 genotype gives rise to a unique amplification pattern.

**INFLUENCE ON THE INTERPRETATION OF HLA-B\*52 SUBTYPINGS BY NON-HLA-B\*52 ALLELES:**

None frequently occurring.

**MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:**

No suggestions received.