

Lot No.: **18L**

Olerup SSP[®] HLA-B*57

Product number: 101.567-12 – including *Taq* polymerase
101.567-12u - without *Taq* polymerase
Lot number: 18L
Expiry date: 2013-August-01
Number of tests: 12
Number of wells per test: 24

CHANGES COMPARED TO THE PREVIOUS HLA-B*57 LOT (66G):

Well	5'-primer	3'-primer	rationale
6	-	Added	Primer added for the B*57:30 allele.
15	Added	-	Primer added for the B*57:33 allele.
17	-	Added	Primer added for the B*57:31 allele.
21	Added	-	Primer added for the B*57:33 allele.
22	New	New	New primer pair for the B*57:34 allele.
23	New	New	New primer pairs for the B*57:35 and B*57:36 alleles.
24	New	New	New primer pair for the B*57:37 allele.

THE NUMBER OF WELLS has been increased from 21 to 24.

ALLELE COVERAGE:

B*57:01 to B*57:37, i.e. all the currently recognized HLA-B*57 alleles, give rise to unique amplification patterns; www.ebi.ac.uk/imgt/hla, 2010-October-15, release 3.2.0.

RESOLUTION IN HLA-B*57 HOMO- AND HETEROZYGOTES:

The B*57:01,57:01 genotype give rise to a unique amplification pattern.

INFLUENCE ON THE INTERPRETATION OF HLA-B*57 SUBTYPINGS BY NON-HLA-B*57 ALLELES:

None frequently occurring.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.