

Olerup SSP[®] HLA-B*57

Product number: 101.567-12 – including *Taq* polymerase
101.567-12u - without *Taq* polymerase
Lot number: 31E
Expiry date: 2010-February-01
Number of tests: 12
Number of wells per test: 16

CHANGES COMPARED TO THE PREVIOUS HLA-B*57 LOT (Y36):

Well	5'-primer	3'-primer	rationale
6	Added	Added	Primer pair moved from vial 7 to vial 6.
7	New	New	New primer pair for the B*5715 allele.
8	Added	Added	Primer pair added for the B*5716 allele.
11	-	Removed	Primer removed for increased resolution.

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

B*5701 to B*5716, i.e. all the currently recognized HLA-B*57 alleles, give rise to unique amplification patterns; www.ebi.ac.uk/imgt/hla, 2008-January-11, release 2.20.0.

RESOLUTION IN HLA-B*57 HOMO- AND HETEROZYGOTES:

The B*5701,5701 genotype give rise to a unique amplification pattern.

INFLUENCE ON THE INTERPRETATION OF HLA-B*57 SUBTYPINGS BY NON-HLA-B*57 ALLELES:

None frequently occurring.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.