

Olerup SSP® HLA-B*57:01

Product number: 101.572-12 – including *Taq* polymerase
101.572-12u - without *Taq* polymerase
Lot number: 02S
Expiry date: 2015-August-01
Number of tests: 12
Number of wells per test: 15+1

CHANGES COMPARED TO THE PREVIOUS HLA-B*57:01 LOT (14R):

Well	5'-primer	3'-primer	rationale
12	-	Added	3'-primers added for the B*57:60 allele and for improved allelic resolution.
13	Added	-	5'-primer added for the B*57:58 allele.
14	Added	-	5'-primer added for the B*57:59 allele.

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

B*57:01:01 to B*57:01:14, i.e. all the currently recognized HLA-B*57:01 alleles, gives rise to a unique amplification pattern; www.ebi.ac.uk/imgt/hla, 2012-October-17, release 3.10.0.

The HLA-B*57:01 typing kit cannot distinguish the silent mutations in the B*57:01:01 to B*57:01:14 alleles.

RESOLUTION IN HLA-B*57:01 HOMO- AND HETEROZYGOTES:

Good.

INFLUENCE ON THE INTERPRETATION OF HLA-B*57:01 SUBTYPINGS BY NON-HLA-B*57 ALLELES:

None of importance.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.