

Olerup SSP® HLA-B*73

Product number:	101.555-06 – including <i>Taq</i> polymerase 101.555-06u – without <i>Taq</i> polymerase
Lot number:	92V
Expiry date:	2016-November-01
Number of tests:	6
Number of wells per test:	3+1

CHANGES COMPARED TO THE PREVIOUS HLA-B*73 LOT (11R):

Well	5'-primer	3'-primer	rationale
4	-	-	Negative Control.

THE NUMBER OF WELLS is increased from 3 to 4 wells.

ALLELE COVERAGE:

B*73:01 to B*73:02, i.e. all the currently recognized HLA-B*73 alleles, give rise to unique amplification patterns; www.ebi.ac.uk/imgt/hla, 2014-April-14, release 3.16.0.

The HLA-B*73 kit enables separation of the confirmed HLA-B*73 alleles as listed in the IMGT/HLA database. An HLA allele is listed as confirmed by IMGT/HLA if it has been sequenced by more than a single laboratory or from multiple sources.

The HLA-B*73 kit also enables identification of polymorphisms in exons outside of the region encoding the peptide binding domain and of null and alternatively expressed alleles

RESOLUTION IN HLA-B*73 HOMO- AND HETEROZYGOTES:

Excellent.

INFLUENCE ON THE INTERPRETATION OF HLA-B*73 SUBTYPINGS BY NON-HLA-B*73 ALLELES:

None.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.