

101.552-06 – including *Taq* polymerase101.552-06u – without *Taq* polymerase

Lot No.: 51V

Olerup SSP® HLA-B*82

Product number: 101.552-06 – including *Taq* pol.
101.552-06u – without *Taq* pol.
Lot number: 51V
Expiry date: 2016-August-01
Number of tests: 6
Number of wells per test: 5+1

CHANGES COMPARED TO THE PREVIOUS HLA-B*82 LOT (58R):

Well	5'-primer	3'-primer	rationale
6	-	-	Negative Control.

THE NUMBER OF WELLS is increased from 5 to 6 wells.

ALLELE COVERAGE:

B*82:01 to B*82:03, i.e. all the currently recognized HLA-B*82 alleles, give rise to unique amplification patterns; www.ebi.ac.uk/imgt/hla, 2014-January-17, release 3.15.0.

RESOLUTION IN HLA-B*82 HOMO- AND HETEROZYGOTES:

Excellent.

INFLUENCE ON THE INTERPRETATION OF HLA-B*82 SUBTYPINGS BY NON-HLA-B*82**ALLELES:**

None of importance.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No suggestions received.