

101.622-12 – including *Taq* polymerase

101.622-12u – without *Taq* polymerase

Lot No.: **99V**

Olerup SSP® HLA-C*02

Product number: 101.622-12 – including *Taq* polymerase
 101.622-12u – without *Taq* polymerase

Lot number: 99V

Expiry date: 2016-December-01

Number of tests: 12

Number of wells per test: 31+1

CHANGES COMPARED TO THE PREVIOUS HLA-C*02 LOT (95R):

Well	5'-primer	3'-primer	rationale
4	Added	Added	Primer pair added for the C*02:56 allele.
6	Added	-	5'-primer added for the C*02:64 allele.
13	-	Added	3'-primer added for the C*02:49 allele.
15	-	Added	3'-primer added for the C*02:71 allele.
20	Added	Added	Primer pair added for the C*02:65 allele, 3'-primers added for the C*02:71 and C*02:72 alleles.
21	Added	-	5'-primers added for the C*02:64 and C*02:67Q alleles.
22	Added	Added	Primer pair added for the C*02:56 allele.
23	-	Added	3'-primers added for the C*02:69 and C*02:70 alleles.
25	Added	-	5'-primer added for the C*02:60 allele.
26	-	Added	3'-primer added from well 32.
27	-	Added	3'-primer added from well 32.
28	Added	-	5'-primers added for the C*02:60 and C*02:67 alleles.
32	Removed	Moved	3'-primers moved to wells 26 and 27.

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

C*02:02 to C*02:81, i.e. all the currently recognized HLA-C*02 alleles, will be amplified by the primers in the HLA-C*02 SSP kit¹; www.ebi.ac.uk/imgt/hla, 2014-April-14, release 3.16.0.

The HLA-C*02 kit enables separation of the confirmed HLA-C*02 alleles as listed in the IMGT/HLA database. An HLA allele is listed as confirmed by IMGT/HLA if it has been sequenced by more than a single laboratory or from multiple sources.

The HLA-C*02 kit also enables identification of polymorphisms in exons outside of the region encoding the peptide binding domain and of null and alternatively expressed alleles

101.622-12 – including *Taq* polymerase

101.622-12u – without *Taq* polymerase

Lot No.: 99V

The following HLA-C*02 alleles can be distinguished by the different sizes of the HLA-specific PCR product:

Alleles	Primer mix	Alleles	Primer mix
C*02:05:02-02:05:03, 02:22	5	C*02:25Q, 02:30	21
C*02:15, 02:21	15	C*02:35, 02:69, 02:70	23
C*02:24, 02:65	20	C*02:37, 02:52N	28

The HLA-C*02 primer set cannot distinguish the following silent mutations: the C*02:02:01-02:02:03, 02:02:06-02:02:12 and 02:02:15-02:02:25 alleles, the C*02:02:05 and 02:02:14 alleles, the C*02:05:02-02:05:03 alleles or the C*02:26:01-02:26:03 alleles.

¹Alleles that have been deleted from or renamed in the official WHO HLA Nomenclature up to and including the last IMGT/HLA database release can be retrieved from web page <http://hla.alleles.org/alleles/deleted.html>.

RESOLUTION IN HLA-C*02 HOMO- AND HETEROZYGOTES:

The C*02:02,02:02 and C*02:02,02:10 genotypes give rise to identical amplification patterns.

INFLUENCE ON THE INTERPRETATION OF HLA-C*02 SUBTYPINGS BY NON-HLA-C*02 ALLELES:

None frequently occurring.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.