

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	
HLA-C allele^{1,2,3}																																						
*02:02:01-02:02:03, 02:02:06-02:02:12, 02:02:15-02:02:25, 02:02:27-02:02:30, 02:48, 02:50, 02:54, 02:57, 02:59, 02:61-02:63, 02:66, 02:74, 02:77-02:80, 02:82, 02:84-02:86, 02:88, 02:90-02:91, 02:93, 02:96, 02:98-02:100, 02:102-02:104	1	2							9	10																												
*02:02:05, 02:02:13-02:02:14, 02:51, 02:73, 02:76, 02:95	1								9	10																												
*02:02:26	1	2							9																													
*02:03			3						9	10					16		19																					
*02:04	1	2		4					9	10																												
*02:05:01	1				5				9	10																												
*02:05:02-02:05:03, 02:22 ⁵	1	2			5				9	10																												
*02:06	1					6			9	10														24														
*02:07, 02:97	1	2					7		9	10																												
*02:08	1	2						8	9	10																												
*02:09	1	2							9	10	11																											
*02:10, 02:89, 06:18 ⁴	1									10																												
*02:11	1	2							9	10		12																										
*02:12	1								9	10			13															29										
*02:13	1	2							9	10				14																								
*02:14:01-02:14:02	1									10		12																										
*02:15, 02:21 ⁵	1	2							9	10					15																							
*02:16:01									9	10						16																						
*02:16:02			3						9	10						16																						
*02:17	1				5				9	10		12																										
*02:18			3						9	10	11				16																							
*02:19	1	2							9	10															25													
*02:20	1	2	3						9	10																												
*02:23	1	2							9	10							17							24	25													
*02:24	1	2							9	10											20																	
*02:25Q, 02:30 ⁵	1	2							9	10												21																
*02:26:01-02:26:03	1	2							9								18																					
*02:27:01	1	2							9	10			13																									36
*02:27:02	1								9	10			13																									36
*02:28	1	2							9	10								19																				
*02:29	1	2								10													23															
*02:31	1	2	3						9	10							17																					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	

Negative Control

*02:32								9	10	11																														
*02:33	1						8	9	10																															
*02:34	1	2						9	10											22																				
*02:35, 02:69-02:70 ⁵	1	2						9	10											23																				
*02:36, 02:68	1	2						9	10													24																		
*02:37, 02:52N ⁵	1	2						9	10																															
*02:38N	1	2						9	10																															
*02:39	1	2						9																																
*02:40:01	1	2						9	10																															
*02:40:02	1							9	10																															
*02:42	1	2						9	10																															
*02:43	1	2						9	10				14		17																									
*02:44	1	2						9	10																															
*02:45	1	2						9																																
*02:46	1	2					6	9	10																															
*02:47	1	2					6	9	10																															
*02:49	1	2						9	10				13																											
*02:53:01-02:53:02	1	2						9	10																															
*02:55	1	2						9	10																															
*02:56	1	2				4		9	10																															
*02:58	1								10																															
*02:60	1	2						9	10																															
*02:64	1	2					6	9	10																															
*02:65	1	2						9	10																															
*02:67Q	1	2						9	10																															
*02:71	1	2						9	10					15																										
*02:72								9	10																															
*02:75	1	2						9	10				13																											
*02:81	1	2					5	9	10																															
*02:83	1	2						9	10																															
*02:87	1	2						9																																
*02:92N	1	2						9	10																															
*02:94	1							9	10																															
*02:101	1	2						9	w																															
*02:105N	1	2						9	10																															
*02:106	1	2						9	10																															
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37			
*01:02:34, 04:140, 04:166, 08:103, B*67:02																																								

Negative Control

*01:04, 04:94:01-04:94:02, 06:02:09, 06:02:21, 06:11, 06:33, 06:82, 06:104, 06:119-06:120, 06:125, 06:147, 08:27, 08:29, 12:02:01-12:03:16, 12:03:18-12:03:22, 12:03:24-12:03:37, 12:06-12:08, 12:10:01-12:13, 12:17, 12:22-12:23, 12:25-12:32, 12:34-12:40, 12:42Q-12:59, 12:61-12:71, 12:73-12:82, 12:84N, 12:86-12:105N, 12:107-12:112, 12:114, 12:116-12:118, 12:120-12:121, 12:123, 12:125-12:134, 12:136, 12:138-12:145, 12:148N-12:152, 12:155Q-12:161, 14:16, 16:04:01, 16:04:03, 16:33, 16:42, 16:55, 16:61, 16:66, 16:78, 16:82	1																																														
*01:09 *01:10, 16:50, B*07:02:07, B*07:239, B*14:46, B*27:05:15, B*40:243, B*48:04:02	1																				25																										
*01:21 *01:22, 04:04:01-04:04:02, 04:13, 04:58, 04:122, 08:01:01-08:01:19, 08:03:01-08:04:03, 08:06, 08:08:01-08:09, 08:11, 08:13, 08:16:01-08:16:02, 08:20-08:22, 08:26N, 08:36N, 08:38-08:40, 08:42, 08:44, 08:46, 08:50, 08:54, 08:56-08:61, 08:65-08:66, 08:72:01-08:72:02, 08:78-08:79, 08:81-08:89N, 08:91, 08:93, 08:95-08:99, 08:101-08:102, 08:104-08:106, 08:109, 08:117, 08:121N-08:122, 12:14:01-12:14:02, 12:18:01-12:18:02, 12:20, 14:06, 14:15, 14:53, 16:35, 16:40, 17:01:09, 17:22	1																																														
*01:35, 04:34, 08:41, 12:83 *01:90, 01:101-01:102, 03:81, 03:175, 03:199, 03:245, 04:108, 07:123, 07:173, 07:294, 16:20																																															
*01:107, 04:08, 08:115, 14:20 *03:03:22, 03:04:34																																															
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37										

Negative Control

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37		
*05:11, 05:17, 05:68, 05:70, 05:79, 05:115, 06:04, 06:118, 06:153, 08:10, 15:02:18, 15:02:22, 15:04:01-15:04:02, 15:30, 15:40, 15:64-15:66, 15:76, 15:105Q, 15:110, 16:48, 17:01:01-17:01:08, 17:01:10-17:06, 17:08- 17:16, 17:18-17:20, 17:23-17:30																16	18																						
*05:26, 07:272, 07:326										10																													
*05:27, 15:77, 17:07																16	18																						
*05:39, 18:08																																							
*05:98																		18																					
*05:105					5													18																					
*05:106	1	2												14			17	18																					
*06:05												w						18																					
*06:08	1				5													18																					
*06:81	1																	18										28											
*06:89	1																	18						24															
*06:96	1																	18																					
*06:107	1																	18							25														
*07:02:09, 16:80												12																											
*07:101, 07:148, 07:161, A*11:166, A*30:56, A*31:85, A*80:01:01:01-80:03, B*18:96, B*44:157, B*55:74																																							
*08:14, 08:80																		16																					
*08:24																		16											29										
*08:31, 12:119, 16:29	1				5																																		
*08:37, 16:21, B*27:84												12																											
*08:119				4														16																					
*12:03:17	1	2																14		17																			
*12:03:23	1							8																															
*12:15	1				6																			24															
*12:16, 12:147	1											12																											
*12:106, 12:122	1																																						
*12:113	1																							24															
*12:115	1			4																																			
*12:124	1																																						
*12:162	1				6																																		
*14:25					5							12																											
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37		

Negative Control

