

Olerup SSP[®] HLA-C*07 Add-on

Product number:	101.863-12 – including <i>Taq</i> polymerase 101.863-12u – without <i>Taq</i> polymerase
Lot number:	26N
Expiry date:	2014-August-01
Number of tests:	12
Number of wells per test:	5

THE NUMBER OF WELLS is 5 in an 8 well plate.

ALLELE COVERAGE:

The HLA-C*07:06 and C*07:18 and the C*07:01, the C*07:50 and the C*07:02 and the C*07:11 and the C*07:04 alleles are uniquely separated from each other in the HLA-C*07 Add-on kit¹; www.ebi.ac.uk/imgt/hla, 2012-January-12, release 3.7.0.

¹The HLA-C*07 Add-on kit cannot distinguish the silent mutations in the C*07:01:01-07:01:24 alleles, the 07:02:01:01-07:02:29 alleles or the C*07:04:01-C*07:04:07 alleles.

RESOLUTION IN HOMO- AND HETEROZYGOTES:

Good.

INFLUENCE ON THE INTERPRETATION OF HLA-C*07 ADD-ON SUBTYPINGS BY NON-HLA-C*07 ALLELES:

None frequently occurring.