

Olerup SSP® HLA-C*08 Add-on

Product number:	101.864-12 – including <i>Taq</i> polymerase 101.864-12u – without <i>Taq</i> polymerase
Lot number:	6E7
Expiry date:	2019-05-01
Number of tests:	12
Number of wells per test:	4+1

CHANGES COMPARED TO THE PREVIOUS HLA-C*08 ADD-ON LOT (85X):

Well	5'-primer	3'-primer	rationale
4	-	Modified	3'-primer modified for improved HLA-specific amplification.
5	-	-	Updated Negative Control.

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

The HLA-C*08:01, 08:22, 08:99 and 08:102 alleles give different amplification patterns in the HLA-C*08 Add-on kit¹; www.ebi.ac.uk/imgt/hla, 2016-July-14, release 3.25.0.

The HLA-C*08 Add-on kit cannot distinguish the silent mutations in the C*08:01:01-08:01:19 alleles.

¹Alleles that have been deleted from or renamed in the official WHO HLA Nomenclature up to and including the last IMGT/HLA database release can be retrieved from web page <http://hla.alleles.org/alleles/deleted.html>.

RESOLUTION IN HOMO- AND HETEROZYGOTES:

Good.

INFLUENCE ON THE INTERPRETATION OF HLA-C*08 ADD-ON SUBTYPINGS BY NON-HLA-C*08 ALLELES:

None frequently occurring.